


Rogue2 / Rogue XP Caster Mount

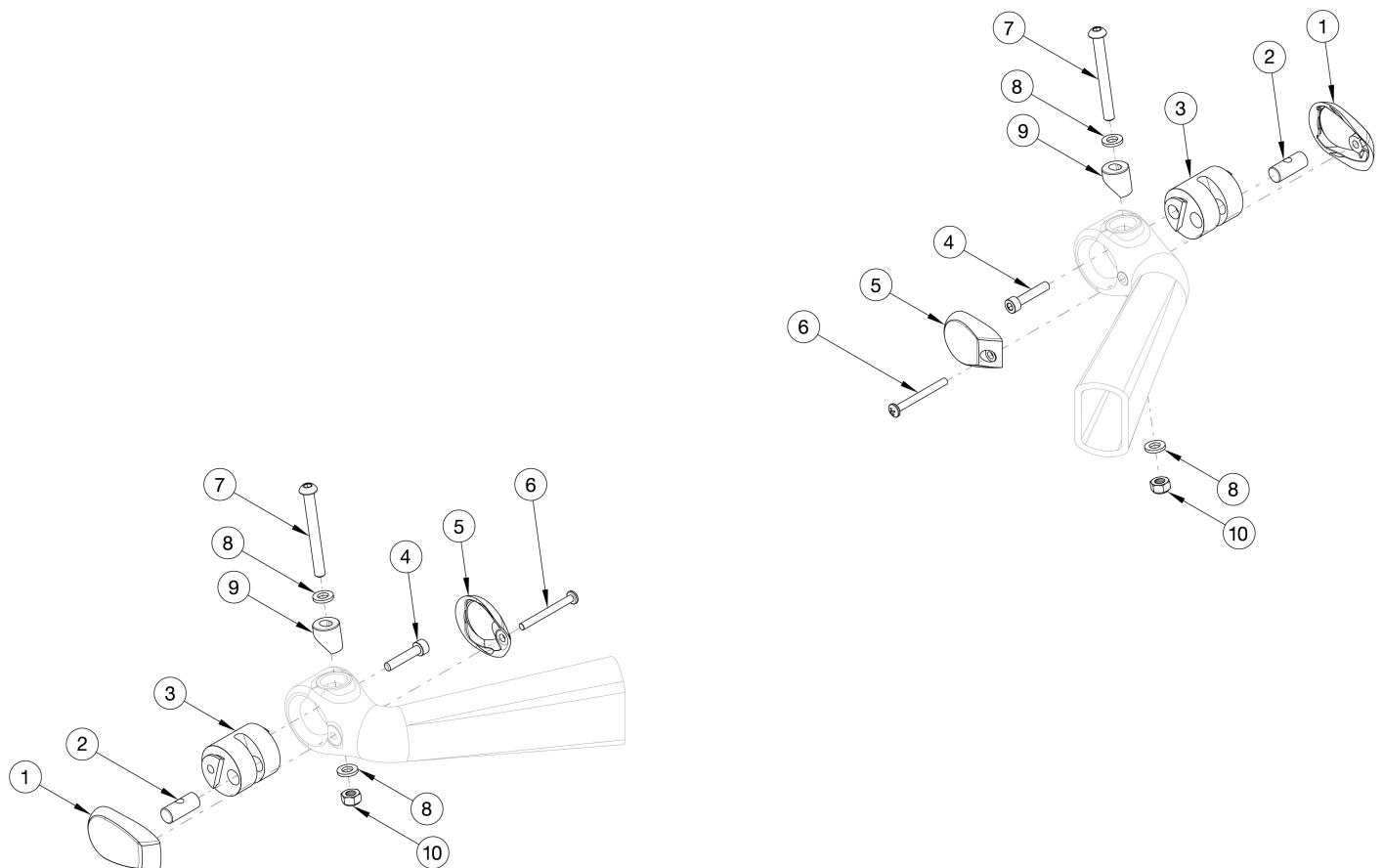

Used on Rogue XP chairs invoiced starting on 3/12/25

Item Number	Part Number	Description	Quantity
1, 2, 3a, 4-10	117593	Caster Wing Mechanism, Right, Rogue2 / Rogue XP	1
1, 2, 3b, 4-10	117594	Caster Wing Mechanism, Left, Rogue2 / Rogue XP	1
1, 2, 3a, 3b, 4-10	117595	Caster Wing Mechanism, Pair, Rogue2 / Rogue XP	1
1	115762	Caster Wing Cover - Outer	2
2	113987	Caster Lock Threaded Barrel	2
3a	114025	Caster Lock Barrel, Right	1
3b	114026	Caster Lock Barrel, Left	1
4	114044	Shcs, M5X0.8X25, Blk Oxide, Stainless Steel	2
5	113982	Caster Wing Cover - Inner	2

Item Number	Part Number	Description	Quantity
6	114024	Phms, M4X0.7 X 40, Phl, Black Ox, 18-8, Din 7985	2
7	113983	BHCS, M6 x1.0 x 55, Black Oxide, 18-8, Iso 7380	2
8	114043	Washer, Flat, M6, Blk Oxide, Stainless Steel	4
9	113985	Caster Lock Saddle	2
10	114023	Nut, Hex, M6X1.0, Nylok, Stainless Steel	2