


Catalyst E Armrests - Height Adjustable Flip Back T-Arm

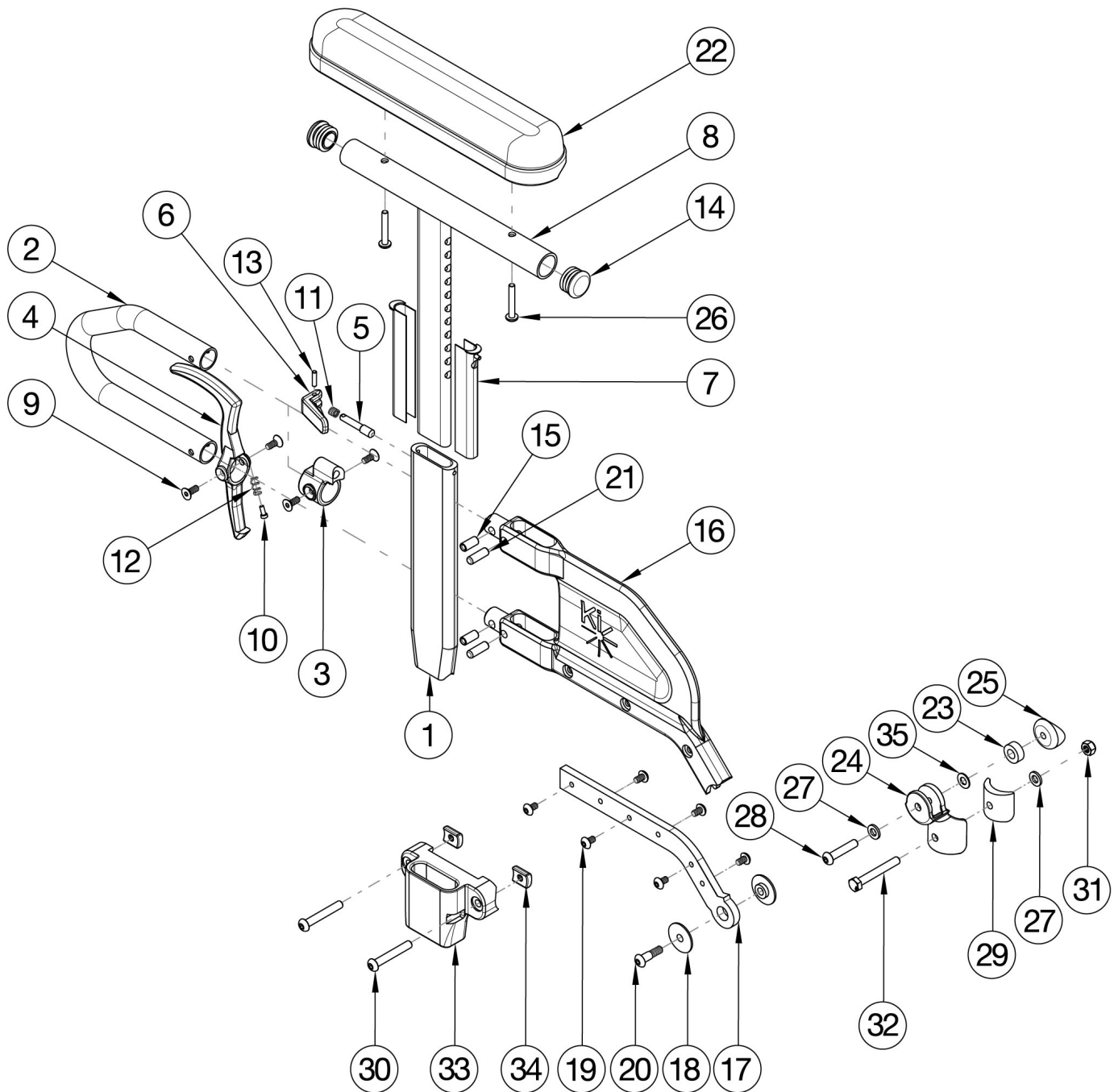

Short assemblies used with 14" - 17" Seat Depths. Long assemblies used with 18" - 20" Seat Depths.

| Item Number | Part Number | Description | Quantity | Retail |
|---------------------------------|-------------|--|----------|---------|
| 1-7, 9-13, 15, 16a, 17a, 19, 21 | 004158 | Height Adjustable Flip Back T-Arm Lower Assembly Short <i>Does not include t-arm upper, arm pad or arm pad screws</i> | 1 | \$59.09 |

| Item Number | Part Number | Description | Quantity | Retail |
|--|-------------|---|----------|----------|
| 1-7, 9-13, 15, 16b, 17b, 19, 21 | 004159 | Height Adjustable Flip Back T-Arm Lower Assembly Long <i>Does not include t-arm upper, arm pad or arm pad screws</i> | 1 | \$59.09 |
| 1-7, 8a, 9-15, 16a, 17a, 19, 21 | 003459 | Height Adjustable Flip Back T-Arm Short Desk <i>Does not include arm pad or arm pad screws</i> | 1 | \$137.90 |
| 1-7, 8b, 9-15, 16a, 17a, 19, 21 | 003460 | Height Adjustable Flip Back T-Arm Short Full <i>Does not include arm pad or arm pad screws</i> | 1 | \$137.90 |
| 1-7, 8a, 9-15, 16b, 17b, 19, 21 | 003461 | Height Adjustable Flip Back T-Arm Long Desk <i>Does not include arm pad or arm pad screws</i> | 1 | \$137.90 |
| 1-7, 8b, 9-15, 16b, 17b, 19, 21 | 003462 | Height Adjustable Flip Back T-Arm Long Full <i>Does not include arm pad or arm pad screws</i> | 1 | \$137.90 |
| 1-7, 9-13, 15, 16a, 17a, 18-21, 24, 25, 31, 32 | 504567 | Height Adjustable Flip Back T-Arm Short Retrofit Assembly, Catalyst 5 <i>This kit is used to retrofit from a Height Adjustable T-Arm to a Height Adjustable Flip Back T-Arm. This kit is to retrofit one side.</i> | 1 | \$209.59 |
| 1-7, 9-13, 15, 16b, 17b, 18-21, 24, 25, 31, 32 | 504568 | Height Adjustable Flip Back T-Arm Long Retrofit Assembly, Catalyst 5 <i>This kit is used to retro fit from a Height Adjustable T-Arm to a Height Adjustable Flip Back T-Arm. This kit is to retrofit one side.</i> | 1 | \$209.59 |
| 1-7, 8a, 9-15, 16a, 17a, 18-21, 22b, 24-26, 31, 32 | 504569 | Height Adjustable Flip Back T-Arm Short Desk with Standard Pad Assembly, Catalyst 5 | 1 | \$209.59 |
| 1-7, 8b, 9-15, 16a, 17a, 18-21, 22a, 24-26, 31, 32 | 504570 | Height Adjustable Flip Back T-Arm Short Full with Standard Pad Assembly, Catalyst 5 | 1 | \$209.59 |
| 1-7, 8a, 9-15, 16b, 17b, 18-21, 22b, 24-26, 31, 32 | 504571 | Height Adjustable Flip Back T-Arm Long Desk with Standard Pad Assembly, Catalyst 5 | 1 | \$209.59 |
| 1-7, 8b, 9-15, 16b, 17b, 18-21, 22a, 24-26, 31, 32 | 504572 | Height Adjustable Flip Back T-Arm Long Full with Standard Pad Assembly, Catalyst 5 | 1 | \$209.59 |
| 18, 20, 24, 25 | 112958 | Armrest Flip Back Height Adj T-Arm Hinge Assembly, PAIR, Catalyst 5 <i>This kit is for a pair of hinge assemblies. A pair is required as a replacement as of May 19th, 2021</i> | 1 | \$43.26 |
| 30, 33-34 | 116058 | Height Adjustable Flip Back Receiver Assm Cat E | 1 | \$18.00 |
| 31, 32 | 504577 | Height Adjustable Flip Back T-Arm Mounting Hardware Assembly <i>This kit contains various mounting hardware for multiple configurations</i> | 1 | \$14.34 |

| Item Number | Part Number | Description | Quantity | Retail |
|--------------------|-------------|--|----------|---------|
| 3, 5, 6, 9, 11, 13 | 504560 | Height Adjustable Flip Back T-Arm Latch Assembly | 1 | \$47.72 |
| 4, 9, 10, 12 | 504561 | Height Adjustable Flip Back T-Arm Release Lever Assembly | 1 | \$12.22 |
| 1 | 003420 | T-Arm Post | 1 | \$24.27 |
| 2 | 000095 | Height Adj T Armrest Transfer Tube | 1 | \$29.47 |
| 7 | 000090 | Height Adj T Armrest Shim | 2 | \$2.68 |
| 8a, 14 | 112906 | Height Adj T Armrest Upper Desk w/ Tube End Plug | 1 | \$80.50 |
| 8b, 14 | 112907 | Height Adj T Armrest Upper Full w/ Tube End Plug | 1 | \$80.50 |
| 14 | 100803 | Tube End Plug 7/8" | 2 | \$1.66 |
| 15, 16a, 21 | 504562 | Height Adjustable Flip Back T-Arm Side Shield Short | 1 | \$30.89 |
| 15, 16b, 21 | 504563 | Height Adjustable Flip Back T-Arm Side Shield Long | 1 | \$30.89 |
| 17a, 19 | 504564 | Height Adjustable Flip Back T-Arm Metal Insert Short | 1 | \$35.30 |
| 17b, 19 | 504565 | Height Adjustable Flip Back T-Arm Metal Insert Long | 1 | \$37.51 |
| 22a | 100500 | Arm Pad Full | 1 | \$24.82 |
| 22b | 100501 | Arm Pad Desk | 1 | \$24.82 |
| 23 | 003260 | Spacer, .62ODx.316IDx.090L | 2 | \$0.41 |
| 26 | 100696 | M5 x 30mm PPH Capscrew | 2 | \$1.66 |
| 27 | 100746 | M6 Flat Washer Blk Zn | 2 | \$1.66 |
| 28 | 102625 | M6 X 30 BHCS BLK ZC | 1 | \$1.66 |
| 29 | 111976 | Flip Back T-Arm Spacer | 1 | \$6.65 |
| 30 | 100797 | M6 x 40 BHCS BLK ZC | 2 | \$1.10 |
| 33 | 112010 | T-Arm Receiver | 1 | \$3.20 |
| 34 | 111954 | Threaded Half Nut Plate | 1 | \$6.30 |
| 35 | 100831 | 1/2" Washer OD .040" Thick | 1 | \$1.66 |