


Catalyst E Back Upholstery - Tension Adjustable

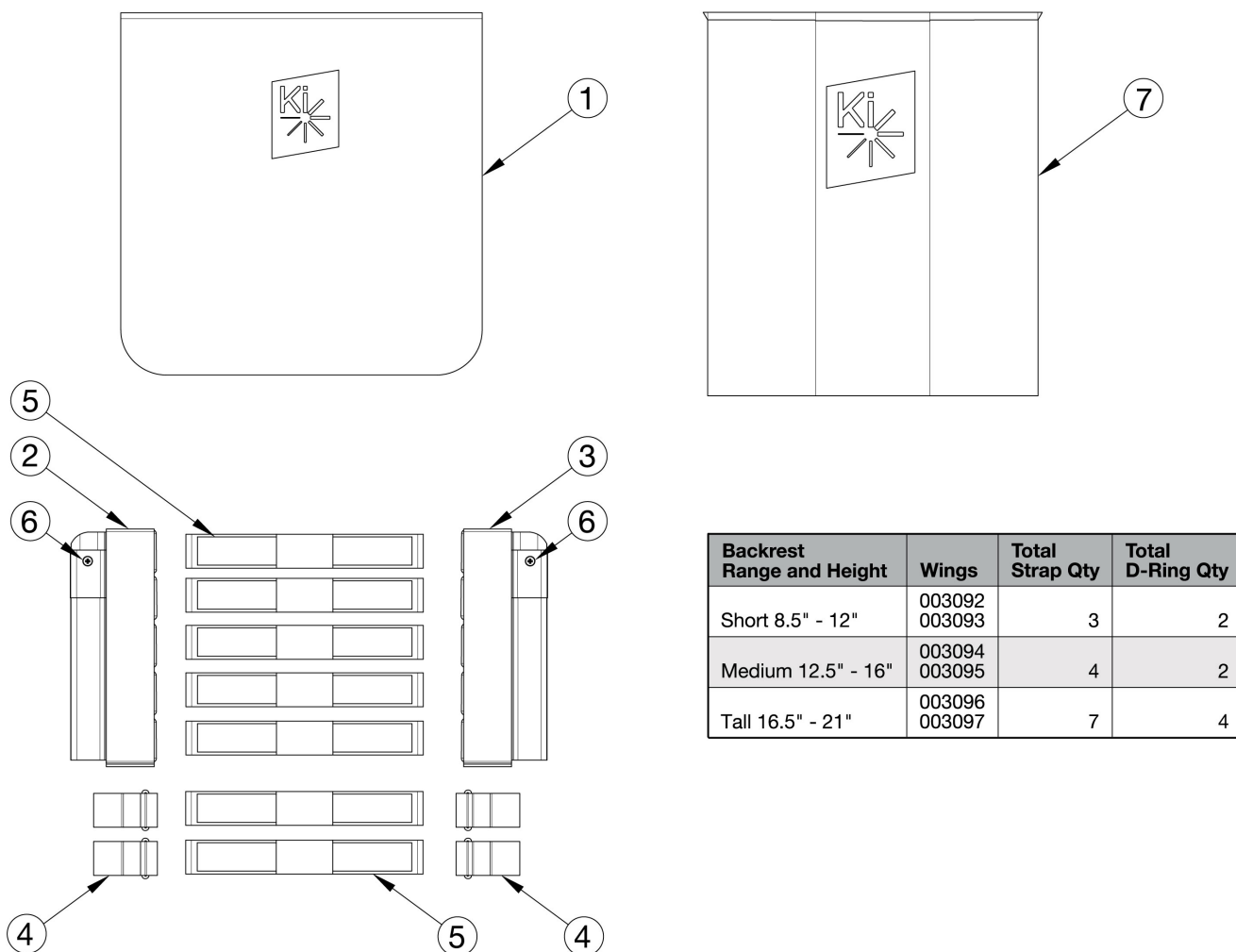

Assemblies contain the cover, amount of straps and d-rings used for the tallest height listed in the assembly. Attaching hardware is not included with the back upholstery assemblies.

Item Number	Part Number	Description	Quantity
1a, 2a, 3a, 4, 5a	116000	Back Upholstery Assembly Tension Adjustable with Standard Cover 14W, 8.5" - 12"	1
1b, 2b, 3b, 4, 5a	116001	Back Upholstery Assembly Tension Adjustable with Standard Cover 14W, 12.5"-16"	1
1c, 2c, 3c, 4, 5a	116002	Back Upholstery Assembly Tension Adjustable with Standard Cover 14W, 16.5" - 21"	1
1d, 2a, 3a, 4, 5b	117476	Back Upholstery Assembly Tension Adjustable with Standard Cover 15W, 8.5" - 12"	1
1e, 2b, 3b, 4, 5b	117477	Back Upholstery Assembly Tension Adjustable with Standard Cover 15W, 12.5"-16"	1
1g, 2a, 3a, 4, 5c	116003	Back Upholstery Assembly Tension Adjustable with Standard Cover 16W, 8.5" - 12"	1

Item Number	Part Number	Description	Quantity
1h, 2b, 3b, 4, 5c	116004	Back Upholstery Assembly Tension Adjustable with Standard Cover 16W, 12.5" - 16"	1
1i, 2c, 3c, 4, 5c	116005	Back Upholstery Assembly Tension Adjustable with Standard Cover 16W, 16.5" - 21"	1
1j, 2a, 3a, 4, 5d	117479	Back Upholstery Assembly Tension Adjustable with Standard Cover 17W, 8.5" - 12"	1
1k, 2b, 3b, 4, 5d	117480	Back Upholstery Assembly Tension Adjustable with Standard Cover 17W, 12.5" - 16"	1
1l, 2c, 3c, 4, 5d	117481	Back Upholstery Assembly Tension Adjustable with Standard Cover 17W, 16.5" - 21"	1
1m, 2a, 3a, 4, 5e	116006	Back Upholstery Assembly Tension Adjustable with Standard Cover 18W, 8.5" - 12"	1
1n, 2b, 3b, 4, 5e	116007	Back Upholstery Assembly Tension Adjustable with Standard Cover 18W, 12.5" - 16"	1
1o, 2c, 3c, 4, 5e	116008	Back Upholstery Assembly Tension Adjustable with Standard Cover 18W, 16.5" - 21"	1
1p, 2a, 3a, 4, 5f	117482	Back Upholstery Assembly Tension Adjustable with Standard Cover 19W, 8.5" - 12"	1
1s, 2a, 3a, 4, 5g	116009	Back Upholstery Assembly Tension Adjustable with Standard Cover 20W, 8.5" - 12"	1
1t, 2b, 3b, 4, 5g	116010	Back Upholstery Assembly Tension Adjustable with Standard Cover 20W, 12.5" - 16"	1
1u, 2c, 3c, 4, 5g	116011	Back Upholstery Assembly Tension Adjustable with Standard Cover 20W, 16.5" - 21"	1
1v, 2a, 3a, 4, 5h	117519	Back Upholstery Assembly Tension Adjustable with Standard Cover 21W, 8.5" - 12"	1
1w, 2b, 3b, 4, 5h	117486	Back Upholstery Assembly Tension Adjustable with Standard Cover 21W, 12.5" - 16"	1
1x, 2c, 3c, 4, 5h	117487	Back Upholstery Assembly Tension Adjustable with Standard Cover 21W, 16.5" - 21"	1
1y, 2b, 3b, 4, 5i	116012	Back Upholstery Assembly Tension Adjustable with Standard Cover 22W, 12.5" - 16"	1
1z, 2c, 3c, 4, 5i	116013	Back Upholstery Assembly Tension Adjustable with Standard Cover 22W, 16.5" - 21"	1
2a, 3a, 4, 5a, 7a	116014	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 14W, 8.5" - 12"	1
2b, 3b, 4, 5a, 7b	116015	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 14W, 12.5" - 16"	1
2c, 3c, 4, 5a, 7c	116016	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 14W, 16.5" - 21"	1
2a, 3a, 4, 5b, 7d	117488	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 15W, 8.5" - 12"	1
2b, 3b, 4, 5b, 7e	117489	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 15W, 12.5" - 16"	1
2c, 3c, 4, 5b, 7f	117490	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 15W, 16.5" - 21"	1
2a, 3a, 4, 5b, 7g	116017	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 16W, 8.5" - 12"	1
2b, 3b, 4, 5b, 7h	116018	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 16W, 12.5" - 16"	1
2c, 3c, 4, 5b, 7i	116019	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 16W, 16.5" - 21"	1
2a, 3a, 4, 5c, 7j	117491	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 17W, 8.5" - 12"	1

Item Number	Part Number	Description	Quantity
2b, 3b, 4, 5c, 7k	117492	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 17W, 16.5" - 21"	1
2c, 3c, 4, 5c, 7l	117493	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 17W, 12.5" - 16"	1
2a, 3a, 4, 5d, 7m	116020	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 18W, 8.5" - 12"	1
2b, 3b, 4, 5d, 7n	116021	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 18W, 12.5" - 16"	1
2c, 3c, 4, 5d, 7o	116022	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 18W, 16.5" - 21"	1
2a, 3a, 4, 5e, 7p	117494	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 19W, 8.5" - 12"	1
2b, 3b, 4, 5e, 7q	117495	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 19W, 12.5" - 16"	1
2c, 3c, 4, 5e, 7r	117496	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 19W, 16.5" - 21"	1
2a, 3a, 4, 5f, 7s	116023	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 20W, 8.5" - 12"	1
2b, 3b, 4, 5f, 7t	116024	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 20W, 12.5" - 16"	1
2c, 3c, 4, 5f, 7u	116025	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 20W, 16.5" - 21"	1
2a, 3a, 4, 5g, 7v	117497	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 21W, 8.5" - 12"	1
2b, 3b, 4, 5g, 7w	117498	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 21W, 12.5" - 16"	1
2c, 3c, 4, 5g, 7x	117499	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 21W, 16.5" - 21"	1
2b, 3b, 4, 5h, 7y	116026	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 22W, 12.5" - 16"	1
2c, 3c, 4, 5g, 7z	116027	Back Upholstery Assembly Tension Adjustable with Air Mesh Cover 22W, 16.5" - 21"	1
1a	002497	Backrest Cover 13" Rigid / 14" Cat x 9-12"	1
1b	002499	Backrest Cover 13" Rigid / 14" Cat x 13-16"	1
1c	003043	Backrest Cover 13" Rigid - 14" CAT/Liberty x 17-20"	1
1d	002501	Backrest Cover 14" Rigid / 15" Cat x 9-12"	1
1e	002503	Backrest Cover 14" Rigid / 15" Cat x 13-16"	1
1f	003044	Backrest Cover 14" Rigid - 15" CAT/Liberty x 17-20"	1
1g	002505	Backrest Cover 15" Rigid / 16" Cat x 9-12"	1
1h	002507	Backrest Cover 15" Rigid / 16" Cat x 13-16"	1
1i	003045	Backrest Cover 15" Rigid - 16" CAT/Liberty x 17-20"	1
1j	002509	Backrest Cover 16" Rigid / 17" Cat x 9-12"	1
1k	002511	Backrest Cover 16" Rigid / 17" Cat x 13-16"	1

Item Number	Part Number	Description	Quantity
1l	003046	Backrest Cover 16" Rigid - 17" CAT/Liberty x 17-20"	1
1m	003048	Backrest Cover 17" Rigid / 18" Cat x 9-12"	1
1n	003050	Backrest Cover 17" Rigid / 18" Cat x 13-16"	1
1o	003052	Backrest Cover 17" Rigid - 18" CAT/Liberty x 17-20"	1
1p	003054	Backrest Cover 18" Rigid / 19" Cat x 9-12"	1
1q	003056	Backrest Cover 18" Rigid / 19" Cat x 13-16"	1
1r	003058	Backrest Cover 18" Rigid - 19" CAT/Liberty x 17-20"	1
1s	003060	Backrest Cover 19" Rigid / 20" Cat x 9-12"	1
1t	003062	Backrest Cover 19" Rigid / 20" Cat x 13-16"	1
1u	003064	Backrest Cover 19" Rigid - 20" CAT/Liberty x 17-20"	1
1v	003066	Backrest Cover 20" Rigid / 21" Cat x 9-12"	1
1w	003068	Backrest Cover 20" Rigid / 21" Cat x 13-16"	1
1x	003070	Backrest Cover 20" Rigid - 21" CAT/Liberty x 17-20"	1
1y	003072	Backrest Cover 21" Rigid / 22" Cat x 13-16"	1
1z	003074	Backrest Cover 21" Rigid - 22" CAT/Liberty x 17-20"	1
2a	003093	Tension Adjustable Upholstery Wing 3.5" Height Left	1
2b	003095	Tension Adjustable Upholstery Wing 5.5" Height Left	1
2c	003097	Tension Adjustable Upholstery Wing 9.5" Height Left	1
3a	003092	Tension Adjustable Upholstery Wing 3.5" Height Right	1
3b	003094	Tension Adjustable Upholstery Wing 5.5" Height Right	1
3c	003096	Tension Adjustable Upholstery Wing 9.5" Height Right	1
4	100973	Back Upholstery D-Ring	2
5a	003487	Back Upholstery Strap 15.3" <i>Used on 14" chair widths</i>	3
5b	002532	Back Upholstery Strap 17.3" <i>Used on 15" chair widths</i>	3
5c	002533	Back Upholstery Strap 19.3" <i>Used on 16" chair widths</i>	3
5d	002534	Back Upholstery Strap 21.3"	3

Item Number	Part Number	Description	Quantity
		<i>Used on 17" chair widths</i>	
5e	002535	Back Upholstery Strap 23.3" <i>Used on 18" chair widths</i>	3
5f	002536	Back Upholstery Strap 25.3" <i>Used on 19" chair widths</i>	3
5g	002537	Back Upholstery Strap 27.3" <i>Used on 20" chair widths</i>	3
5h	002538	Back Upholstery Strap 29.3" <i>Used on 21" chair widths</i>	3
5i	002539	Back Upholstery Strap 31.3" <i>Used on 22" chair widths</i>	3
6	100734	M5 x 16 PHMS ZC	2
7a	003896	Air Mesh Backrest Cover 14C 13R Width x 9-12" Height	1
7b	003898	Air Mesh Backrest Cover 14C 13R Width x 13-16" Height	1
7c	003913	Air Mesh Backrest Cover 14C 13R Width x 17-20" Height	1
7d	003900	Air Mesh Backrest Cover 15C 14R Width x 9-12" Height	1
7e	003902	Air Mesh Backrest Cover 15C 14R Width x 13-16" Height	1
7f	003914	Air Mesh Backrest Cover 15C 14R Width x 17-20" Height	1
7g	003904	Air Mesh Backrest Cover 16C 15R Width x 9-12" Height	1
7h	003906	Air Mesh Backrest Cover 16C 15R Width x 13-16" Height	1
7i	003915	Air Mesh Backrest Cover 16C 15R Width x 17-20" Height	1
7j	003909	Air Mesh Backrest Cover 17C 16R Width x 9-12" Height	1
7k	003911	Air Mesh Backrest Cover 17C 16R Width x 13-16" Height	1
7l	003916	Air Mesh Backrest Cover 17C 16R Width x 17-20" Height	1
7m	003918	Air Mesh Backrest Cover 18C 17R Width x 9-12" Height	1
7n	003920	Air Mesh Backrest Cover 18C 17R Width x 13-16" Height	1
7o	003922	Air Mesh Backrest Cover 18C 17R Width x 17-20" Height	1
7p	003924	Air Mesh Backrest Cover 19C 18R Width x 9-12" Height	1
7q	003926	Air Mesh Backrest Cover 19C 18R Width x 13-16" Height	1
7r	003928	Air Mesh Backrest Cover 19C 18R Width x 17-20" Height	1

Item Number	Part Number	Description	Quantity
7s	003930	Air Mesh Backrest Cover 20C 19R Wide x 9-12" Height	1
7t	003932	Air Mesh Backrest Cover 20C 19R Width x 13-16" Height	1
7u	003934	Air Mesh Backrest Cover 20C 19R Width x 17-20" Height	1
7v	003936	Air Mesh Backrest Cover 21C 20R Width x 9-12" Height	1
7w	003938	Air Mesh Backrest Cover 21C 20R Width x 13-16" Height	1
7x	003940	Air Mesh Backrest Cover 21C 20R Width x 17-20" Height	1
7y	003942	Air Mesh Backrest Cover 22C 21R Width x 13-16" Height	1
7z	003944	Air Mesh Backrest Cover 22C 21R Width x 17-20" Height	1