


Clik / Rogue XP Swing Away Armrest

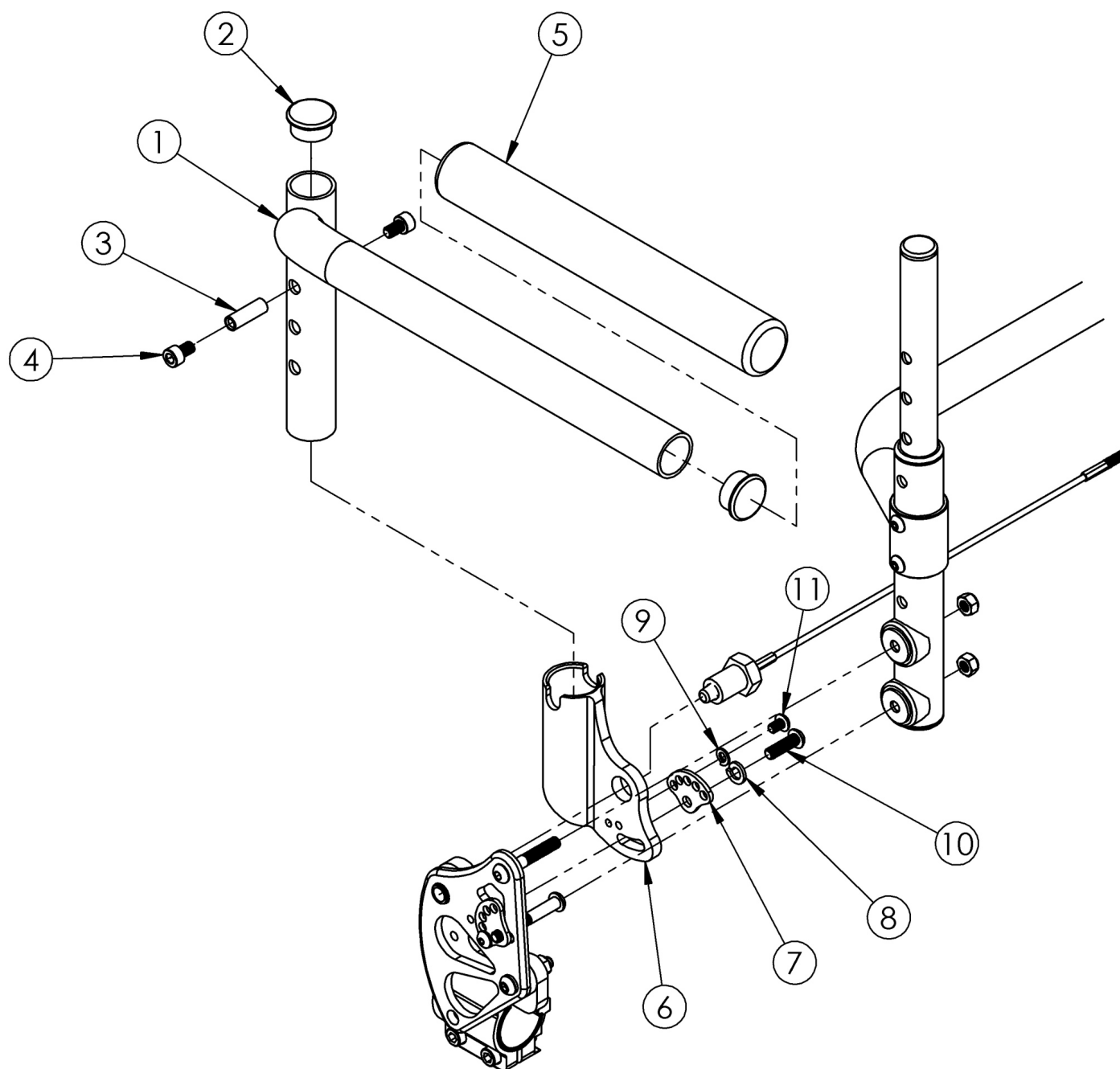

Swing Away Arm	Height Range
Short	7"-9"
Tall	9.25"-11.25"

When to use Short vs. Tall Swing Away Arms: 18" Wheel: Rear Seat Height = 11.5" use Tall // Rear Seat Height = 12" or greater use Short 20" Wheel: Rear Seat Height = 12.5" use Tall // Rear Seat Height = 13" or greater use Short 22" Wheel: Rear Seat Height = 13.5-14.5" use Tall // Rear Seat Height = 15" or greater use Short 24" Wheel: Rear Seat Height = 14.5-16.5" use Tall // Rear Seat Height = 17" or greater use Short 25" Wheel: Rear Seat Height = 14.5-17.5" use Tall // Rear Seat Height = 18" or greater use Short

Item Number	Part Number	Description	Quantity
1a, 2-11	502492	Swing Away Short Armrest w/ Receiver Assembly Right	1
1b, 2-11	502493	Swing Away Short Armrest w/ Receiver Assembly Left	1
1a, 1b, 2-11	502494	Swing Away Short Armrest w/ Receiver Assembly PAIR	1
1c, 2-11	502495	Swing Away Tall Armrest w/ Receiver Assembly Right	1
1d, 2-11	502496	Swing Away Tall Armrest w/ Receiver Assembly Left	1
1c, 1d, 2-11	502497	Swing Away Tall Armrest w/ Receiver Assembly PAIR	1
1a, 2 - 5	502485	Swing Away Short Armrest Assembly Right	1
1b, 2 - 5	502486	Swing Away Short Armrest Assembly Left	1
1a, 1b, 2 - 5	502487	Swing Away Short Armrest Assembly PAIR	1
1c, 2 - 5	502488	Swing Away Tall Armrest Assembly Right	1
1d, 2 - 5	502489	Swing Away Tall Armrest Assembly Left	1
1c, 1d, 2 - 5	502490	Swing Away Tall Armrest Assembly PAIR	1
6-11	502491	Swing Away Armrest Receiver Assembly One Side	1
2	100804	Tube End Plug 1"	2
3	000065	M6 x 23.68 Threaded Barrel	1
4	101179	M6 X 10 Shcs Blk Zc W/Patch	2
6	001769	Arm Receiver LW	1
7	001762	Arc Adjustment Cam	1
8	101835	M6 Lock Washer Blk Zn	1
9	102614	M5 Split Lock Washer BLK ZC	1
10	102623	M6 x 20 BHCS BLK ZC	1
11	102621	M5 x 8 BHCS BLK ZC	1