


Superlight Handrim

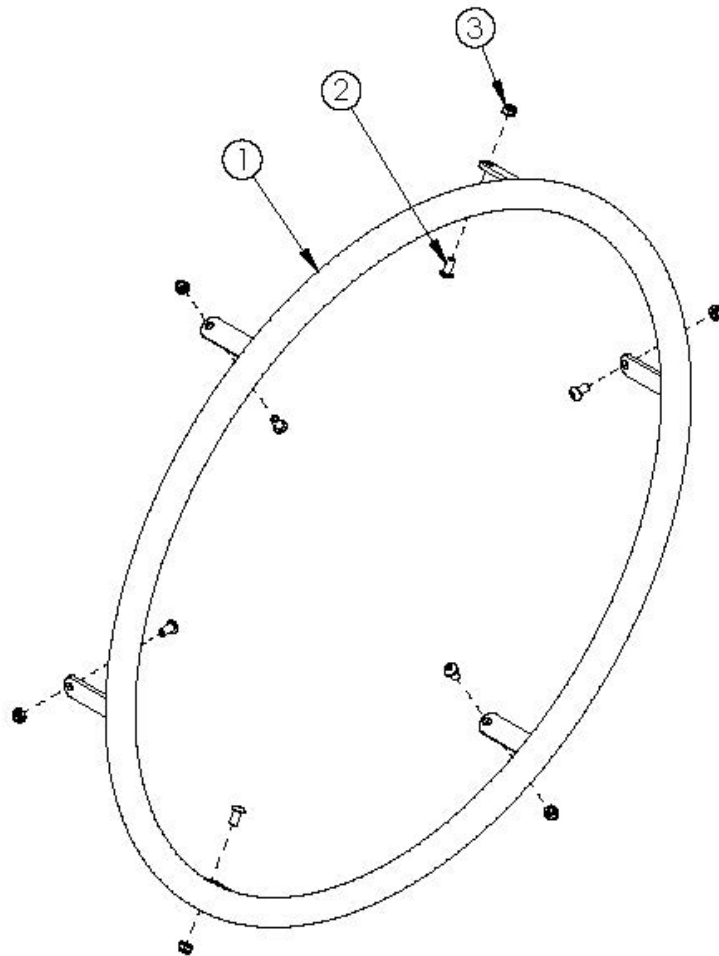

Superlight Handrim with Micro Grip



For more information see Handrim Hardware Chart. Quantity in Handrim Hardware Kits are for one wheel. <p>

Not available with Heavy Duty Option<p>

Tab mounted handrims must be installed prior to the tire being installed.

Item Number	Part Number	Description	Quantity	Retail
2b, 3	502174	Handrim Hardware Kit, 6 Tab, EACH <i>Used with Spinergy, Topolino and MAXX Spoke wheels</i>	1	\$16.55
2a	502170	Handrim Hardware Kit, Superlight, EACH <i>Used with Superlight Spoke</i>	1	\$16.55
1a	101161	Handrim Superlight Aluminum Anodized 24", 6 Tab	1	\$98.18
1b	505585	Handrim Superlight Aluminum Anodized 24" w/ Micro Grip	1	\$137.50
1c	101160	Handrim Superlight Aluminum Anodized 25", 6 Tab	1	\$98.18
1d	505586	Handrim Superlight Aluminum Anodized 25" w/ Micro Grip	1	\$138.68

Item Number	Part Number	Description	Quantity	Retail
2a	100536	M5 x 12 BTN SHCS ZC <i>Used with Superlight Spoke</i>	6	\$2.21
2b	100669	M5 x 10 BTN SHCS BLKZC w/Patch <i>Used with Spinergy, Topolino and MAXX Spoke wheels</i>	6	\$1.66
3	100657	M5 Nylock Nut Blk Zn <i>Used with Spinergy, Topolino and MAXX Spoke wheels</i>	6	\$0.56

Handrim Hardware Chart

Wheel		Handrim Connection				Handrim						
Wheel	Wheel Part No.	Connection Points	Tab	Spacer*	Screw*	Aluminum Anodized	Superlight	Plastic Coated	Projection	Natural Fit Standard	Natural Fit LT	Flex Rim
18" Ki Spoke	200529	3	100698	100653	Aluminum, Plastic Ctd, Projection: 100654	100206		101106				
20" Ki Spoke	200530					200536		200542				
22" Ki Spoke	200531					100560		100576	100569	200538	200201	
24" Ki Spoke	200532					200349	101161	100577	200547	100793	200202	
25" Ki Spoke	200533				Natural Fit: 100835	200350	101870	200548	200539	200540		
26" Ki Spoke	200534				200351		100578	200549	100907	101454		
24" Superlight	101159	6			100536	100754	101161	100836		100830	100828	
25" Superlight	101160							101160	101091		101464	101460
20" 5-Spoke X Core	101961	5		Aluminum, Projection, Natural Fit: 100629	100724	101897		101963				
22" 5-Spoke X Core	101962						101898		101964			
24" 5-Spoke X Core	100960				Plastic Ctd: 101756	101893	100975		100976	200546		
24" 5-Spoke X Core	100960										100768	100769
20" Spinergy Spox	See Spinergy Spox Page	6			Screw: 100669 Nut: 100657	103125		103179				
22" Spinergy Spox						100827		100808		100889	100888	
24" Spinergy Spox						100766	101161	100615		100830	100828	
25" Spinergy Spox						100767	101160	100765		101464	101460	
26" Spinergy Spox						101477		101148		200200	100950	
22" Spinergy LX	See Spinergy LX Page	6			Screw: 100669 Nut: 100657	100827		100808		100889	100888	
24" Spinergy LX						100766	101161	100615		100830	100828	200213
25" Spinergy LX						100767	101160	100765		101464	101460	200208
26" Spinergy LX						101477		101148		200200	100950	
18" MAXX Spoke	109285	6	100698 (Not used w/ Superlight Handrim)	100653 (Not used w/ Superlight Handrim)	Aluminum, Plastic Ctd, Projection: 100654	100206		101106				
20" MAXX Spoke	108244					200536		200542				
22" MAXX Spoke	105135				Natural Fit: 100835	100560		100576	100569	200538	200201	
24" MAXX Spoke	105136				Superlight: 100835	200349	101161	100577	200547	100793	200202	
25" MAXX Spoke	107436				Screw: 100669	200350		101870	200548	200539	200540	
26" MAXX Spoke	107437				Nut: 100657	200351		100578	200549	100907	101454	

* Spacer and screw part numbers listed in the chart are for standard handrim mounting. If using close mount handrim mounting on aluminum anodized, plastic coated or projection handrims, use part number 000240 for the spacer, part number 102616 for the screw on Ki Spoke and MAXX Spoke wheels or part number 100666 for the screw on 5-Spoke X Core wheels.