


Focus CR O2 Holder

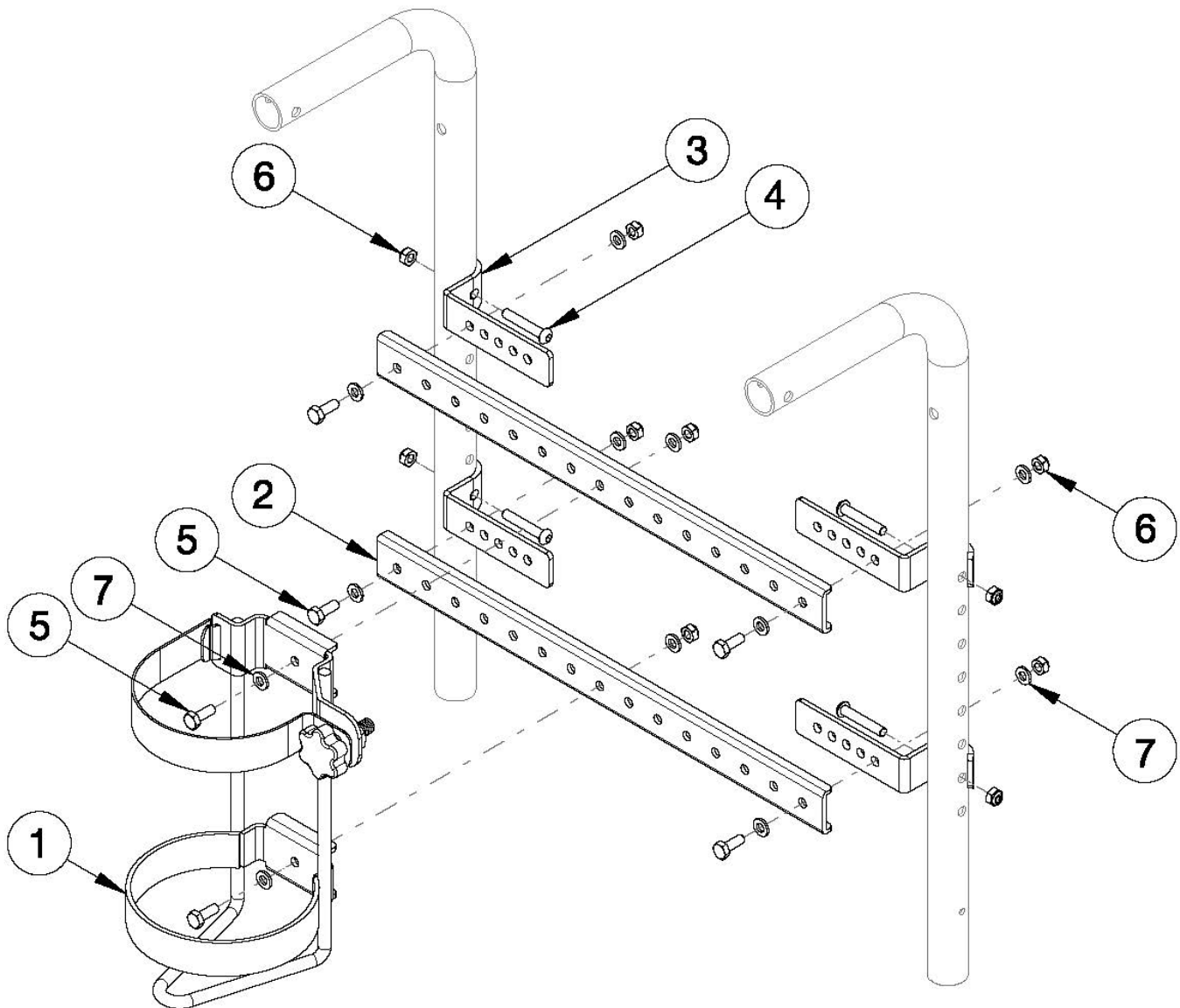

Item Number	Part Number	Description	Quantity	Retail
1, 2a, 3 - 7	501441	O2 Holder Assembly, 14-16 Width, Tilt	1	\$170.00
1, 2b, 3 - 7	501442	O2 Holder Assembly, 16-18 Width, Tilt	1	\$170.00
1, 2c, 3 - 7	501443	O2 Holder Assembly, 18-20 Width, Tilt	1	\$170.00
1, 2d, 3 - 7	501444	O2 Holder Assembly, 20-24 Width, Tilt	1	\$170.00
1	001416	O2 Holder Assembly	1	\$136.68
2a	001385	Strut, Channel, 14-16 Width, Tilt	2	\$21.87

Item Number	Part Number	Description	Quantity	Retail
2b	001386	Strut, Channel, 16-18 Width, Tilt	2	\$21.87
2c	001387	Strut, Channel, 18-20 Width, Tilt	2	\$22.97
2d	001388	Strut, Channel, 20-24 Width, Tilt	2	\$27.34
3	001280	Mount Bracket, 1" Tube Tilt	4	\$10.93
4	101811	M6 x 35 BHCS Blk Zn	4	\$1.05
5	102327	M6 x 16 HHCS Blk Zn	6	\$1.05
6	100658	M6 Nylock Nut Blk Zn	10	\$1.05
7	100746	M6 Flat Washer Blk Zn	12	\$1.58