


# Catalyst Caster Pin Locks

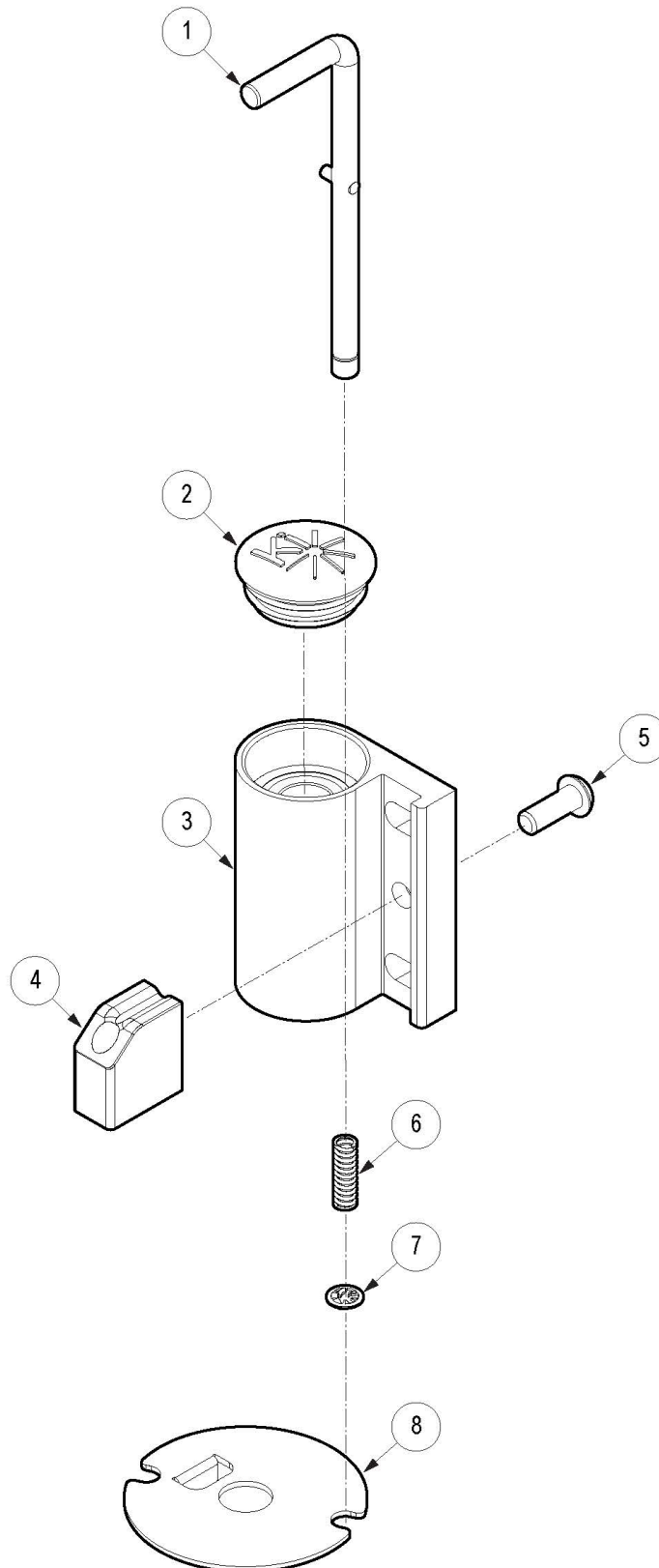

If retrofitting to add Caster Pin Locks, you must order an assembly that contains caster housings.

Item Number	Part Number	Description	Quantity
1a, 2, 3a, 4 - 7, 8a	109744	Caster Pin Lock Assembly with Caster Housing, Black, Left, Standard Stem, Catalyst	1
1a, 2, 3c, 4 - 7, 8a	109745	Caster Pin Lock Assembly with Caster Housing, Black, Right, Standard Stem, Catalyst	1
1a, 2, 3a, 3c, 4 - 7, 8a, 8b	109746	Caster Pin Lock Assembly with Caster Housings, Black, Standard Stem, PAIR, Catalyst	1
1b, 2, 3a, 4 - 7, 8a	109747	Caster Pin Lock Assembly with Caster Housing, Black, Left, 3/4 Stem, Catalyst	1
1b, 2, 3c, 4 - 7, 8b	109748	Caster Pin Lock Assembly with Caster Housing, Black, Right, 3/4 Stem, Catalyst	1
1b, 2, 3a, 3c, 4 - 7, 8a, 8b	109749	Caster Pin Lock Assembly, with Caster Housing, Black, 3/4 Stem, PAIR, Catalyst	1
1a, 4 - 7, 8a	502627	Caster Pin Lock Assembly, Left, Standard, Catalyst <i>This assembly does not contain a caster housing or caster housing cap</i>	1
1a, 4 - 7, 8b	502628	Caster Pin Lock Assembly, Right, Standard, Catalyst <i>This assembly does not contain a caster housing or caster housing cap</i>	1
1a, 4 - 7, 8a, 8b	500300	Caster Pin Lock Assembly, Standard, PAIR, Catalyst <i>This assembly does not contain caster housings or caster housing caps</i>	1
1b, 4 - 7, 8a	502629	Caster Pin Lock Assembly, Left, 3/4, Catalyst <i>This assembly does not contain a caster housing or caster housing cap</i>	1
1b, 4 - 7, 8b	502630	Caster Pin Lock Assembly, Right, 3/4, Catalyst <i>This assembly does not contain a caster housing or caster housing cap</i>	1
1b, 4 - 7, 8a, 8b	500301	Caster Pin Lock Assembly, 3/4, PAIR, Catalyst <i>This assembly does not contain caster housings or caster housing caps</i>	1
1a	200414	Caster Lock Pin Std CAT <i>Used with Standard Stem</i>	1
1b	200415	Caster Lock Pin 3/4 CAT <i>Used with + 3/4" Stem</i>	1
2	102005	Ki Caster Housing Cap	1
3a	001431	Standard Caster Housing for Caster Pin Lock, Left, Catalyst, Black	1
3b	001431_ _ _	Color Anodized Standard Caster Housing for Caster Pin Lock, Left, Catalyst	1
3c	001432	Standard Caster Housing for Caster Pin Lock, Right, Catalyst, Black	1
3d	001432_ _ _	Color Anodized Standard Caster Housing for Caster Pin Lock, Right, Catalyst	1
4	200417	Caster Lock Mounting Block CAT	1

Item Number	Part Number	Description	Quantity
5	101721	M6 X 16 Btn Shcs W/ Patch	1
6	200418	Caster Pin Lock Spring	1
7	102043	1/4" Pushnut Plated	1
8a	000397L	Caster Pin Lock Plate Left	1
8b	000397R	Caster Pin Lock Plate Right	1

# Color Anodizing Samples

