



Rogue ALX / Little Wave XP Single Sided Fork (Formerly Tsunami)

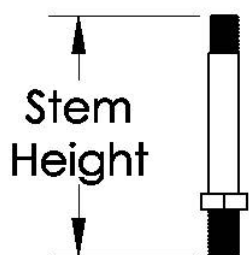

Std	2.8"
+ 3/4"	3.4"
+ 1"	3.9"

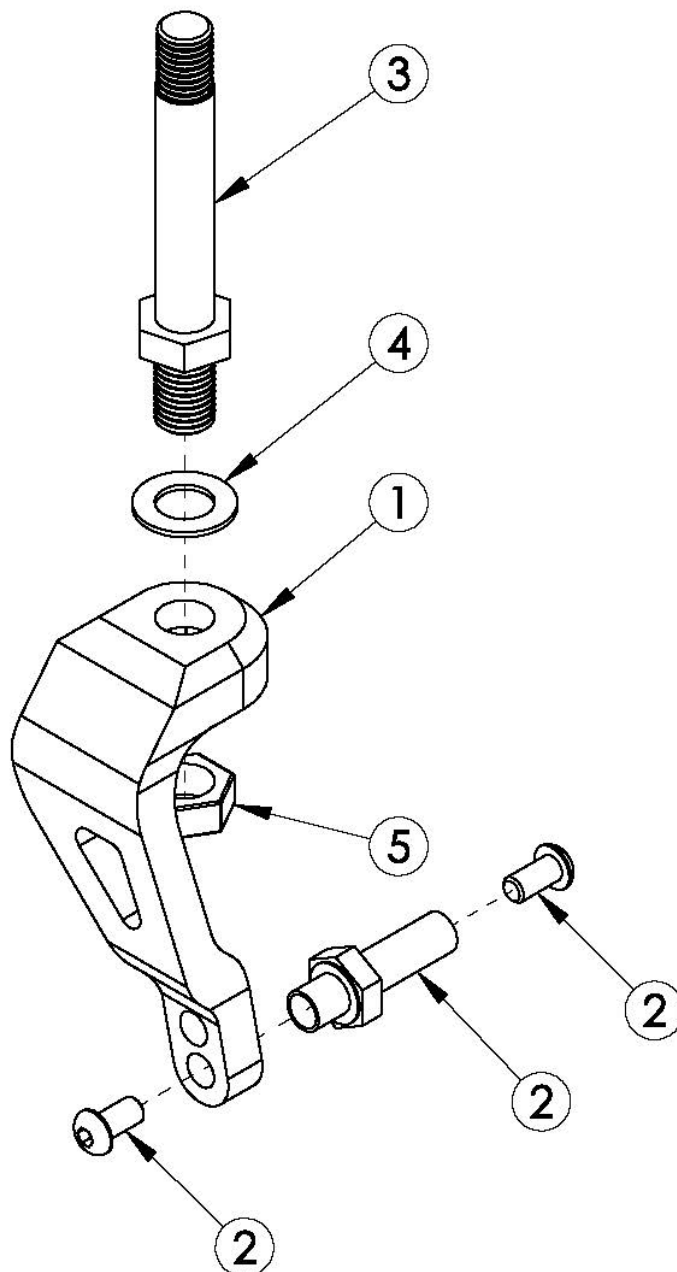

Item Number	Part Number	Description	Quantity
1, 2, 3a, 4, 5	500996	Single Sided Fork Assembly Std Stem, RG ALX/LW XP	1
1, 2, 3b, 4, 5	501495	Single Sided Fork Assembly 3/4" Stem, RG ALX/LW XP	1
1, 2, 3c, 4, 5	500998	Single Sided Fork Assembly 1" Stem, RG ALX/LW XP	1
1, 2, 5	100980INST	Fork Single Sided Black, RG ALX/LW XP	1

Item Number	Part Number	Description	Quantity
2	102063	Axle w/Hardware PAIR for Single Sided Fork, RG ALX/LW XP	1
3a, 5	105073INST	Frog Legs Stem 1/2 x 1.85", Std	1
3b, 5	105059INST	Frog Legs Fork Stem 1/2 x 2.5", +3/4"	1
3c, 5	101136INST	Single Sided Fork Stem, +1"	1
4	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	1