


Focus CR Caster Arm

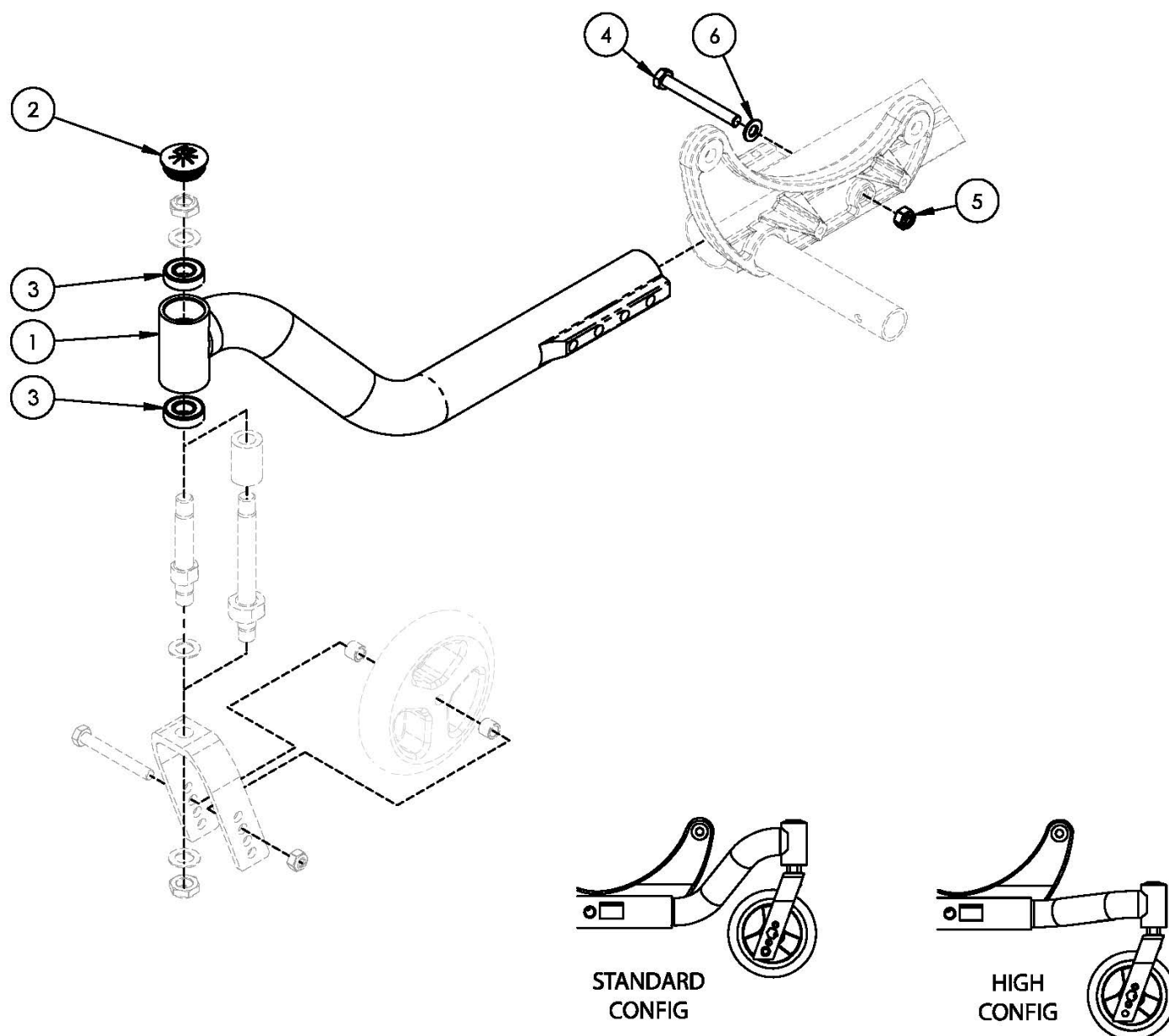

Item Number	Part Number	Description	Quantity
1a, 2, 3	501067_ _ _	Frame Assembly Focus Caster Arm Right _ _ _	1
1b, 2, 3	501068_ _ _	Frame Assembly Focus Caster Arm Left _ _ _	1
1c, 2, 3	501426_ _ _	Frame Assembly Focus Caster Arm High Right _ _ _	1
1d, 2, 3	501427_ _ _	Frame Assembly Focus Caster Arm High Left _ _ _	1
2	102005	Ki Caster Housing Cap	1

Item Number	Part Number	Description	Quantity
3	100902	Bearing 1 1/8" x 1/2" R8RS	2
4	101830	M8 x 70 HHCS Blk Zn	1
5	100659	M8 Nylock Nut Blk Zn	1
6	101858	Washer Flat M8 Blk Zn	1

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer