


Catalyst 5 Flip Back Armrest

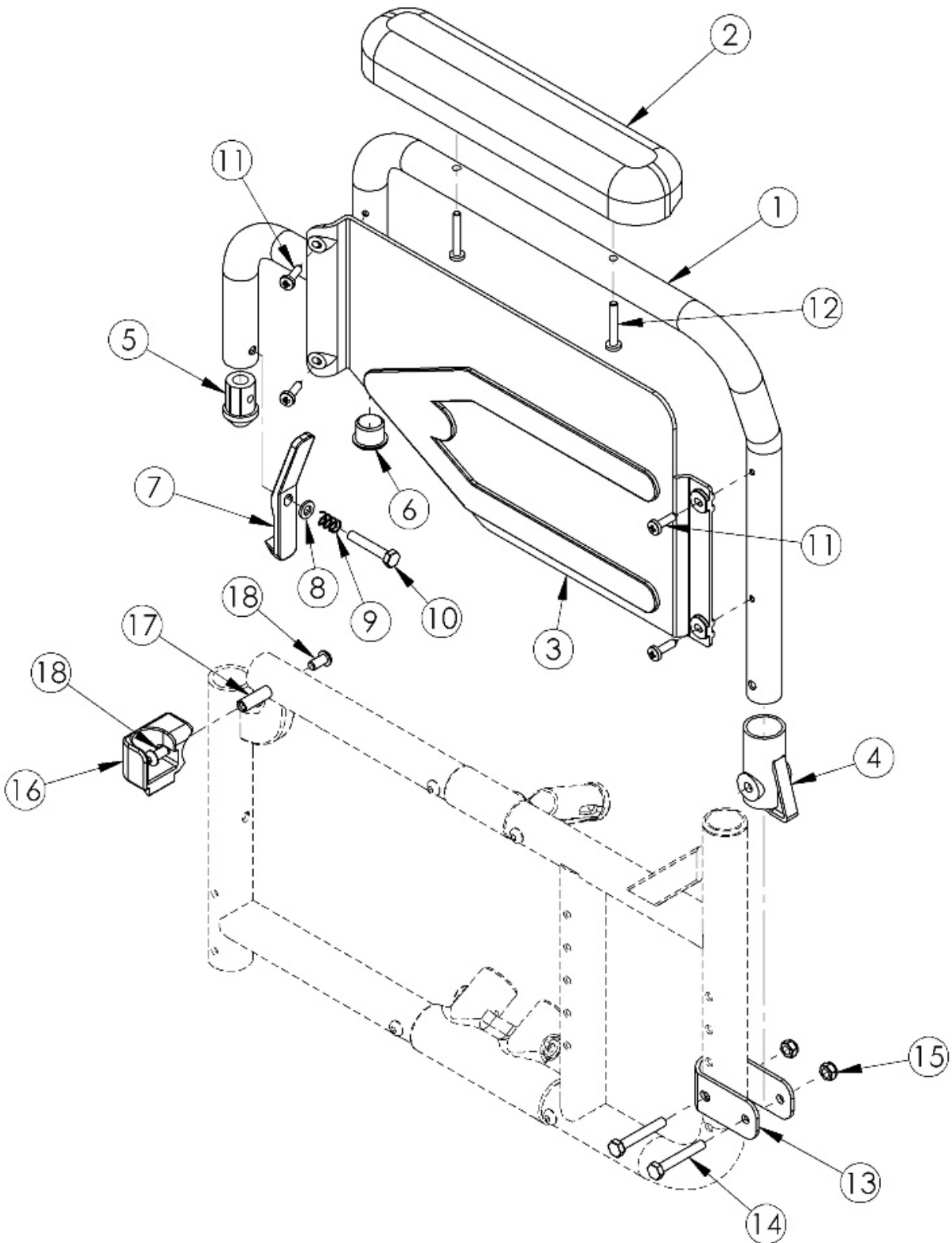

If retrofitting to Flip Back Armrests, please note the following: <p>

New front frames are required <p>

If One Arm Drive is present, a new drive shaft may be required. In addition to a new drive shaft, order the corresponding hardware kit: 109438 for Blade Style One Arm Drive, 503433 for Hex Style One Arm Drive or 112898 for Quick Release Style One Arm Drive <p>

Extended wheellock clamps may be needed

Item Number	Part Number	Description	Quantity	Retail
1a, 2a, 3b, 4 - 12, 13, 14, 15, 16a, 17, 18	502259	Armrest Flip Back w/ Receiver and Bracket Kit Left Desk CAT5	1	\$163.29
1c, 2a, 3a, 4 - 12, 13, 14, 15, 16b, 17, 18	502260	Armrest Flip Back w/ Receiver and Bracket Kit Right Desk CAT5	1	\$163.29
1b, 2b, 3b, 4 - 12, 13, 14, 15, 16a, 17, 18	502261	Armrest Flip Back w/ Receiver and Bracket Kit Left Full CAT5	1	\$163.29
1d, 2b, 3a, 4 - 12, 13, 14, 15, 16b, 17, 18	502262	Armrest Flip Back w/ Receiver and Bracket Kit Right Full CAT5	1	\$163.29
1b, 1d, 2a, 3a, 3b, 4 - 12, 13, 14 - 18	502281	Armrest Flip Back w/ Receiver and Bracket Kit PAIR Desk CAT5	1	\$326.57
1b, 1d, 2b, 3a, 3b, 4 - 12, 13, 14 - 18	502282	Armrest Flip Back w/ Receiver and Bracket Kit PAIR Full CAT5	1	\$326.57
1a, 2a, 3b, 4 - 12	500734	Armrest Assembly Std Flip Back Desk Left CAT	1	\$146.38
1c, 2a, 3a, 4 - 12	500735	Armrest Assembly Std Flip Back Desk Right CAT	1	\$146.38
1b, 2b, 3b, 4 - 12	500736	Armrest Assembly Std Flip Back Full Left CAT	1	\$146.38
1d, 2b, 3a, 4 - 12	500737	Armrest Assembly Std Flip Back Full Right CAT	1	\$146.38
13, 14, 15	502252	Armrest Flip Back Bracket Kit CAT5	1	\$13.31
16a, 17, 18	502253	Armrest Flip Back Receiver Kit Left CAT	1	\$16.93
16b, 17, 18	502254	Armrest Flip Back Receiver Kit Right CAT	1	\$16.93
2a	100501	Arm Pad Desk	1	\$24.82
2b	100500	Arm Pad Full	1	\$24.82
3a	100687	Armrest Flip Back Panel Fixed Right	1	\$35.87
3b	100688	Armrest Flip Back Panel Fixed Left	1	\$35.87
4	100691	Armrest Flip Back Rear Pivot Bracket	1	\$3.53
5	100690	Armrest Flip Back Front Tube Plug	1	\$1.88
6	100803	Tube End Plug 7/8"	1	\$1.66
7	100689	Armrest Flip Back Release Lever	1	\$4.11

Item Number	Part Number	Description	Quantity	Retail
8	102072	Washer Nylon .234 ID x .406 OD x .062 Thick	1	\$0.56
9	100693	Compression Spring 0.34" OD x 0.70 LG	1	\$1.69
10	100860	M6 x 35 HHCS P/THD ZC	1	\$1.66
11	100723	10 x 3/4 PH Pan HD T-A T/S ZC	4	\$1.10
12	100696	M5 x 30mm PPH Capscrew	2	\$1.66
13	000280	Armrest Flip Back Bracket CAT 5	1	\$8.63
14	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	2	\$1.66
15	100658	M6 Nylock Nut Blk Zn	2	\$1.10
16a	000081	Armrest Flip Back Receiver Left CAT	1	\$10.06
16b	000082	Armrest Flip Back Receiver Right CAT	1	\$10.06
17	000065	M6 x 23.68 Threaded Barrel	1	\$4.34
18	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.66