


Arc Cross Braces - Folding

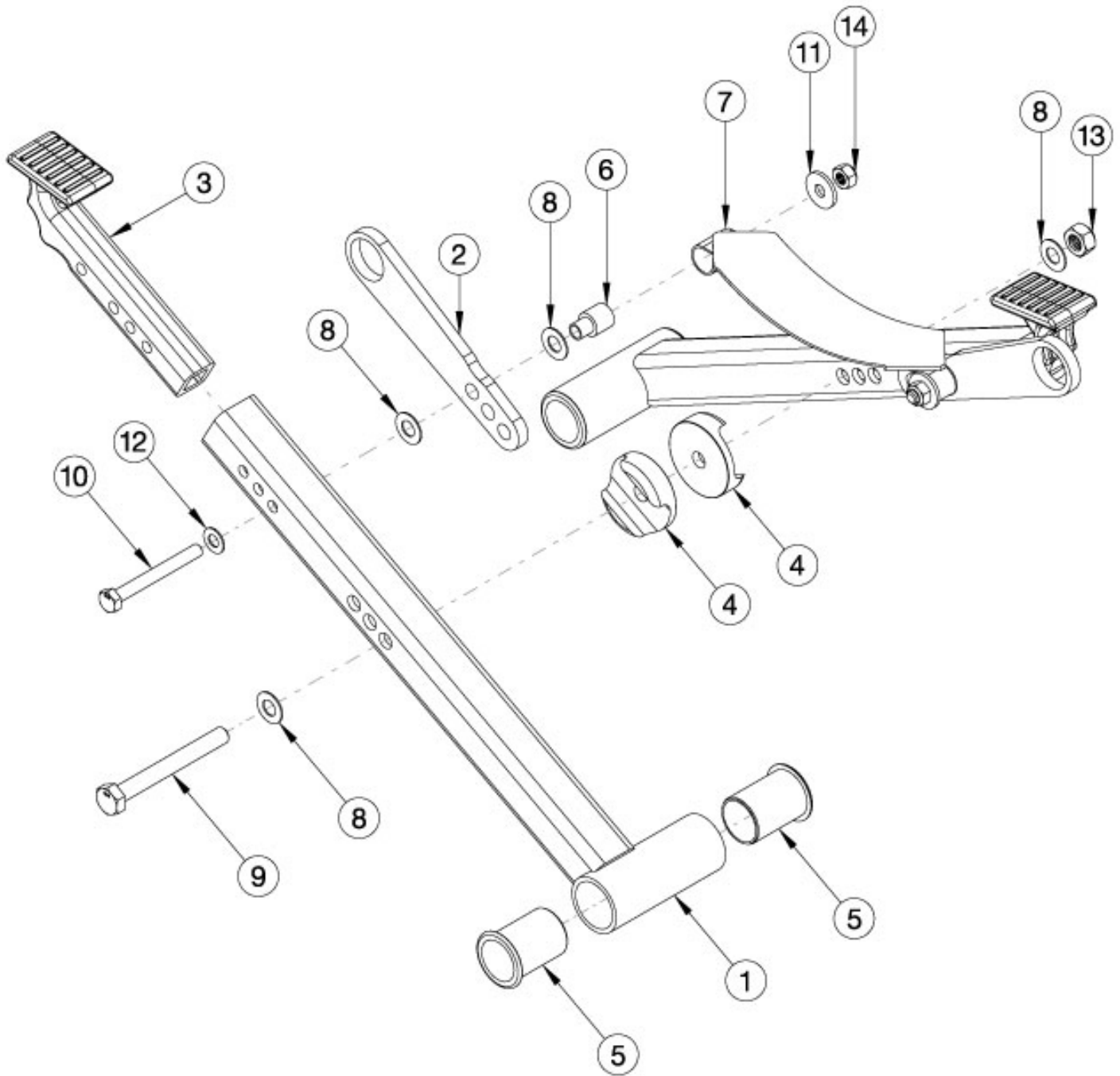

Configuration	Width Range	Item 1 Part #	Item 2 Part #	Item 3 Part #
Small	10-12"	110170	110174	110178
Medium	12-14"	110171	110175	110179
Large	14-16"	110172	110176	110180
Extra Large	16-18"	110173	110177	110181

Item Number	Part Number	Description	Quantity
1a, 2a, 3a, 4-14	112944	Folding Cross Brace Kit Small, Arc <i>Includes parts for both sides</i>	1
1b, 2b, 3b, 4-14	112945	Folding Cross Brace Kit Medium, Arc <i>Includes parts for both sides</i>	1
1c, 2c, 3c, 4-14	112946	Folding Cross Brace Kit Large, Arc <i>Includes parts for both sides</i>	1
1d, 2d, 3d, 4-14	112947	Folding Cross Brace Kit X-Large, Arc <i>Includes parts for both sides</i>	1
1a	110170	X-Tube, Small, Arc	2
1b	110171	X-Tube, Medium, Arc	2
1c	110172	X-Tube, Large, Arc	2
1d	110173	X-Tube, Extra Large, Arc	2
2a	110174	X-Hinge, Small, Arc	2
2b	110175	X-Hinge, Medium, Arc	2
2c	110176	X-Hinge, Large, Arc	2
2d	110177	X-Hinge, Extra Large, Arc	2
3a	110178	Extension, X-Frame, Small, Arc	2
3b	110179	Extension, X-Frame, Medium, Arc	2
3c	110180	Extension, X-Frame, Large, Arc	2
3d	110181	Extension, X-Frame, Extra Large, Arc	2
4	110183	Saddle, X-Tube	2
5	110184	Bushing, Pivot, X-Tube	4
6	110187	X-Hinge Spacer	2
7	000673	Cross Tube Lift Strap, Spark/Arc	1
8	110295	Washer, Nylon, .323ID x .625OD x .031	6
9	101830	M8 x 70 HHCS Blk Zn	1
10	100676	M6 x 55 CLS10.9 DIN931 HHCS ZC	2
11	101853	M6 Washer Blk Zn	2
12	100831	1/2" Washer OD .040" Thick	2

Item Number	Part Number	Description	Quantity
13	110287	Nut, Toplock, M8x1.25, CLS10, BLZ, DIN980V	1
14	100658	M6 Nylock Nut Blk Zn	2