


70 Degree Extension Mount Hanger

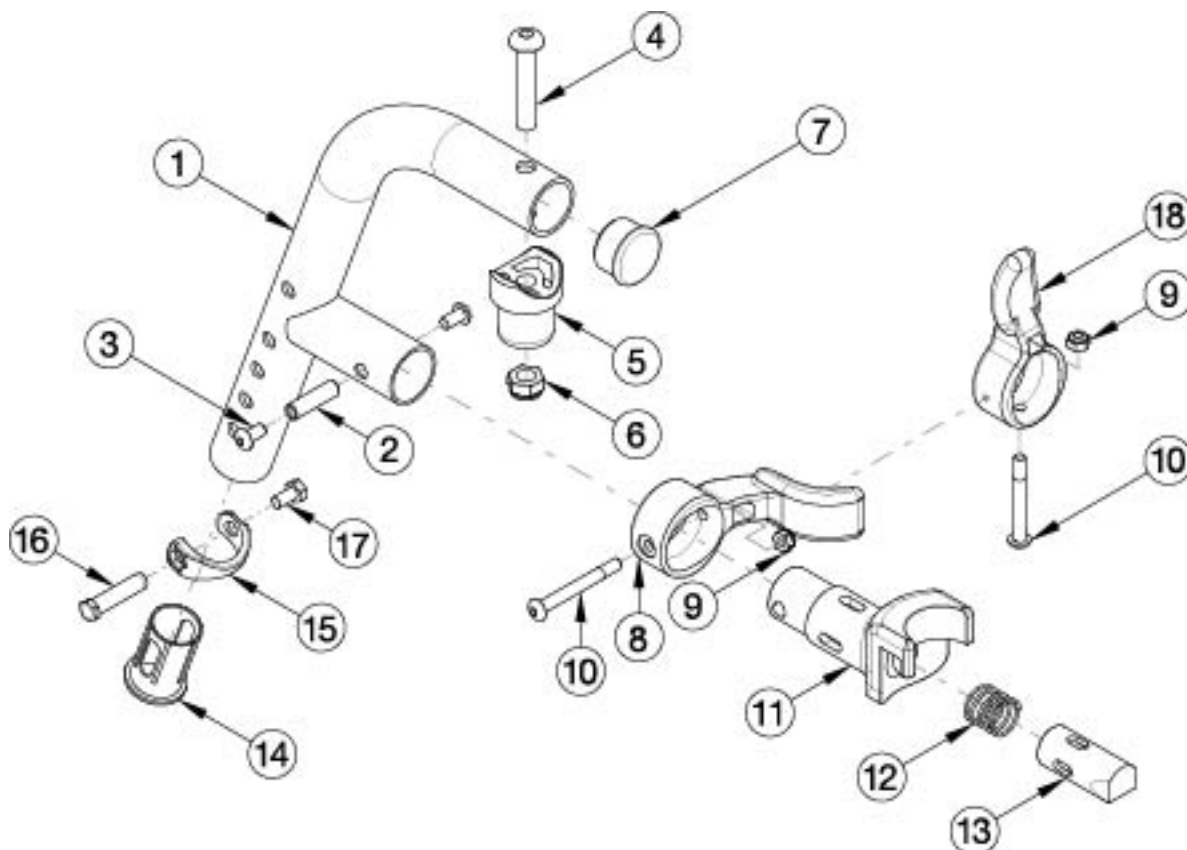

If chair was built with a fixed front frame or hangers were omitted, then latch blocks need to be ordered with hangers. Please reference the "Front Frame" parts manual page for Catalyst chairs and the "Seat Frame" parts manual page for Focus and Flip chairs. If changing degree of hanger, new extension tubes may be needed. If switching from a contracture footrest to a standard hanger, front seat tubes and latch blocks with hardware will be needed.

| Item Number | Part Number | Description | Quantity |
|-----------------------|-------------|--|----------|
| 1 - 17 | 100187_ _ _ | Hanger Assm 70 Extension Mount Classic Latch Left _ _ _ | 1 |
| 1 - 17 | 100186_ _ _ | Hanger Assm 70 Extension Mount Classic Latch Right _ _ _ | 1 |
| 1-7, 9, 10, 12, 14-20 | 504635_ _ _ | Hanger Assm 70 Extension Mount 4-Way - In-Line | 1 |
| 1-7, 9, 10, 12, 14-20 | 504657_ _ _ | Hanger Assm 70 Extension Mount 4-Way - Out Left | 1 |
| 1-7, 9, 10, 12, 14-20 | 504658_ _ _ | Hanger Assm 70 Extension Mount 4-Way - Out Right | 1 |
| 1, 7, 14 | 100180_ _ _ | Hanger Tube Assm 70 Extension Mount _ _ _ | 1 |
| 8 - 13 | 003417 | Classic Hanger Latch Assembly, Left | 1 |
| 8 - 13 | 003416 | Classic Hanger Latch Assembly, Right | 1 |
| 9, 10, 12, 18 - 20 | 003225 | Assembled 4-Way In Line Hanger Release | 1 |

| Item Number | Part Number | Description | Quantity |
|--------------------|-------------|--|----------|
| 9, 10, 12, 18 - 20 | 003225INST | Assembled 4-Way In Line Hanger Release with Instruction | 1 |
| 9, 10, 12, 18 - 20 | 003643 | Assembled 4-Way Out Hanger Release, Left | 1 |
| 9, 10, 12, 18 - 20 | 003643INST | Assembled 4-Way Out Hanger Release, Left with Instruction | 1 |
| 9, 10, 12, 18 - 20 | 003642 | Assembled 4-Way Out Hanger Release, Right | 1 |
| 9, 10, 12, 18 - 20 | 003642INST | Assembled 4-Way Out Hanger Release, Right with Instruction | 1 |
| 2 | 000073 | M5 x 24 Threaded Barrel | 1 |
| 3 | 100669 | M5 x 10 BTN SHCS BLKZC w/Patch | 2 |
| 4 | 100673 | M8 x 45 BTN SHCS BLKZC | 1 |
| 5 | 000032 | Footrest Hanger Pivot Saddle | 1 |
| 6 | 100659 | M8 Nylock Nut Blk Zn | 1 |
| 7 | 100804 | Tube End Plug 1" | 1 |
| 8 | 001158 | Lever Swing Away Short | 1 |
| 9 | 100657 | M5 Nylock Nut Blk Zn | 1 |
| 10 | 001374 | M5 x 40 BHCS P/THD BLKZC | 1 |
| 11 | 003226 | Hanger Latch Bracket | 1 |
| 11 | 003226INST | Hanger Latch Bracket with 4-way Release Instruction | 1 |
| 12 | 000119 | Hanger Spring | 1 |
| 13 | 003227 | Hanger Latch Pin | 1 |
| 13 | 003227INST | Hanger Latch Pin with 4-Way Release Instruction | 1 |
| 14 | 000034 | Sleeve 3/4" x 1 1/2" | 1 |
| 15 | 000988 | Fastener Retainer | 1 |
| 16 | 000987 | Threaded Fastener | 1 |
| 17 | 200427 | M5 x 10 CLS8.8 HHCS ZC w/Patch DIN933 | 1 |
| 18 | 003228 | Hanger Latch, 4-Way, Lever | 1 |
| 18 | 003228INST | Hanger Latch, 4-Way, Lever with Instruction | 1 |

| Matte Finish | |
|--------------|---------------------|
| Code | Color Name |
| CGM | Charcoal Gray |
| EGM | Emerald Green |
| GGM | Graphite Gray* |
| IBM | Iridescent Burgundy |
| INM | Iridescent Navy |
| AGM | Military Green** |
| RRM | Retro Red |
| RBM | Royal Blue |
| SBM | Shadow Black |
| SWM | Smooth White |

* Discontinued Color

** Formerly called Army Green

| Gloss Finish | |
|--------------|------------------------|
| Code | Color Name |
| BLK | Black |
| BLU | Blue |
| SKY | Blue Skies |
| BRO | Bright Orange* |
| PNK | Bubblegum Pink |
| BUR | Burgundy* |
| CBL | Candy Blue |
| PUR | Candy Purple |
| RED | Candy Red |
| COP | Copper |
| DPR | Deep Purple* |
| EBL | Electric Blue* |
| GRN | Forest Green* |
| FUS | Fuchsia* |
| GSM | Granny Smith |
| MTF | Metallic Fuchsia* |
| MOR | Metallic Orange |
| ORG | Orange* |
| SAP | Sapphire* |
| SIL | Silver |
| TEA | Teal |
| YLW | Yellow |
| FLM | Graphic Image Transfer |