Leckey Bath Chair
Leckey Duschliege
Sdraio da Bagno Leckey
Transat de bain Leckey Bath Chair

LECKEY[®]

USER MANUAL
GEBRAUCHSANLEITUNG
USER GUIDA D'USO
GUIDE DE L'UTILISATEUR

Otto Bock[®]
QUALITY FOR LIFE

0.1 Preface

The Leckey Bath Chair and shower trolley has been designed to reach new levels of functionality and adjustability for the use in the bath and shower. This manual shows you how you can quickly and easily make use of all of the functions. The instructions on the safety and maintenance of the product will ensure that you will enjoy the use of this product for a long time.

0.2 Intended Use

The Leckey Bath Chair is for use in the bath or the shower if used with a shower trolley. The Leckey Bath Chair is intended for children with disabilities from the age of 1 to 18 years old who may have a lack of trunk control or excessive muscle tension due to conditions such as cerebral palsy and or those who are unable to support themselves during the bathing process.

0.3 Declaration of Conformity

James Leckey Design Ltd. as manufacturer with sole responsibility declares that the Leckey Bath Chair conforms to the requirements of the 93/42/EEC Guidelines.

0.4 Terms of Warranty

The Warranty applies only when the product is used according to the specified conditions and for the intended purposes, following all manufacturer's recommendations (also see our general terms of sales, delivery and payment).

Getting Started

1.1

1.0 HOW TO UNPACK AND ASSEMBLE

Congratulations on purchasing your Leckey Bath Chair. Your Bath Chair will come bubble wrapped, flat packed and without the cover attached. To attach the cover position the bath seat with the calf support in the maximum position as shown, then firstly slip the cover over the backrest and then secondly the calf support. Then loop the straps in through the frame securing the fabric under the seat base as shown in 1.2.

Your Leckey Bath Chair will come with a hip belt already attached and chest belt with lateral supports. Additional straps as ordered will also be included.

PLEASE CHECK THAT YOU HAVE ALL THE PARTS YOU ORDERED.


1.2

2.0 HOW TO ADJUST

2.1 HEIGHT ADJUSTMENT

The Leckey Bath Chair can be adjusted in height to allow for a range of positions from flat in the bath to over bath height. The height is adjusted by pulling down on both sets of levers (a) and adjusting the front and back legs to the required position. Once the required position is obtained simply release the levers (a). For stability reasons we do not recommend that both sets of legs be adjusted to their highest position at the same time.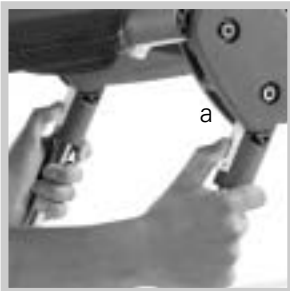

2.1

2.2 TILT IN SPACE

The independent height adjustability of the Leckey Bath Chair front and back legs allows the seat to be adjusted for a range of tilt in space positions. These adjustments are carried out in the exact same way as the height adjustment described above.

2.2


2.3

2.3 BACK RECLINE ADJUSTMENT

The back recline of the Leckey Bath Chair can be adjusted to four positions from horizontal to 90 degrees. The back recline is adjusted by pulling the lever, moving to the required position and then releasing the lever, ensuring it slots back into position.

2.3.1 CALF SUPPORT ADJUSTMENT

The calf support can be adjusted into four positions by pulling down on the levers (a), moving to the required position and then releasing the lever into position.

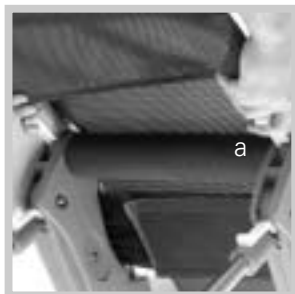2.3.1

2.3.2 HIP BELT

The hip belt simply secures around the user with Velcro.

2.3.3 CHEST BELT

To attach the chest belt, loop it around the back of the seat and Velcro securely at the front around the user. The strap also has two foam wedges attached which may be used to position the user more securely. To do so, position the foam pads under the user's armpit, while looping the strap around the back of the frame and secure with Velcro at the front as shown. The lateral supports can also be positioned out of the way at the back of the seat if the user does not require this level of support.

2.3.3

3.0 ACCESSORIES

In addition to the hip belt and chest belt, an abduction strap and head laterals are also available for the Leckey Bath Chair

3.1 ABDUCTION STRAP

To attach the abduction strap, simply loop around the tubular frame and secure at the back with Velcro. The legs are then positioned and secured by the smaller Velcro straps.


3.1

3.2 HEAD LATERALS

The Head laterals are secured in the same way by simply looping around the tubular frame and securing at the back with Velcro.

Shower Trolley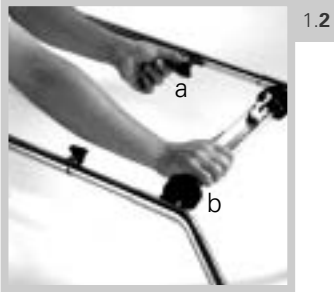

SHOWER TROLLEY

- 1.1 Congratulations on purchasing a shower trolley for the Leckey Bath Chair. This will allow you to simply convert your Leckey Bath Chair into a shower chair. Your shower trolley will arrive complete and packaged in bubble wrap.

PLEASE CHECK THAT YOU HAVE ALL THE PARTS YOU ORDERED

- 1.2 To attach the Leckey Bath Chair to the shower trolley, simply alter the length of the frame, to correspond with the adjusted position of the legs on the Leckey Bath Chair. This is done by loosening the bolts 'a' and adjusting the bar 'b' to the required position. Once in position re-tighten the bolts 'a' securely.
- 1.3 Then simply drop the front and back legs of the Leckey Bath Chair into the channels as shown, fastening the Velcro straps securely around the leg tubes.

Safety Instructions

1. The weight limit on all four sizes of seat is 72kg. This should NEVER be exceeded.
2. With the exception of the back recline adjustment the advance bath seat should not be adjusted whilst the user is in the seat. If adjusting the back recline with a user in the seat ensure while the lever is pulled up that the backrest of the chair is secure by holding onto the handles at the top of the bath seat frame.
3. We recommend that children should not be left unattended at any time whilst using Leckey equipment.
4. If you are in any doubt as to the safe use of your Leckey Advance Bath seat or Shower Trolley, please cease using the product immediately and contact your local dealer.
5. We recommend that when the shower trolley is stationary that all four castors are locked and facing away from the trolley base.
6. Although mobility is an important feature of this product we do not recommend that users be moved over uneven surfaces whilst in the equipment. All due care and attention should be taken if transporting the user in the shower chair from one room to another.
7. If moving your shower chair from one room to another please steer by gripping onto the handles at the top of the backrest.

How to Maintain

1. The bath seat is manufactured from stainless steel and plastic. When cleaning the bath seat frame we would recommend that you use warm water and a non-abrasive detergent.
2. The bath seat cover can be removed and machine-washed at 60 degrees. To remove the cover reverse the process of attaching the cover as described in 1.0 "How to unpack and assemble"
3. At least once a week check that all fixings and fastenings are in good condition.
4. We recommend that you do not use any type of medication during the bathing process as this may contribute to the breakdown of the adhesive glue used to bond the moulding to the stainless steel leg.

Re-issuing Leckey Products

1. Most Leckey product have been assessed and ordered to meet individual needs, therefore we recommend that before a product is being re-issued a compatibility check has been carried out.
2. For hygiene reasons when reissuing the bath seat we recommend that a new set of covers is ordered.
3. Ensure that the product has been cleaned thoroughly.
4. A complete safety check of the product must be carried out prior to re-issuing, this must include the following checks:

Check that all locking mechanisms are functioning properly see 2.1,2.2,2.3,2.4

If Bath Seat is used in conjunction with shower Trolley, please check castors are moving freely and are in good condition.

Check that all locking bolts are working properly and in place.

5. Where possible please ensure that a copy of the relevant instruction manual is kept with the product and passed onto the new user.

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Otto Bock HealthCare GmbH