

USING THE JACK



Category:	ACCESSORIES	Doc. No:	6725	Auth'd	CD	Rel. Date	28APR21	Issue	A
-----------	-------------	----------	------	--------	----	-----------	---------	-------	---

Find a level surface on which to work. The jack works best and is safest when you use it on flat, even ground.

Remove the occupant from the wheelchair.

Before using the jack, ensure drive wheels are engaged.

Position Jack underneath the wheelchair as shown in the pictures below, ensuring the as much surface of the lifting plate as possible is in contact with the lifting surface.

Raise wheelchair by lowering jack. Exert some forwards pressure in addition to the downward pressure.

Before working on the wheelchair ensure the chair is fully on the jack

Do not put any part of your body underneath the wheelchair while it is on the jack; in the case that the wheelchair falls, it could cause serious injury.

Lower the wheelchair slowly by lifting the end of the jack, carefully keep hold of the handle in case the wheelchair drops suddenly.



Magic 360 – note under front bogie arm pivot

USING THE JACK

Category:	ACCESSORIES	Doc. No:	6725	Auth'd	CD	Rel. Date	28APR21	Issue	A
-----------	-------------	----------	------	--------	----	-----------	---------	-------	---



Frontier V6



Extreme X8 – note turn front wheels away from jack



Frontier V4 RWD

USING THE JACK



Category:	ACCESSORIES	Doc. No:	6725	Auth'd	CD	Rel. Date	28APR21	Issue	A
-----------	-------------	----------	------	--------	----	-----------	---------	-------	---



Frontier V4 FWD

Version	Description of Change	DCR No.	Author:	Reviewed By:	Date
A	Initial Release	1450	LD	CD	28APR21