


Clik / Rogue XP Height Adjustable Integrated Handle

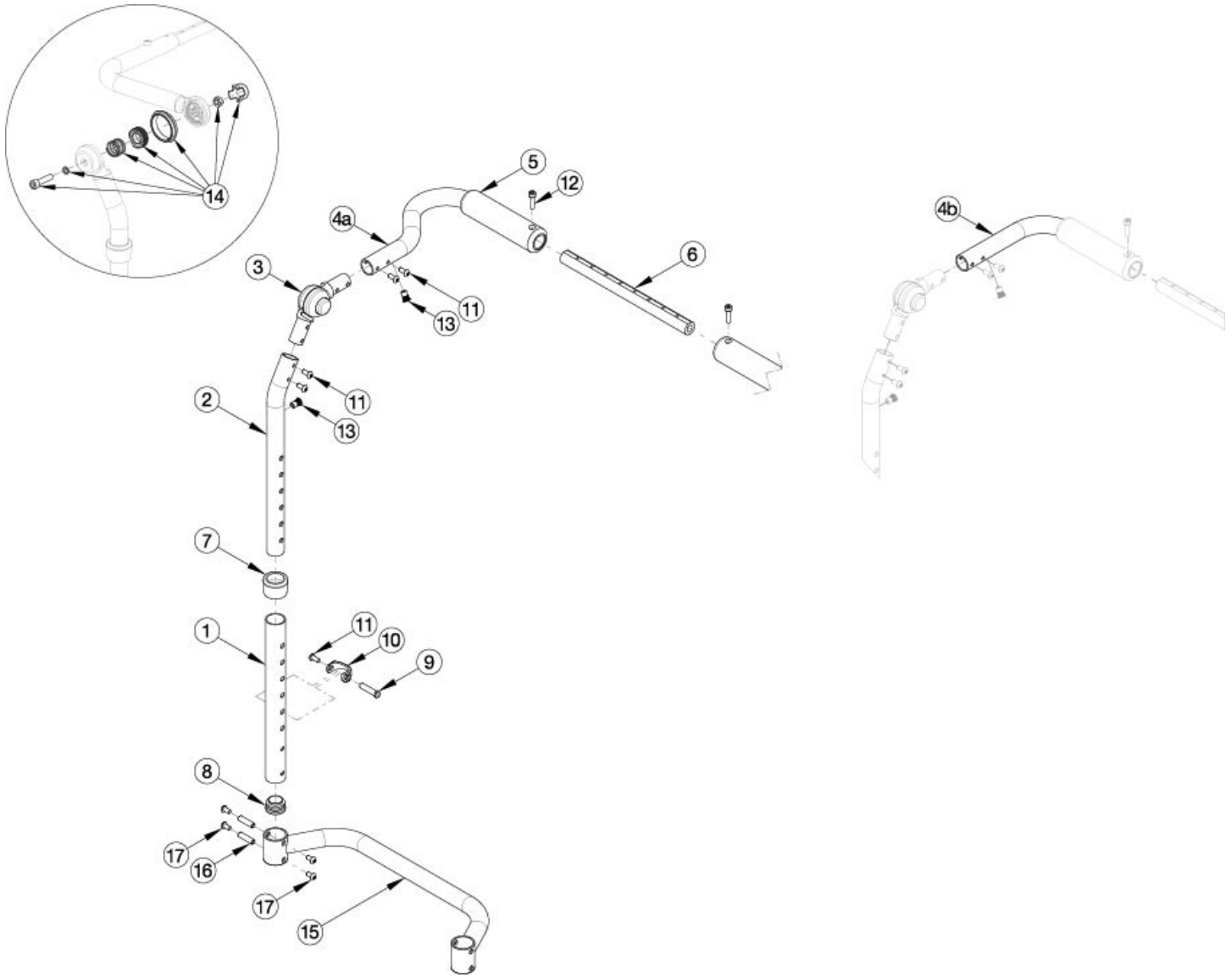

For Backrest heights of 13"-18" use short range kits, for Backrest heights of 15"-22" use tall range kits.

Item Number	Part Number	Description	Quantity
2a, 3, 4a, 5, 11, 13	122182	Upper Tube Assembly, Short, Right, 10-13W <i>Used with Short Range Integrated Stroller Handle</i>	1
2a, 3, 4b, 5, 11, 13	122183	Upper Tube Assembly, Short, Right, 14-18W <i>Used with Short Range Integrated Stroller Handle</i>	1
2b, 3, 4c, 5, 11, 13	122184	Upper Tube Assembly, Short, Left, 10-13W <i>Used with Short Range Integrated Stroller Handle</i>	1
2b, 3, 4d, 5, 11, 13	122185	Upper Tube Assembly, Short, Left, 14-18W <i>Used with Short Range Integrated Stroller Handle</i>	1

Item Number	Part Number	Description	Quantity
2c, 3, 4a, 5, 11, 13	122186	Upper Tube Assembly, Tall, Right, 10-13W <i>Used with Tall Range Integrated Stroller Handle</i>	1
2c, 3, 4b, 5, 11, 13	122187	Upper Tube Assembly, Tall, Right, 14-18W <i>Used with Tall Range Integrated Stroller Handle</i>	1
2d, 3, 4c, 5, 11, 13	122188	Upper Tube Assembly, Tall, Left, 10-13W <i>Used with Tall Range Integrated Stroller Handle</i>	1
2d, 3, 4d, 5, 11, 13	122189	Upper Tube Assembly, Tall, Left, 14-18W <i>Used with Tall Range Integrated Stroller Handle</i>	1
1a, 7 - 11	122190	Lower Tube Assembly, Short <i>Used with Short Range Integrated Stroller Handle</i>	2
1b, 7 - 11	122191	Lower Tube Assembly, Tall <i>Used with Tall Range Integrated Stroller Handle</i>	2
9 - 11	122192	Height Adjustable Hardware Assembly	2
6, 12	122193	Growth Strut Assembly	1
1a, 2a, 2b, 3, 4a, 4c, 5 - 13	122194	Height Adjustable Back Retrofit Assembly, Short, 10-13W <i>Used with Short Range Integrated Stroller Handle</i>	1
1a, 2a, 2b, 3, 4b, 4c, 5 - 13	122195	Height Adjustable Back Retrofit Assembly, Short, 14-18W <i>Used with Short Range Integrated Stroller Handle</i>	1
1b, 2c, 2d, 3, 4a, 4c, 5 - 13	122196	Height Adjustable Back Retrofit Assembly, Tall, 10-13W <i>Used with Tall Range Integrated Stroller Handle</i>	1
1b, 2c, 2d, 3, 4b, 4d, 5 - 13	122197	Height Adjustable Back Retrofit Assembly, Tall, 14-18W <i>Used with Tall Range Integrated Stroller Handle</i>	1
2a, 13	122198	Upper Tube w/Rivnut, Short, Right	1
2b, 13	122199	Upper Tube w/Rivnut, Short, Left	1
2c, 13	122200	Upper Tube w/Rivnut, Tall, Right	1
2d, 13	122201	Upper Tube w/Rivnut, Tall, Left	1
4b, 5, 13	122202	Push Handle Assembly, Standard, Right <i>Used on Widths 14" - 18"</i>	1
4d, 5, 13	122203	Push Handle Assembly, Standard, Left <i>Used on Widths 14" - 18"</i>	1
4a, 5, 13	122204	Push Handle Assembly, Pediatric, Right <i>Used on Widths 10" - 13"</i>	1
4c, 5, 13	122205	Push Handle Assembly, Pediatric, Left <i>Used on Widths 10" - 13"</i>	1

Item Number	Part Number	Description	Quantity
1a	121860	Tube, Lower, HA Back, Short	2
1b	121861	Tube, Lower, HA Back, Tall	2
3	116066	Locking Hinge Assembly	2
5	001098	Foam Push Handle Tilt	2
6	002578	Growth Strut, Stroller, Short	1
7	001083	Bushing Height Adjustable Back Tilt	2
9	000987	Threaded Fastener	2
10	000988	Fastener Retainer	2
11	102799	M5 x 12, BHCS, BZ w/Patch	10
12	101843	M5 x 20 SHCS Blk Zn	2
16	000073	M5 x 24 Threaded Barrel	4
17	103524	M5 x 10 BTN SHCS BLK ZN	8