


Flip Tipping Lever

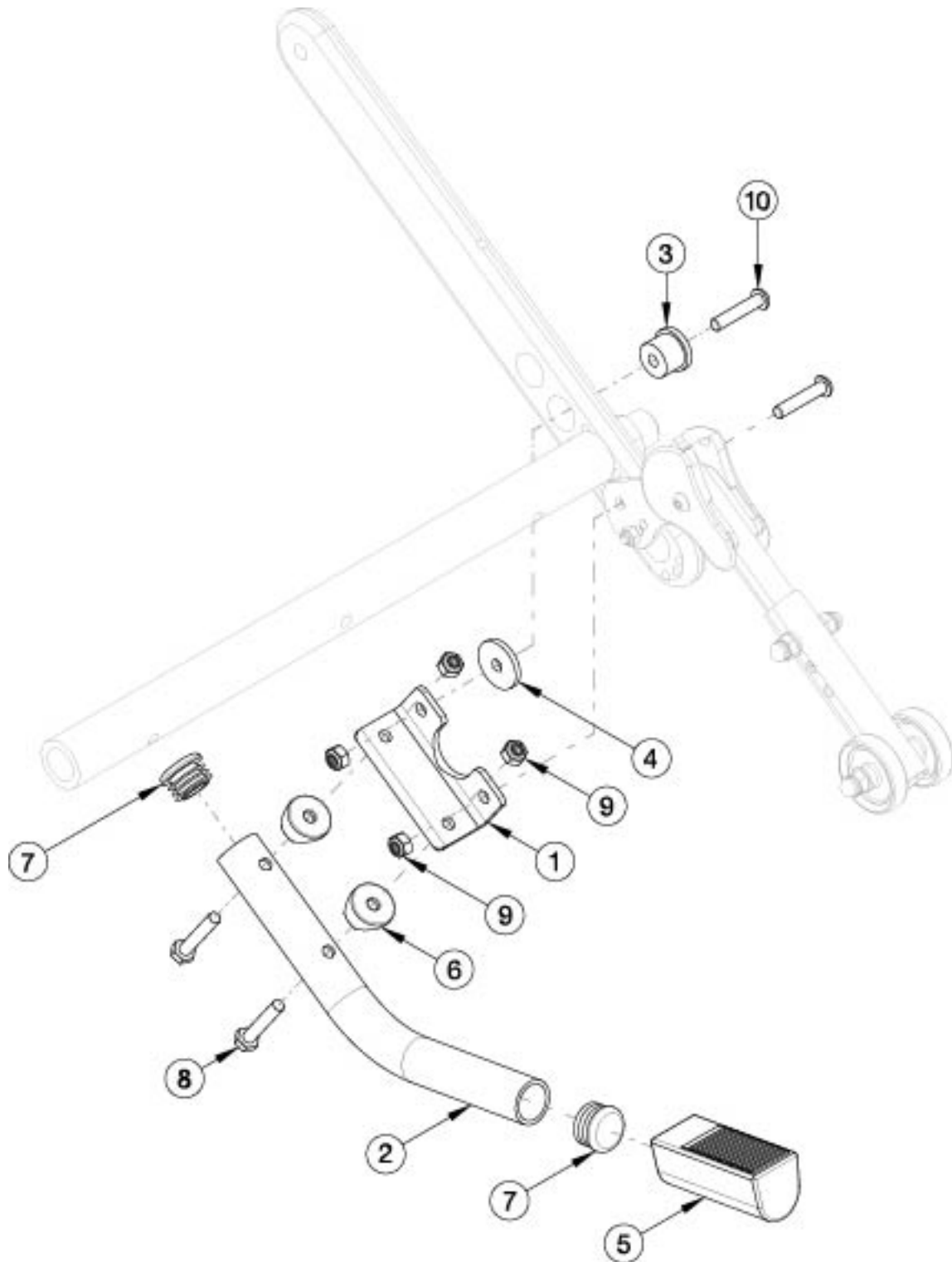

Item Number	Part Number	Description	Quantity
1 - 10	120816	Tipping Lever Assembly Flip	1
1,3,4,6,8-10	120817	Tipping Lever Mounting Hardware Flip	1

Item Number	Part Number	Description	Quantity
1	119590	Mount Bracket, Tipping Lever	1
3	119592	Bushing, Mount, Tipping Lever	1
4	119593	Washer, 1.00 OD x .25 ID x .120	1
6	000054	Saddle 7/8"	2
7	100803	Tube End Plug 7/8"	2
8	102386	M6 x 35mm HHCS Blk Zc	2
9	100658	M6 Nylock Nut Blk Zn	4
10	102625	M6 X 30 BHCS BLK ZC	2