


# Aeromesh™ Calf Strap and Calf Panel


## INSTALLATION AND USER'S INSTRUCTIONS

ART NOS. SP102S, SP102M, SP102L, SP103S, SP103M & SP103L

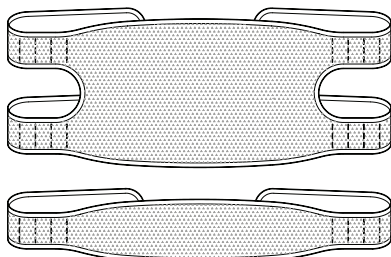

### PACKAGE CONTENTS

- Aeromesh™ Calf Panel
- or
- Aeromesh™ Calf Strap
- Adhesive Backed Hook and Loop Patch

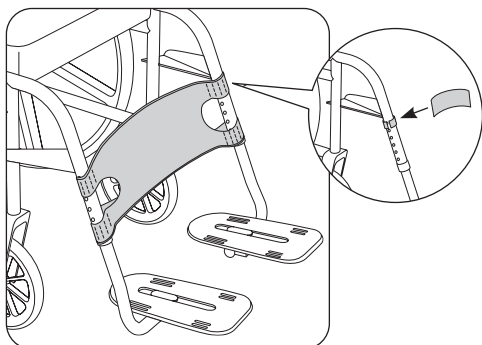

### PERIODIC SAFETY AND PERFORMANCE

**CHECK:** To ensure user safety, this product must be checked periodically for function and signs of wear. If the product does not function correctly or if significant wear is found in the buckles, mounting points, webbing, padding, or stitches, stop using it and contact your supplier for qualified repair or replacement by Bodypoint. Under no circumstance should this product be altered or repaired by unqualified persons — health and safety depend on it!

**⚠ WARNING!** If a serious incident occurs related to the use of this product, it should be reported to the manufacturer (Bodypoint, Inc.) and the local Competent Authority.

**🧺🔥🌀🧺 CLEANING:** Machine wash, hot, 60°C (140°F). Do not bleach. Tumble dry, low temperature, or drip dry. Do not iron. (Placing the product inside a cloth bag during washing helps prevent scratches to the product and the machine.)

**SCRAP/DISPOSAL:** Product is made of materials which can be disposed of safely without special precautions at the end of its useful life.

**WARRANTY:** This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products, and a list of distributors outside the USA go to [www.bodypoint.com](http://www.bodypoint.com).



Bodypoint Europe BV  
Kerkstraat 29  
7396 PD Tervolde  
The Netherlands



558 First Ave. S., Suite 300 | Seattle, WA 98104 | 206.405.4555 | 800.547.5716 | [www.bodypoint.com](http://www.bodypoint.com)