


Axiom Adjustable Swing Away Laterals for AXPB

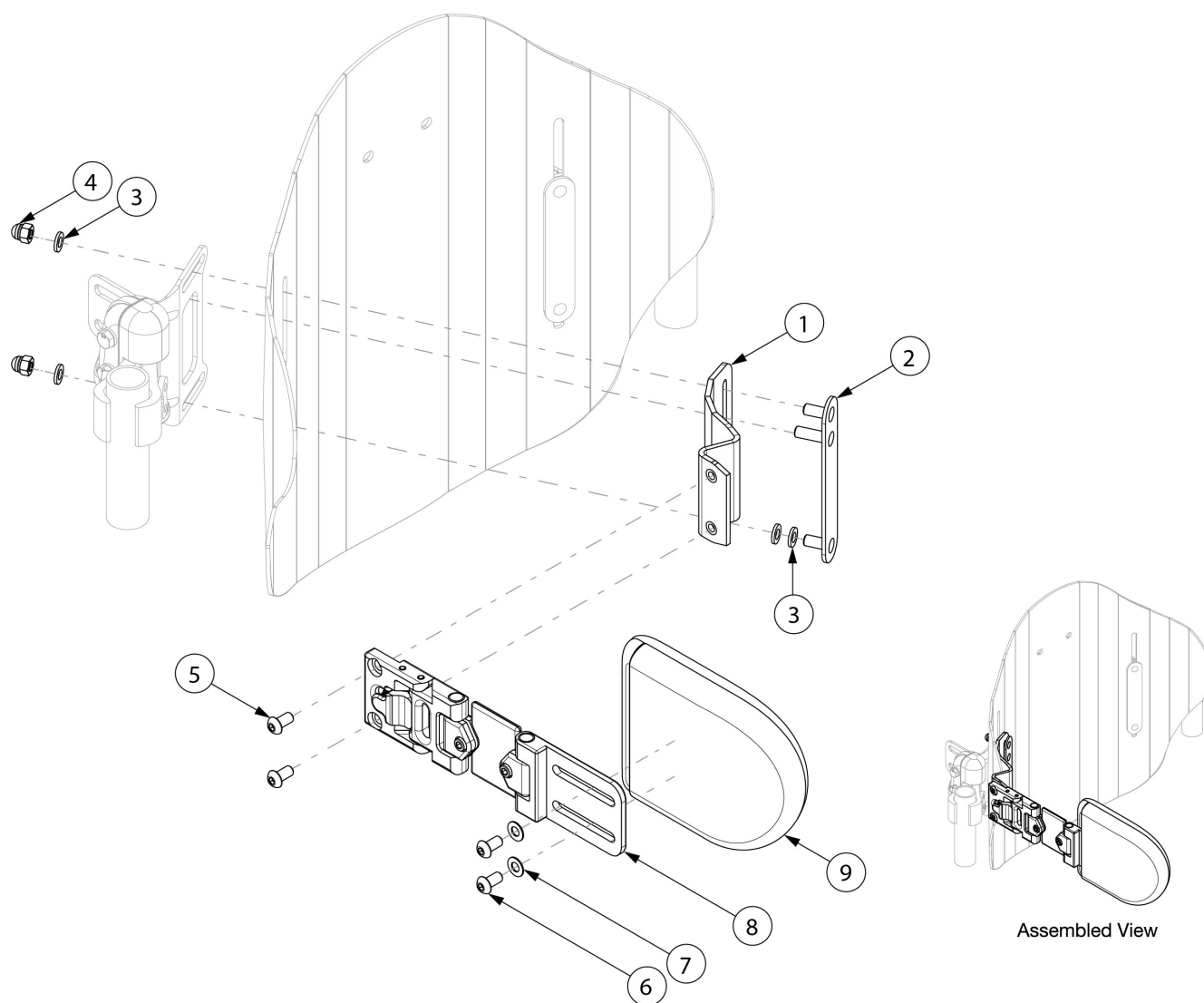

Item Number	Part Number	Description	Quantity	Retail
1a	004884	Axiom PB Back Lateral Mount, Swing Away, Right	1	\$37.49
1b	004885	Axiom PB Back Lateral Mount, Swing Away, Left	1	\$37.49
2	004889	Axiom Back Lateral Stud Plate	1	\$16.07
3	100746	M6 Flat Washer Blk Zn	2	\$1.66
4	100558	M6 NYL INS Domed Cap L/N ZC	2	\$1.10
5	104031	1/4-20 X 3/8 Stainless Button Head Cap Screw	1	\$3.21

Item Number	Part Number	Description	Quantity	Retail
6	104017	1/4-20 X 1/2 Stainless Button Head Cap Screw	2	\$3.21
7	102803	Washer Flat .250 x .500 x .027 HX BLZ	2	\$1.66
8	104488	Lateral, Adjustable Thin Link Swingaway Mount	1	\$236.25
9a	004684	Axiom Back Lateral Pad 3x4	1	\$85.00
9b	004685	Axiom Back Lateral Pad 4x5	1	\$85.00
1b, 2 - 8	114481	Posterior - Adjustable Swing Away Lateral Left <i>Pad not included</i>	1	\$305.00
1a, 2 - 8	114482	Posterior - Adjustable Swing Away Lateral Right <i>Pad not included</i>	1	\$305.00
1a, 1b, 2 - 8, 9a	LAS3X4P	3 x 4 Lateral Pads with Adjustable Swing Away Mounting Hardware for Axiom Posterior Back (Pair)	1	\$790.00
1a, 1b, 2 - 8, 9b	LAS4X5P	4 x 5 Lateral Pads with Adjustable Swing Away Mounting Hardware for Axiom Posterior Back (Pair)	1	\$790.00