


Focus CR Adjustable Height Rigidizer Bar

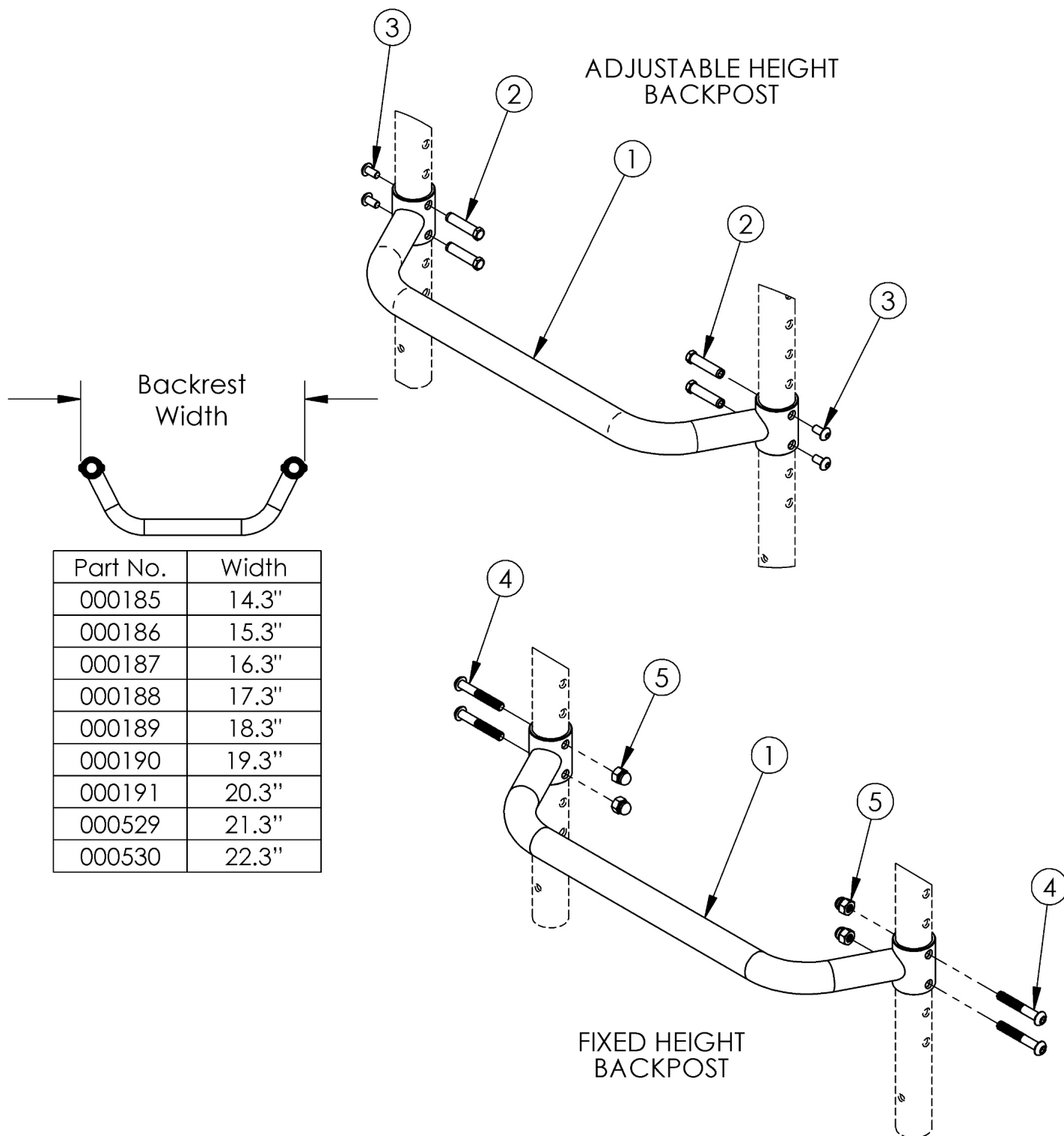

Adjustable Height Rigidizer Bar based on seat width If changing from another style backrest or changing back cane height and the hand tilt is present, a new Hand Tilt Cable/Trigger Kit may be required.

Item Number	Part Number	Description	Quantity
1a	000185	Adjustable Height Rigidizer Bar 14" Wide	1
1b	000186	Adjustable Height Rigidizer Bar 15" Wide	1
1c	000187	Adjustable Height Rigidizer Bar 16" Wide	1
1d	000188	Adjustable Height Rigidizer Bar 17" Wide	1
1e	000189	Adjustable Height Rigidizer Bar 18" Wide	1
1f	000190	Adjustable Height Rigidizer Bar 19" Wide	1
1g	000191	Adjustable Height Rigidizer Bar 20" Wide	1
1h	000529	Adjustable Height Rigidizer Bar 21"	1
1i	000530	Adjustable Height Rigidizer Bar 22"	1
2	000987	Threaded Fastener	4
3	100669	M5 x 10 BTN SHCS BLKZC w/Patch	4
4	100797	M6 x 40 BHCS BLK ZC	4
5	100558	M6 Nyl Ins Domed Cap L/N Zc	4