


# (Discontinued) Catalyst Depth Adjustable Backrest with Hardware

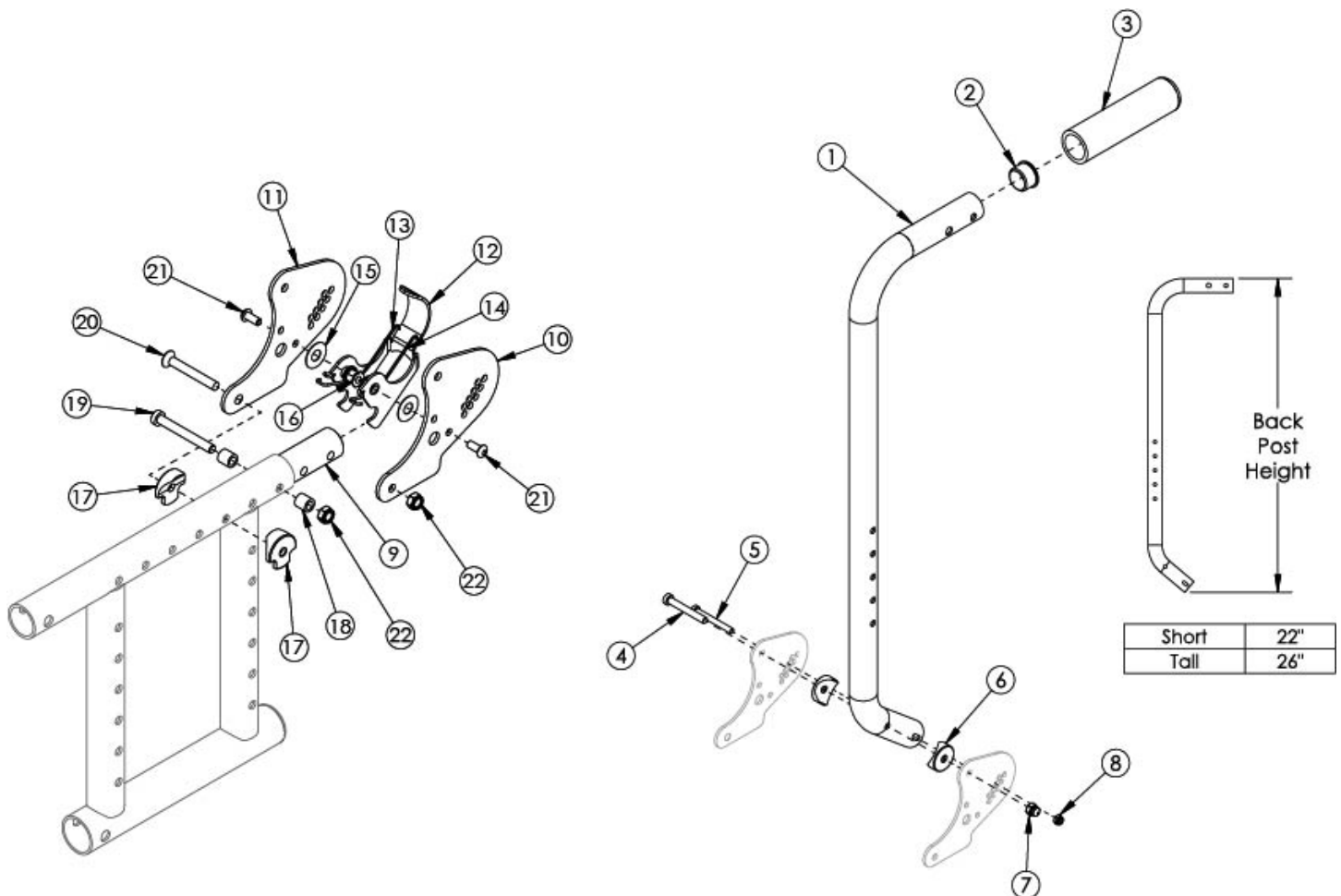

*This backrest style discontinued February 8th, 2021 Not compatible with Drum Brake. A backrest retrofit to the Catalyst Stroller Handle Back Post (Depth Adjustable) is required if retrofitting Drum Brake.*

Item Number	Part Number	Description	Quantity
1a - 9, 10b, 11-22	111330	Back Post Depth Adjustable with Push Handle Short Assm with All Hardware CAT/ SPARK - Right Side	1
1b - 9, 10b, 11-22	111332	Back Post Depth Adjustable with Push Handle Tall Assm with All Hardware CAT/ SPARK - Right Side	1
1a, 2, 3	500613	Back Post Depth Adjustable with Push Handle Short Assm CAT/ SPARK	1
1b, 2, 3	500614	Back Post Depth Adjustable with Push Handle Tall Assm CAT/ SPARK	1
1a - 8	501206	Back Post Depth Adjustable with Push Handle Short Assm with Hardware CAT/ SPARK	1
1b - 8	501207	Back Post Depth Adjustable with Push Handle Tall Assm with Hardware CAT/ SPARK	1

Item Number	Part Number	Description	Quantity
9, 10b, 11-22	500739	Depth Adjustable Backrest Hardware Assm Right CAT	1
9, 11-22	501227	Depth Adjustable Backrest Hardware Assm SPARK/ OPEN SEAT	1
4-8	111164	Depth Adjustable Backrest Attaching Hardware Kit	1
2	100804	Tube End Plug 1"	1
4	101123	M6 x 45 Low SHCS BLK ZC	1
5	101186	M5 X 45 CLS12.9 DIN912 SHCS P/THD BLZ	1
6	000055	Saddle 1"	2
7	100558	M6 Nyl Ins Domed Cap L/N Zc	1
8	100657	M5 Nylock Nut Blk Zn	1
9	000605	Depth Adjustable Frame Extension Tube, Catalyst/Spark	1
10b	000604R	Depth Adjustable Backrest Plate Inside Right CAT/ SPARK	1
11	000611	Depth Adjustable Backrest Side Plate CAT/ SPARK	2
12	000670	Depth Adjustable Backrest Latch CAT/ SPARK	1
13	000613R	Depth Adjustable Backrest Latch Spring Right CAT/ SPARK	1
14	000613L	Depth Adjustable Backrest Latch Spring Left CAT/ SPARK	1
15	000428	Wear Washer	2
16	000420	Backrest Latch Pivot Bushing	2
17	000612	1" Half Saddle Washer	2
18	000240	Backrest Latch Strike Bushing / Close Mount Handrim Spacer	2
19	101124	M6 x 50 Low SHCS BLK ZC	1
20	101592	M6 x 45 CLS10.9 DIN7991 FHCS BLKZC	1
21	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2
22	100658	M6 Nylock Nut Blk Zn	2