


Liberty IV Pole

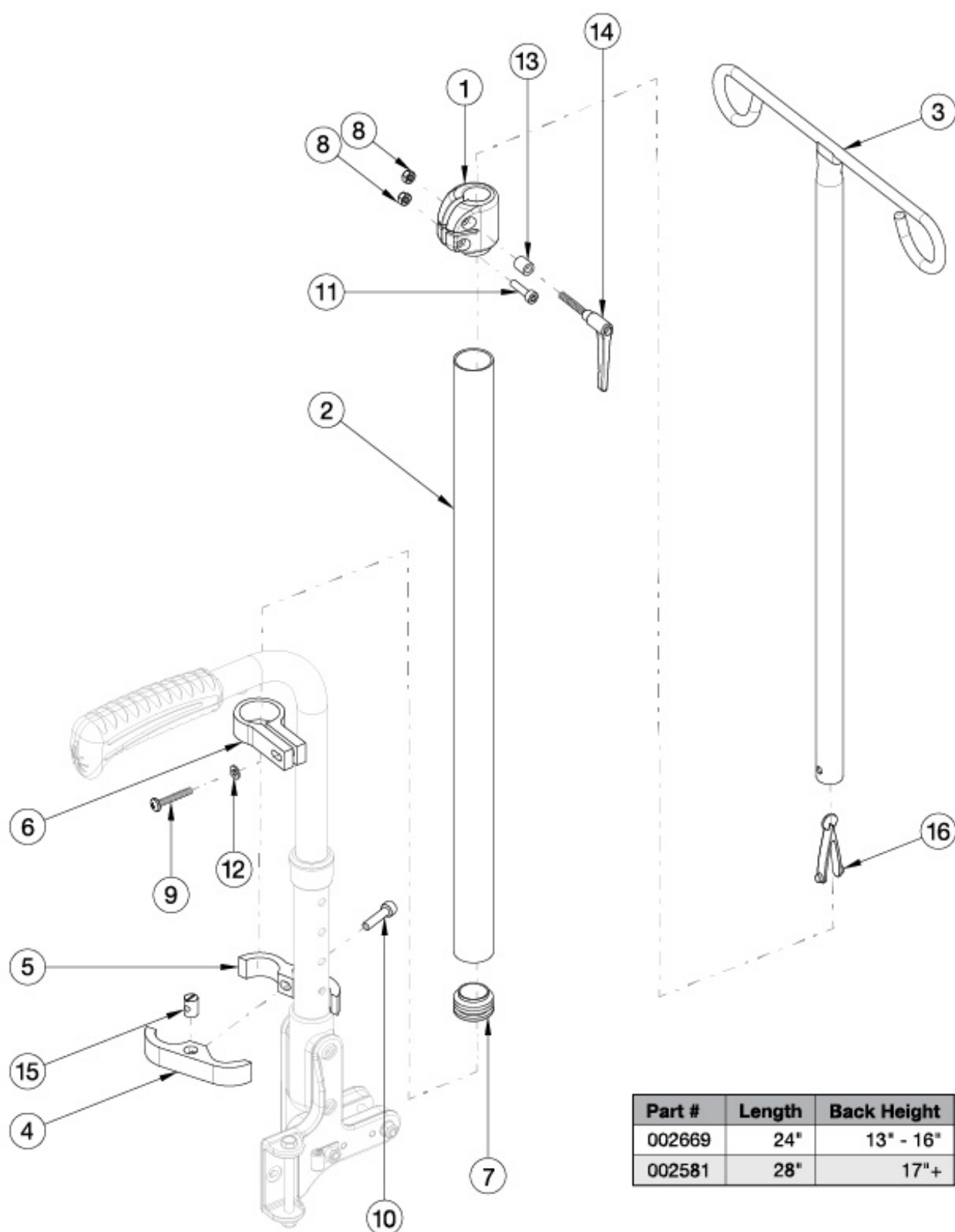

Part #	Length	Back Height
002669	24"	13" - 16"
002581	28"	17"+

Item Number	Part Number	Description	Quantity
1, 2a, 3-16	502405	IV Pole Assembly 13-16" Back Height, Catalyst/Liberty	1
1, 2b, 3-16	502406	IV Pole Assembly 17-20" Back Height, Catalyst/Liberty	1
1	002395	Footrest Clamp Left LW	1
2a	002669	IV Pole Outer Tube 24" Height	1
2b	002581	IV Pole Outer Tube 28" Height	1
3	002585	IV Pole Upper	1
4	002685	IV Pole Clamp 1-1/8" 1" Thru 1-1/4" w/ Cross Dowel Hole	1
5	002686	IV Pole Clamp 1-1/8" 1" Thru 1-1/4"	1
6	002707	IV Pole Clamp Single 1-1/8"	1
7	100532	Tube End Plug 1 1/8"	1
8	100657	M5 Nylock Nut Blk Zn	2
9	100696	M5 x 30mm PPH Capscrew	1
10	100780	M6 x 25 SHCS ZC	1
11	101843	M5 x 20 SHCS Blk Zn	1
12	101856	Flat Washer, M5, Blk Zn	1
13	102766	Spacer 3/8R. X 7/16 x .218 ID.	1
14	102767	Handle wM5 x 32L Threaded Stud	1
15	102768	M6 Barrel Nut	1
16	102769	Snap Button Plastic	1