


Rogue ALX Tipping Lever

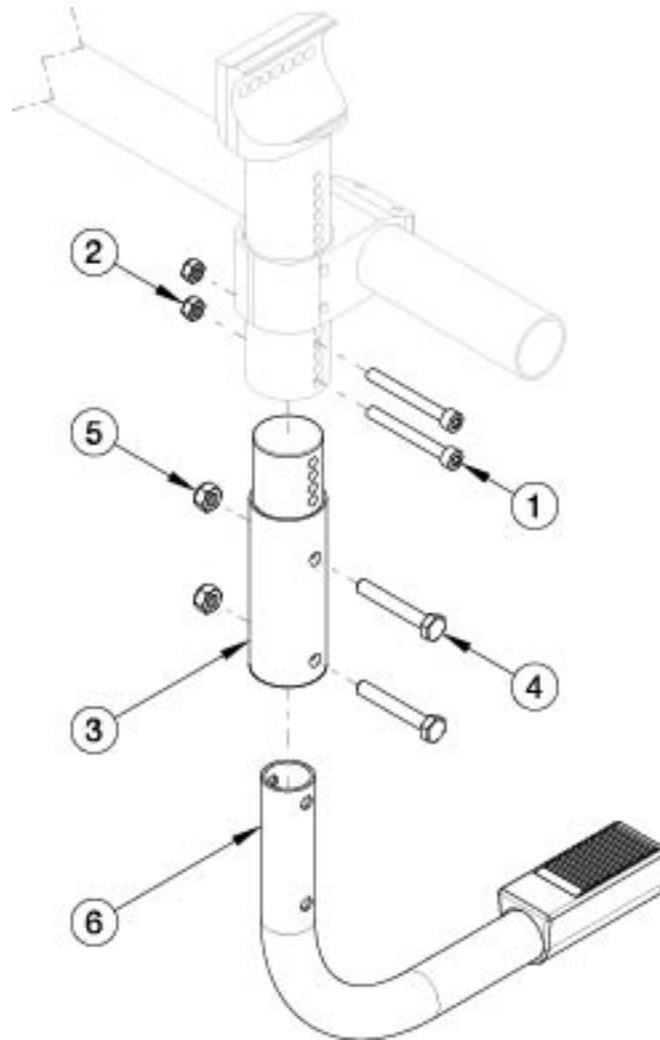

Requires MX Tower Not available with HD option

Item Number	Part Number	Description	Quantity
1 - 6a	119633	Tipping Lever Kit Rouge ALX <i>Used with 22" wheels</i>	1
1 - 6b	120811	Tipping Lever Short Kit Rogue ALX <i>Used with 24"-26" wheels</i>	1
1	101186	M5 X 45 CLS12.9 DIN912 SHCS P/THD BLZ	2
2	100657	M5 Nylock Nut Blk Zn	2
3	119605	Rogue ALX Tipping Lever MX Tower Adapter	1
4	101825	M6 x 40 HHCS Blk Zn	2
5	100658	M6 Nylock Nut Blk Zn	2

Item Number	Part Number	Description	Quantity
6a	119623	Tipping Lever Rigid 3" Assembly <i>Used with 22" wheels</i>	1
6b	119624	Tipping Lever Rigid 3" Assembly Short <i>Used with 24"-26" wheels</i>	1