


# Ethos Backrest Mount and Hardware - Growth

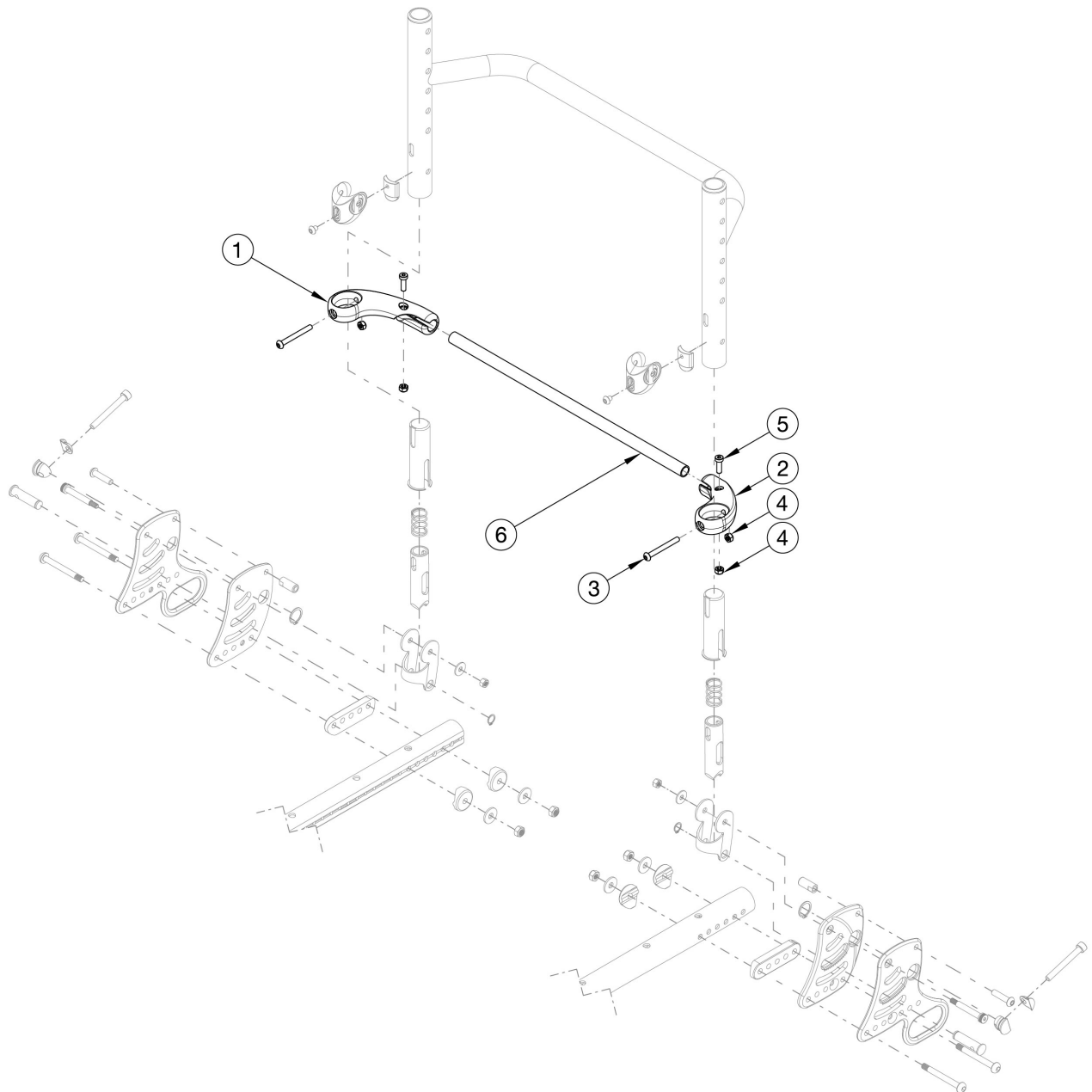

Utilize this page if chair has Rigid Locking Fold Down Back.

Item Number	Part Number	Description	Quantity
1 - 5, 6a	116879	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 12" Wide, Black	1
1 - 5, 6b	116880	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 13" Wide, Black	1
1 - 5, 6c	116881	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 14" Wide, Black	1

Item Number	Part Number	Description	Quantity
1 - 5, 6d	116882	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 15" Wide, Black	1
1 - 5, 6e	116883	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 16" Wide, Black	1
1 - 5, 6f	116884	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 17" Wide, Black	1
1 - 5, 6g	116885	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 18" Wide, Black	1
1 - 5, 6h	116886	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 19" Wide, Black	1
1 - 5, 6i	116887	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 20" Wide, Black	1
1 - 5, 6j	117832_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 12" Wide, Painted	1
1 - 5, 6k	117833_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 13" Wide, Painted	1
1 - 5, 6l	117913_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 14" Wide, Painted	1
1 - 5, 6m	117914_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 15" Wide, Painted	1
1 - 5, 6n	117915_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 16" Wide, Painted	1
1 - 5, 6o	117916_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 17" Wide, Painted	1
1 - 5, 6p	117917_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 18" Wide, Painted	1
1 - 5, 6q	117918_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 19" Wide, Painted	1
1 - 5, 6r	117919_ _ _	Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 20" Wide, Painted	1
3	000142	M5 x 45 BHCS, P/THD, BLZ	2
4	100657	M5 Nylock Nut Blk Zn	4
5	102801	M5 x 16, Low SHCS, Blk Zn	2
6a	115554	Backrest Actuation Bar, 12" Wide, Black	1
6b	115555	Backrest Actuation Bar, 13" Wide, Black	1
6c	115556	Backrest Actuation Bar, 14" Wide, Black	1
6d	115557	Backrest Actuation Bar, 15" Wide, Black	1
6e	115558	Backrest Actuation Bar, 16" Wide, Black	1
6f	115559	Backrest Actuation Bar, 17" Wide, Black	1
6g	115560	Backrest Actuation Bar, 18" Wide, Black	1
6h	115561	Backrest Actuation Bar, 19" Wide, Black	1
6i	115562	Backrest Actuation Bar, 20" Wide, Black	1

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
6j	117921_ _ _	Backrest Actuation Bar, 12" Wide, Painted	1
6k	117922_ _ _	Backrest Actuation Bar, 13" Wide, Painted	1
6l	117923_ _ _	Backrest Actuation Bar, 14" Wide, Painted	1
6m	117924_ _ _	Backrest Actuation Bar, 15" Wide, Painted	1
6n	117925_ _ _	Backrest Actuation Bar, 16" Wide, Painted	1
6o	117926_ _ _	Backrest Actuation Bar, 17" Wide, Painted	1
6p	117927_ _ _	Backrest Actuation Bar, 18" Wide, Painted	1
6q	117928_ _ _	Backrest Actuation Bar, 19" Wide, Painted	1
6r	117929_ _ _	Backrest Actuation Bar, 20" Wide, Painted	1