


Discontinued Clik Caster Mount

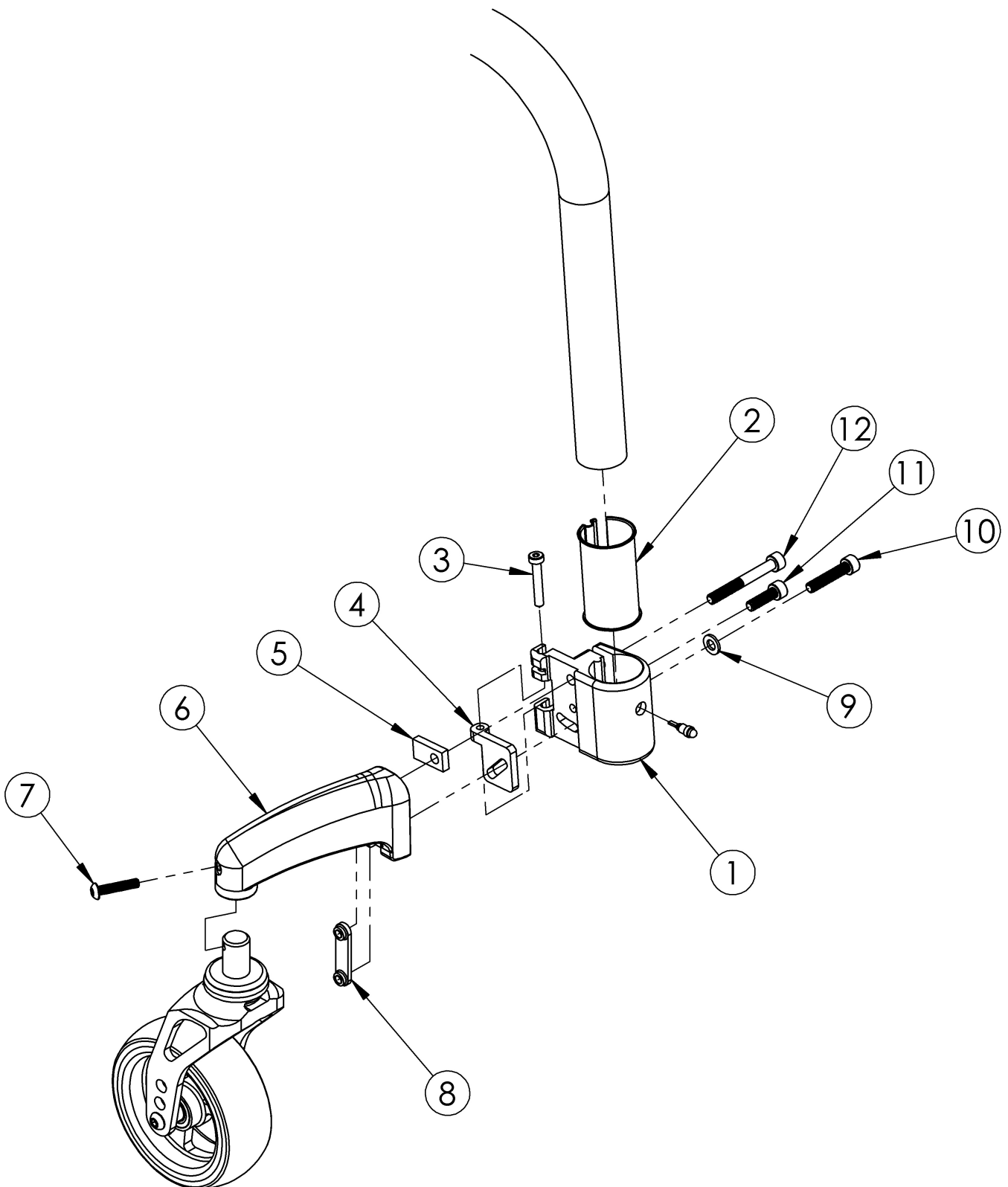

This design discontinued June 1st, 2016

Item Number	Part Number	Description	Quantity	Retail
1a	002074	Caster Arm Mount Right LW	1	\$37.52
1b	002075	Caster Arm Mount Left LW	1	\$37.52
2	002079	Caster Mount Sleeve LW	1	\$1.61
3	102667	M5 X 30 LHCS BK ZC FULL THREAD	1	\$6.30
6a	002561_ _ _ _	Caster Arm Short Right Klik <i>Used with 3" and 4" casters</i>	1	\$48.25
6b	002562_ _ _ _	Caster Arm Short Left Klik <i>Used with 3" and 4" casters</i>	1	\$48.25
6c	002069_ _ _ _	Caster Arm Medium Right Klik <i>Used with 5" casters</i>	1	\$46.84 - \$48.25
6d	002070_ _ _ _	Caster Arm Medium Left Klik <i>Used with 5" casters</i>	1	\$46.84 - \$48.25
6e	002071_ _ _ _	Caster Arm Long Right Klik <i>Used with 6" casters</i>	1	\$48.25
6f	002072_ _ _ _	Caster Arm Long Left Klik <i>Used with 6" casters</i>	1	\$48.25
7	102616	M5 X 25 BHCS BLK ZC	1	\$1.05
8	002462	M6 X 1.0 Dual Hole Nut	1	\$24.65
9	100746	M6 Flat Washer Blk Zn	1	\$1.58
10	102627	M6 X 30 SHCS BLK ZC	1	\$1.05
11	101180	M6 x 20 SHCS BLK ZC	1	\$2.10
12	101472	M6 x 45 SHCS BLK ZC	1	\$1.58