


(Discontinued) Rogue Caster Mount

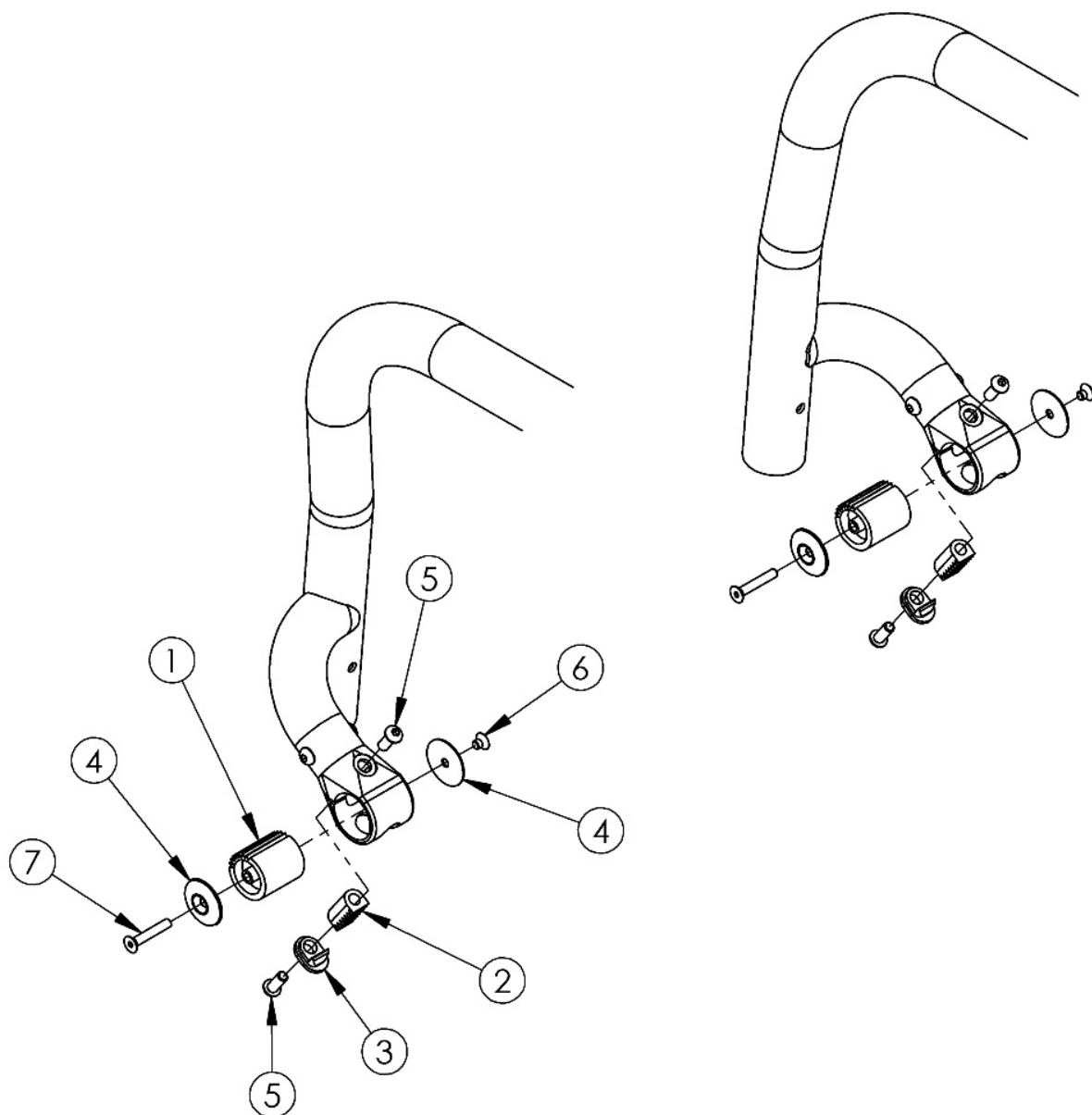

Item Number	Part Number	Description	Quantity
1-7	501225	Caster Pinion Hardware Assm, Rogue	1
1-7	501226	Caster Pinion Hardware Assm PAIR, Rogue	1
1	000964	Caster Mount Pinion Steel RG	1
2	000933	Caster Adjust CG RG	1
3	000934	Pinion Housing Bolt Plate RG	1

Item Number	Part Number	Description	Quantity
4	000932	Caster Pinion Cap RG	2
5	105037	M8x16 CLS10.9 ISO7380 BTN SHCS ENVRBLK W/PRE5	2
6	102304	M5 x 6mm FHCS Blk Zc w/Patch	2
7	102303	M5 x 30mm FHCS Blk Zc w/Patch	2