


# Rogue Backrest Mount and Hardware - Growth

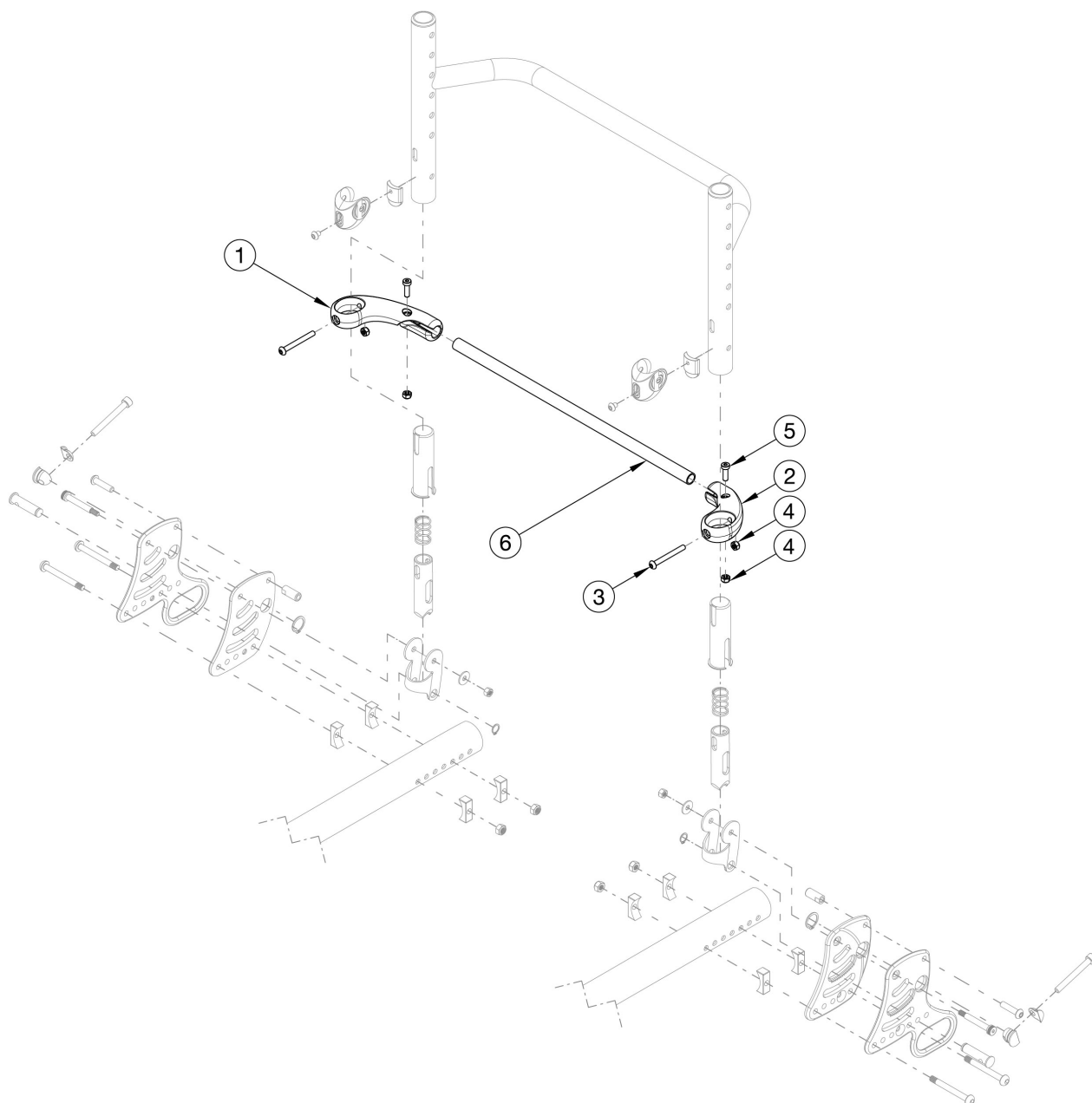

Utilize this page if chair has Rigid Locking Fold Down Back.

| Item Number | Part Number | Description                                                                         | Quantity |
|-------------|-------------|-------------------------------------------------------------------------------------|----------|
| 1 - 5, 6a   | 116879      | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 12" Wide, Black | 1        |
| 1 - 5, 6b   | 116880      | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 13" Wide, Black | 1        |
| 1 - 5, 6c   | 116881      | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 14" Wide, Black | 1        |

| Item Number | Part Number | Description                                                                           | Quantity |
|-------------|-------------|---------------------------------------------------------------------------------------|----------|
| 1 - 5, 6d   | 116882      | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 15" Wide, Black   | 1        |
| 1 - 5, 6e   | 116883      | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 16" Wide, Black   | 1        |
| 1 - 5, 6f   | 116884      | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 17" Wide, Black   | 1        |
| 1 - 5, 6g   | 116885      | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 18" Wide, Black   | 1        |
| 1 - 5, 6h   | 116886      | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 19" Wide, Black   | 1        |
| 1 - 5, 6i   | 116887      | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 20" Wide, Black   | 1        |
| 1 - 5, 6j   | 117832_ _ _ | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 12" Wide, Painted | 1        |
| 1 - 5, 6k   | 117833_ _ _ | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 13" Wide, Painted | 1        |
| 1 - 5, 6l   | 117913_ _ _ | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 14" Wide, Painted | 1        |
| 1 - 5, 6m   | 117914_ _ _ | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 15" Wide, Painted | 1        |
| 1 - 5, 6n   | 117915_ _ _ | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 16" Wide, Painted | 1        |
| 1 - 5, 6o   | 117916_ _ _ | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 17" Wide, Painted | 1        |
| 1 - 5, 6p   | 117917_ _ _ | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 18" Wide, Painted | 1        |
| 1 - 5, 6q   | 117918_ _ _ | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 19" Wide, Painted | 1        |
| 1 - 5, 6r   | 117919_ _ _ | Backrest Actuation Bar Subassembly w/ Release Levers and Fasteners, 20" Wide, Painted | 1        |
| 3           | 000142      | M5 x 45 BHCS, P/THD, BLZ                                                              | 2        |
| 4           | 100657      | M5 Nylock Nut Blk Zn                                                                  | 4        |
| 5           | 102801      | M5 x 16, Low SHCS, Blk Zn                                                             | 2        |
| 6a          | 115554      | Backrest Actuation Bar, 12" Wide, Black                                               | 1        |
| 6b          | 115555      | Backrest Actuation Bar, 13" Wide, Black                                               | 1        |
| 6c          | 115556      | Backrest Actuation Bar, 14" Wide, Black                                               | 1        |
| 6d          | 115557      | Backrest Actuation Bar, 15" Wide, Black                                               | 1        |
| 6e          | 115558      | Backrest Actuation Bar, 16" Wide, Black                                               | 1        |
| 6f          | 115559      | Backrest Actuation Bar, 17" Wide, Black                                               | 1        |
| 6g          | 115560      | Backrest Actuation Bar, 18" Wide, Black                                               | 1        |
| 6h          | 115561      | Backrest Actuation Bar, 19" Wide, Black                                               | 1        |
| 6i          | 115562      | Backrest Actuation Bar, 20" Wide, Black                                               | 1        |

| Item Number | Part Number | Description                               | Quantity |
|-------------|-------------|-------------------------------------------|----------|
| 6j          | 117921_ _ _ | Backrest Actuation Bar, 12" Wide, Painted | 1        |
| 6k          | 117922_ _ _ | Backrest Actuation Bar, 13" Wide, Painted | 1        |
| 6l          | 117923_ _ _ | Backrest Actuation Bar, 14" Wide, Painted | 1        |
| 6m          | 117924_ _ _ | Backrest Actuation Bar, 15" Wide, Painted | 1        |
| 6n          | 117925_ _ _ | Backrest Actuation Bar, 16" Wide, Painted | 1        |
| 6o          | 117926_ _ _ | Backrest Actuation Bar, 17" Wide, Painted | 1        |
| 6p          | 117927_ _ _ | Backrest Actuation Bar, 18" Wide, Painted | 1        |
| 6q          | 117928_ _ _ | Backrest Actuation Bar, 19" Wide, Painted | 1        |
| 6r          | 117929_ _ _ | Backrest Actuation Bar, 20" Wide, Painted | 1        |