


(Discontinued) Rogue with XP Option Fixed Height Backrest with Adjustable Height Rigidizer Bar

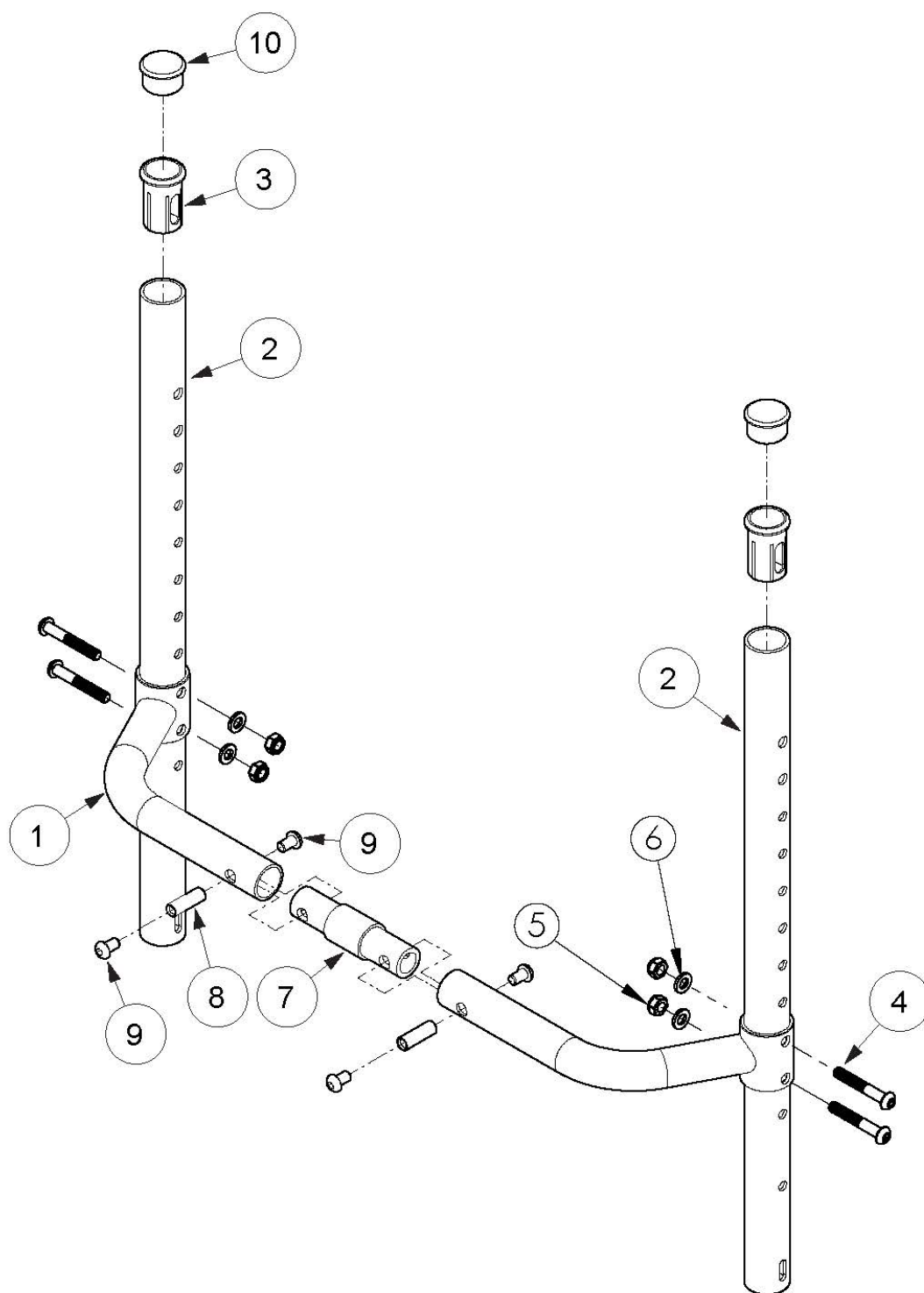

This Rogue with XP Option Backrest is discontinued as of July 25th, 2016 Parts on this page are only for the Discontinued Rogue with XP Option frame/backrest. If you have a Rogue XP, use the Rogue XP Backrest pages for your replacement parts.

Item Number	Part Number	Description	Quantity
1a, 7a, 8, 9	109158	Adjustable Height Rigidizer Bar 12", 0" Connector and Hardware	1
1a, 7b, 8, 9	109159	Adjustable Height Rigidizer Bar 12", 1" Connector and Hardware	1
1a, 7c, 8, 9	109160	Adjustable Height Rigidizer Bar 12", 2" Connector and Hardware	1
1a, 7d, 8, 9	109161	Adjustable Height Rigidizer Bar 12", 3" Connector and Hardware	1
1b, 7a, 8, 9	109162	Adjustable Height Rigidizer Bar 13", 0" Connector and Hardware	1
1b, 7b, 8, 9	109163	Adjustable Height Rigidizer Bar 13", 1" Connector and Hardware	1
1b, 7c, 8, 9	109164	Adjustable Height Rigidizer Bar 13", 2" Connector and Hardware	1
1b, 7d, 8, 9	109165	Adjustable Height Rigidizer Bar 13", 3" Connector and Hardware	1
1c, 7a, 8, 9	109166	Adjustable Height Rigidizer Bar 14", 0" Connector and Hardware	1
1c, 7b, 8, 9	109167	Adjustable Height Rigidizer Bar 14", 1" Connector and Hardware	1
1c, 7c, 8, 9	109168	Adjustable Height Rigidizer Bar 14", 2" Connector and Hardware	1
1c, 7d, 8, 9	109169	Adjustable Height Rigidizer Bar 14", 3" Connector and Hardware	1
1d, 7a, 8, 9	109170	Adjustable Height Rigidizer Bar 15", 0" Connector and Hardware	1
1d, 7b, 8, 9	109171	Adjustable Height Rigidizer Bar 15", 1" Connector and Hardware	1
1d, 7c, 8, 9	109172	Adjustable Height Rigidizer Bar 15", 2" Connector and Hardware	1
1d, 7d, 8, 9	109173	Adjustable Height Rigidizer Bar 15", 3" Connector and Hardware	1
1e, 7a, 8, 9	109174	Adjustable Height Rigidizer Bar 16", 0" Connector and Hardware	1
1e, 7b, 8, 9	109175	Adjustable Height Rigidizer Bar 16", 1" Connector and Hardware	1
1e, 7c, 8, 9	109176	Adjustable Height Rigidizer Bar 16", 2" Connector and Hardware	1
1e, 7d, 8, 9	109177	Adjustable Height Rigidizer Bar 16", 3" Connector and Hardware	1
7b, 8, 9	502318	Backrest Connector Assembly 1" w/ Hardware, Rigid	1
7c, 8, 9	502319	Backrest Connector Assembly 2" w/ Hardware, Rigid	1
7d, 8, 9	502320	Backrest Connector Assembly 3" w/ Hardware, Rigid	1
2	000698	Backrest Tube Fixed Height, 17.25"	2
3	000034	Sleeve 3/4" x 1 1/2"	2

Item Number	Part Number	Description	Quantity
		<i>Used with standard push handles, low profile push handles or fold down push handles</i>	
4	100797	M6 x 40 BHCS BLK ZC	4
5	100658	M6 Nylock Nut Blk Zn	4
6	100746	M6 Flat Washer Blk Zn	4
7a	000589	Backrest Connector 0" LW/RG	1
7b	000590	Backrest Connector 1" LW/RG	1
7c	000591	Backrest Connector 2" LW/RG	1
7d	000592	Backrest Connector 3" LW/RG	1
8	000669	M6 x 20.5 Threaded Barrel	2
9	101184	M6 x 10 BTN SHCS BLKZC w/Patch	4
10	100804	Tube End Plug 1" <i>Used with omit push handles</i>	2