


# Discontinued Flip Height Adjustable T-Arm

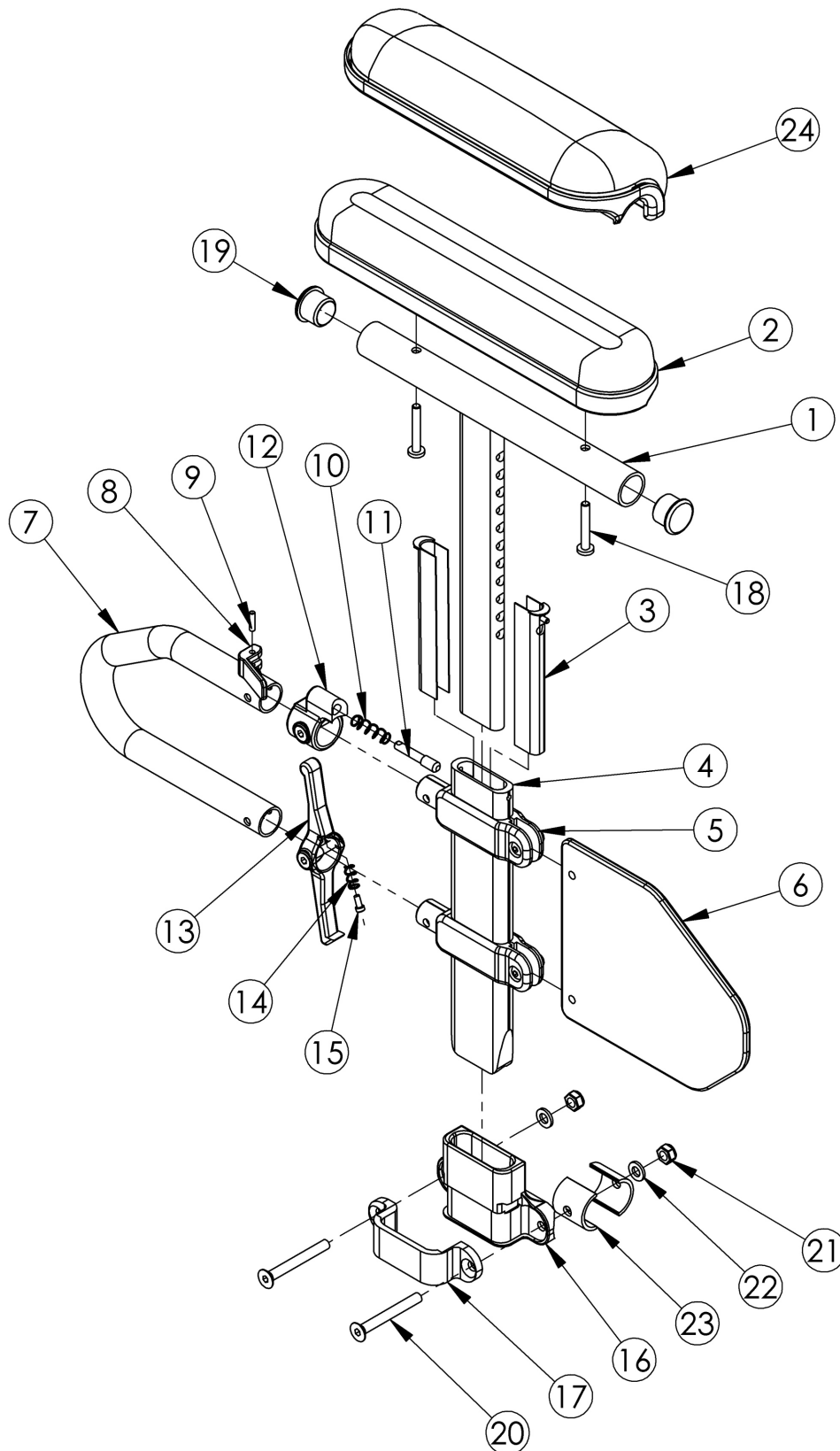

Item Number	Part Number	Description	Quantity
1a, 2b, 3-5, 6, 7-15, 18a, 20	501682	Height Adj T-Arm Assm Short Desk w/ Std Pad Left <i>Replaced with 505646</i>	1
1a, 2b, 3-5, 6, 7-15, 18a, 20	501683	Height Adj T-Arm Assm Short Desk w/ Std Pad Right <i>Replaced with 505646</i>	1
1b, 2a, 3-5, 6, 7-15, 18a, 20	501684	Height Adj T-Arm Assm Short Full w/ Std Pad Left <i>Replaced with 505644</i>	1
1b, 2a, 3-5, 6, 7-15, 18a, 20	501685	Height Adj T-Arm Assm Short Full w/ Std Pad Right <i>Replaced with 505644</i>	1
1a, 2b, 3-5, 6, 7-17, 18a, 20-23	503171	Height Adj T-Arm Assm Short Desk w/ Receiver Left <i>Replaced with 506073</i>	1
1a, 2b, 3-5, 6, 7-17, 18a, 20-23	503172	Height Adj T-Arm Assm Short Desk w/ Receiver Right <i>Replaced with 506073</i>	1
1b, 2a, 3-5, 6, 7-17, 18a, 20-23	503173	Height Adj T-Arm Assm Short Full w/ Receiver Left <i>Replaced with 506074</i>	1
1b, 2a, 3-5, 6, 7-17, 18a, 20-23	503174	Height Adj T-Arm Assm Short Full w/ Receiver Right <i>Replaced with 506074</i>	1
1a, 2b, 3-5, 6, 7-17, 18a, 20-23	502889	Height Adjustable T-Arm Assm Short Desk w/ Receiver PAIR FLIP <i>Replaced with 506075</i>	1
1b, 2a, 3-5, 6, 7-17, 18a, 20-23	502890	Height Adjustable T-Arm Assm Short Full w/ Receiver PAIR FLIP <i>Replaced with 506076</i>	1
1a, 2b, 3-5, 6, 7-15, 18b, 20	501690	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad Left <i>Replaced with 505650</i>	1
1a, 2b, 3-5, 6, 7-15, 18b, 20	501691	Height Adj T-Arm Assm Short Desk w/ Waterfall Pad Right <i>Replaced with 505650</i>	1
1b, 2a, 3-5, 6, 7-15, 18b, 20	501692	Height Adj T-Arm Assm Short Full w/ Waterfall Pad Left <i>Replaced with 505648</i>	1
1b, 2a, 3-5, 6, 7-15, 18b, 20	501693	Height Adj T-Arm Assm Short Full w/ Waterfall Pad Right <i>Replaced with 505648</i>	1
1a, 2b, 3-5, 6, 7-17, 18b, 20-23	503187	Height Adj T-Arm Assm Short Waterfall Desk w/ Receiver Left <i>Replaced with 506077</i>	1
1a, 2b, 3-5, 6, 7-17, 18b, 20-23	503188	Height Adj T-Arm Assm Short Waterfall Desk w/ Receiver Right <i>Replaced with 506077</i>	1

Item Number	Part Number	Description	Quantity
1b, 2a, 3-5, 6, 7-17, 18b, 20-23	503189	Height Adj T-Arm Assm Short Waterfall Full w/ Receiver Left <i>Replaced with 506078</i>	1
1b, 2a, 3-5, 6, 7-17, 18b, 20-23	503190	Height Adj T-Arm Assm Short Waterfall Full w/ Receiver Right <i>Replaced with 506078</i>	1
1a, 2b, 3-5, 6, 7-17, 18b, 20-23	503199	Height Adj T-Arm Assm Short Waterfall Desk w/ Receiver PAIR <i>Replaced with 506079</i>	1
1b, 2a, 3-5, 6, 7-17, 18b, 20-23	503200	Height Adj T-Arm Assm Short Waterfall Full w/ Receiver PAIR <i>Replaced with 506080</i>	1
16, 17, 20-23	503166	Armrest T-Arm Receiver Assembly Flip	1
1a, 3-5, 6, 7-15, 20	100242	Height Adj T Armrest Short Desk Left <i>Replaced with 003463</i>	1
1a, 3-5, 6, 7-15, 20	100238	Height Adj T Armrest Short Desk Right <i>Replaced with 003463</i>	1
1b, 3-5, 6, 7-15, 20	100243	Height Adj T Armrest Short Full Left <i>Replaced with 003464</i>	1
1b, 3-5, 6, 7-15, 20	100239	Height Adj T Armrest Short Full Right <i>Replaced with 003464</i>	1
1a, 19	000091	Height Adj T Armrest Upper Desk	1
1b, 19	000101	Height Adj T Armrest Upper Full	1
2a	100500	Arm Pad Full	1
2b	100501	Arm Pad Desk	1
3	000090	Height Adjustable T Armrest Shim	2
4	000089	Height Adjustable T Armrest Post	1
6	000097	Height Adj T Armrest Side Guard Short	1
7	000095	Height Adjustable T Armrest Transfer Tube	1
9	200257	Dowel Pin 1/8 x 1/2	1
10	000134	Height Adjustable Armrest Lever Spring	1
11	000099	Height Adjustable Armrest Lever Pin	1
12	000096	Height Adjustable T Armrest Pin Lock Housing	1
14	000135	Height Adjustable T Armrest Release Lever Spring	1

Item Number	Part Number	Description	Quantity
15	000152	Height Adjustable T Armrest Release Spring Button	1
16	002036	Receiver, T-Arm, High	1
17	000088	Height Adjustable T Armrest Bracket	1
18a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2
18b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2
19	100803	Tube End Plug 7/8"	2
20	101592	M6 x 45 CLS10.9 DIN7991 FHCS BLKZC	2
21	100658	M6 Nylock Nut Blk Zn	2
22	100746	M6 Flat Washer Blk Zn	2
24a	102452	Waterfall Arm Pad Full	1
24b	102521	Waterfall Arm Pad Desk	1