


Ethos Height Adjustable T-Arm

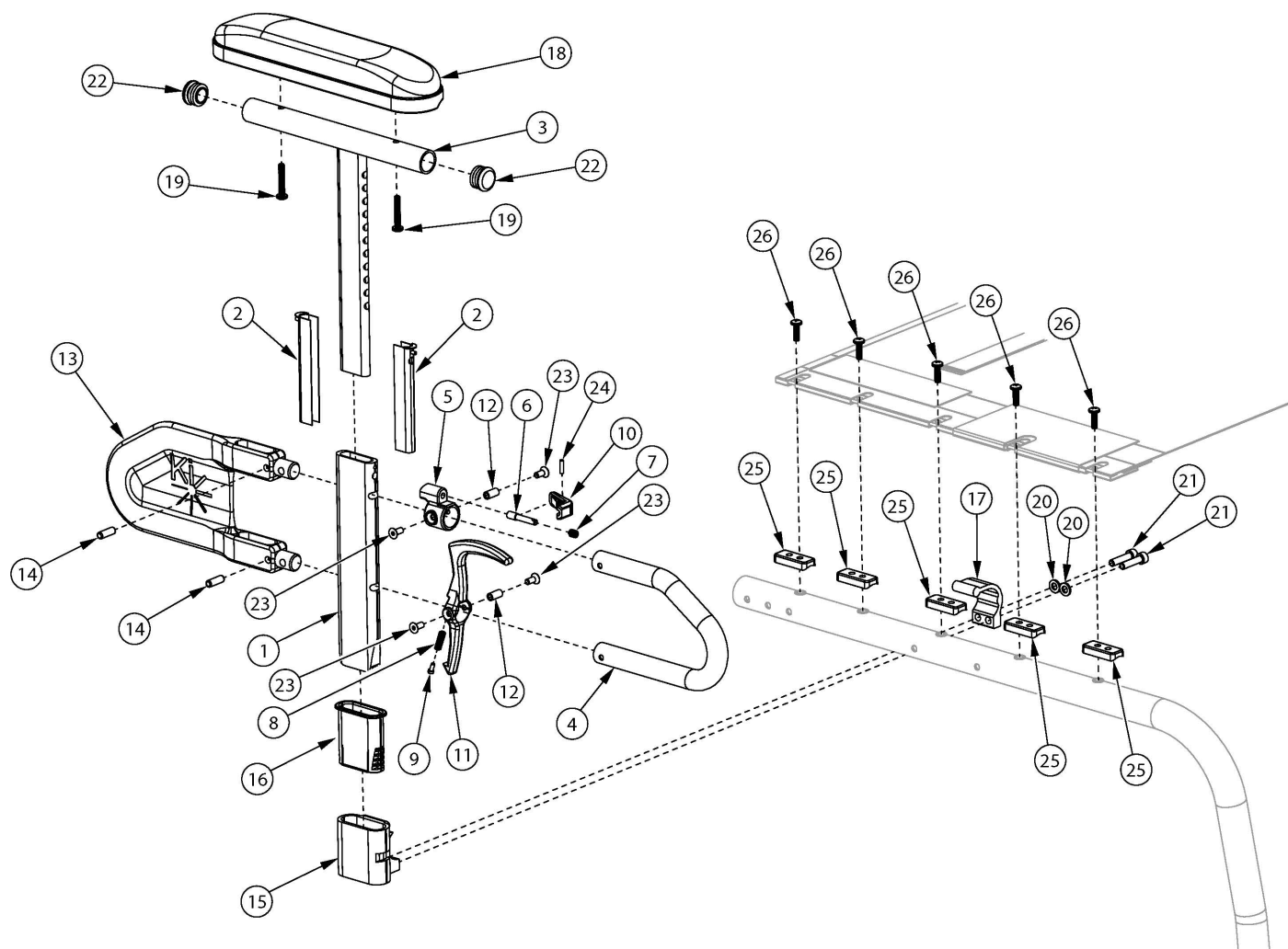

If retrofitting to Height Adjustable T-Arms and wheel locks are present, extended wheel lock clamp assemblies are required. If retrofitting, Receivers are needed.

Item Number	Part Number	Description	Quantity
1, 2, 4-14, 23-24	004160	Height Adjustable T-Arm Lower Assembly Short <i>Does not include t-arm upper, arm pad or arm pad screws</i>	1
1, 2, 3a, 4-14, 22-24	003463	Height Adjustable T-Arm Short Desk <i>Does not include arm pad or arm pad screws</i>	1
1, 2, 3b, 4-14, 22-24	003464	Height Adjustable T-Arm Short Full <i>Does not include arm pad or arm pad screws</i>	1
1, 2, 3a, 4-14, 18b, 19a, 22-24	505646	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Desk	1
1, 2, 3a, 4-14, 18b, 19a, 22-24	505647	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Desk, Pair	1

Item Number	Part Number	Description	Quantity
1, 2, 3b, 4-14, 18a, 19a, 22-24	505644	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Full	1
1, 2, 3b, 4-14, 18a, 19a, 22-24	505645	Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Full, Pair	1
1, 2, 3a, 4-14, 18d, 19b, 22-24	505650	Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Desk	1
1, 2, 3a, 4-14, 18d, 19b, 22-24	505651	Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Desk, Pair	1
1, 2, 3b, 4-14, 18c, 19b, 22-24	505648	Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Full	1
1, 2, 3b, 4-14, 18c, 19b, 22-24	505649	Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Full, Pair	1
1, 2, 3a, 4-17, 18b, 19a, 20-26	108235	Height Adj T-Arm Assm w/ Short Side Shield, Standard Pad Desk with Receiver and Seat Rail Saddles	1
1, 2, 3a, 4-17, 18b, 19a, 20-26	108236	Height Adj T-Arm Assm w/ Short Side Shield, Standard Pad Desk with Receiver and Seat Rail Saddles (PAIR)	1
1, 2, 3b, 4-17, 18a, 19a, 20-26	108237	Height Adj T-Arm Assm w/ Short Side Shield, Standard Pad Full with Receiver and Seat Rail Saddles	1
1, 2, 3b, 4-17, 18a, 19a, 20-26	108238	Height Adj T-Arm Assm w/ Short Side Shield, Standard Pad Full with Receiver and Seat Rail Saddles (PAIR)	1
1, 2, 3a, 4-17, 18d, 19b, 20-26	108239	Height Adj T-Arm Assm w/ Short Side Shield, Waterfall Pad Desk with Receiver and Seat Rail Saddles	1
1, 2, 3a, 4-17, 18d, 19b, 20-26	108240	Height Adj T-Arm Assm w/ Short Side Shield, Waterfall Pad Desk with Receiver and Seat Rail Saddles (PAIR)	1
1, 2, 3b, 4-17, 18c, 19b, 20-26	108241	Height Adj T-Arm Assm w/ Short Side Shield, Waterfall Pad Full with Receiver and Seat Rail Saddles	1
1, 2, 3b, 4-17, 18c, 19b, 20-26	108242	Height Adj T-Arm Assm w/ Short Side Shield, Waterfall Pad Full with Receiver and Seat Rail Saddles (PAIR)	1
5-7, 10, 23, 24	505490	Height Adjustable T-Arm Upper Latch Assembly	1
8, 9, 11, 23	505507	Height Adjustable T-Arm Release Lever Assembly	1
15-17, 20, 21	108234	Height Adjustable T-Arm Receiver Assembly, Ethos	1
15-17, 20, 21	109750	Height Adjustable T-Arm Receiver Assembly, PAIR, Ethos	1
12-14	506082	Height Adjustable T-Arm Short Shield Assembly	1
1	000089	Height Adjustable T Armrest Post	1
2	000090	Height Adjustable T Armrest Shim	2

Item Number	Part Number	Description	Quantity
3a, 22	112906	Height Adjustable T Armrest Upper Desk w/ Tube End Plug	1
3b, 22	112907	Height Adjustable T Armrest Upper Full w/ Tube End Plug	1
4	000095	Height Adjustable T Armrest Transfer Tube	1
8	000135	Height Adjustable T Armrest Release Lever Spring	1
9	000152	Height Adjustable T Armrest Release Spring Button	1
12	003457	M5 x 14mm Barrel Nut	2
14	003641	.25" x .75" Dowel Pin	2
15	005260	T-Arm Receiver, Ethos	1
16	005261	T-Arm Receiver Liner, Ethos	1
17	005262	T-Arm Clamp, Ethos	1
18a	100500	Arm Pad Full	1
18b	100501	Arm Pad Desk	1
18c	102452	Waterfall Arm Pad Full	1
18d	102521	Waterfall Arm Pad Desk	1
19a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2
19b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2
20	100746	M6 Flat Washer Blk Zn	2
21	100780	M6 x 25 SHCS ZC	2
22	100803	Tube End Plug 7/8"	2
23	003458	M5 X 12mm FHCS BLK ZN W/ Patch	4
25	005259	Seat Rail Saddle, Offset 1.125" <i>Use quantity 4 per side for 14" - 15" seat depths, quantity 5 per side for 16" - 18" seat depths and quantity 6 per side for 19" - 20" seat depths. If Seat Option is Omit Seat Sling, Seat Rail Saddles are not used.</i>	6
26	101117	M5 x 22 BTN SHCS BLK ZC <i>Use quantity 4 per side for 14" - 15" seat depths, quantity 5 per side for 16" - 18" seat depths and quantity 6 per side for 19" - 20" seat depths</i>	6