


Arc Front Side Frame

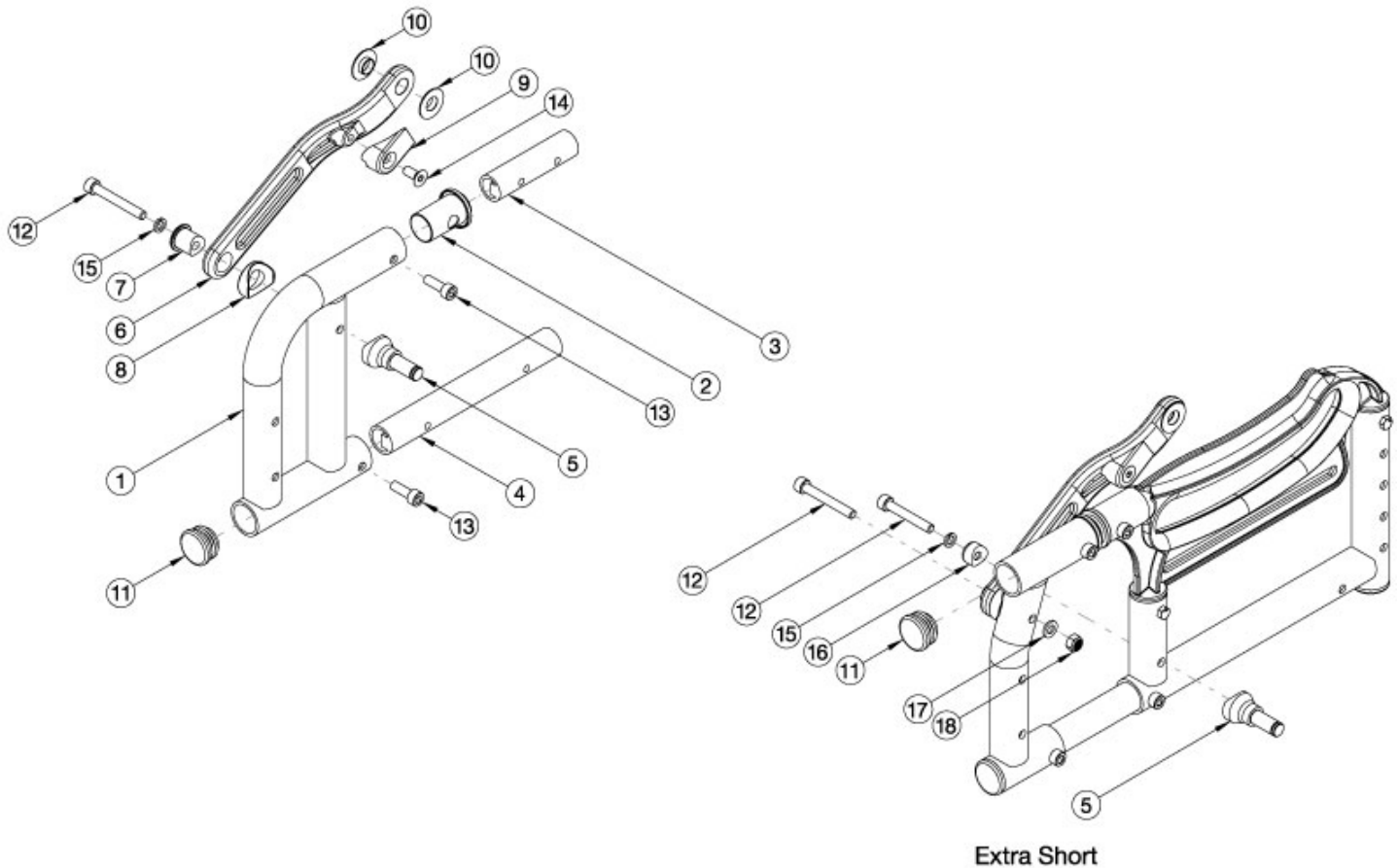

Select color of parts based on Base Frame color of the chair. When changing frame depth ranges, new seat frame assemblies are needed. If quoting a Left Front Frame for Australia or New Zealand, 122921 is required when transit is not present.

Item Number	Part Number	Description	Quantity
1a, 2, 11	113109_ _ _	Front Side Frame, Extra Short, Right, Arc <i>Accommodates 12-15" Depths</i>	1
1b, 2, 11	113110_ _ _	Front Side Frame, Extra Short, Left, Arc <i>Accommodates 12-15" Depths</i>	1
1c, 2, 11	113111_ _ _	Front Side Frame, Short, Right, Arc <i>Accommodates 14-17" Depths</i>	1
1d, 2, 11	113112_ _ _	Front Side Frame, Short, Left, Arc <i>Accommodates 14-17" Depths</i>	1
1e, 2, 11	113113_ _ _	Front Side Frame, Medium, Right, Arc <i>Accommodates 16-19" Depths</i>	1
1f, 2, 11	113114_ _ _	Front Side Frame, Medium, Left, Arc <i>Accommodates 16-19" Depths</i>	1

Item Number	Part Number	Description	Quantity
1g, 2, 11	113115_ _ _	Front Side Frame, Long, Right, Arc <i>Accommodates 18-20" Depths</i>	1
1h, 2, 11	113116_ _ _	Front Side Frame, Long, Left, Arc <i>Accommodates 18-20" Depths</i>	1
2	110149	Bushing, Pivot, X-Hinge	1
3	110162	Side Frame Pivot Tube, Upper	1
4	110163	Side Frame Pivot Tube, Lower	1
5	110210	Mount, Mechlock, Lower	1
6	110212	Link Arm	1
7	110214	Link Arm Lower Bushing	1
8	110215	Saddle, Link Arm	1
9	110270	Bumper, Tilt Stop	1
10	110253	Bushing, Flg, .397IDx.87ODx.175Lg	2
11	100804	Tube End Plug 1"	1
12	101472	M6 x 45 SHCS BLK ZC	1
13	101179	M6 X 10 Shcs Blk Zc W/Patch	2
14	101594	M6 x 14 CLS10.9 DIN7991 FSHCS BLKZC w/Patch	1
15	102810	M6 Lockwasher, Hi-Collar, ZC	1
	122921	Not AS/NZS 3696.19 Compliant On-Chair Label with Instruction	1