

Etac Clean Manual

78644D 18-04-06


etac®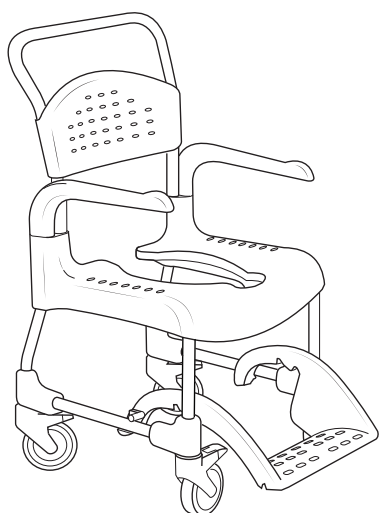

Clean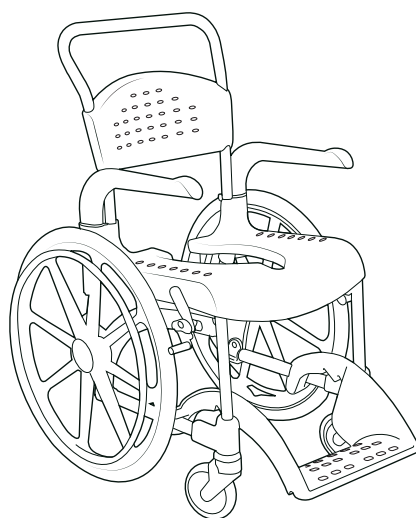

Clean 24"

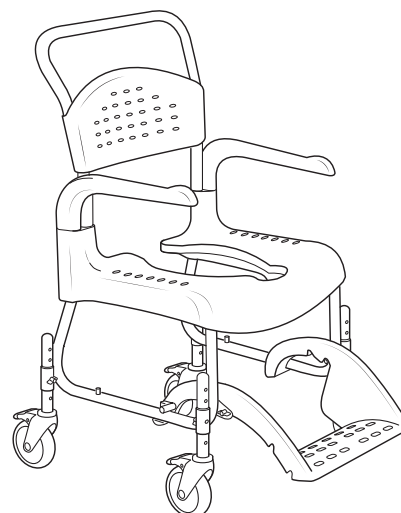Clean HRB

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Clean
Clean 24"
Clean HRB

General

Thank you for choosing a shower chair from Etac.

In order to avoid injury during handling and use, please read through this manual first.

In the manual, the user is the person sitting in the chair. The assistant is the person operating the chair.

Clean is a mobile shower and toilet chair intended to make hygiene care easier for people with reduced mobility and their carers. It is designed for use in showers, with hand basins and over toilets, as well as for transport to and from the sanitary room. Every component of the chair is designed to make things easier for users and carers.

Clean can be used by people weighing a maximum of 130 kg.

Clean has been tested by a testing institute and complies with the requirements stated in EN ISO 10993-1 and SS EN 12182.

This product should be recycled as per national regulations.

This symbol appears in the manual along with text. It indicates instances where the safety of the user or the assistant may be at risk.

The images which are referred to on each page are found at the start of the manual (pages 3 to 7). See Figures A to J for the corresponding text.

Item list: ● = Applies to, ○ = Accessories

See the item list on the right which displays the information applicable to each product.

This manual covers the following products:

Clean	●		
Clean 24"		●	
Clean HRB (Height adjustable)			●

Product description & dimensions table

See Figure A

A1. Backrest **A2.** Armrest **A3.** Seat **A4.** Foot support **A5.** Castor wheel **A6.** Product label (warnings, CE-marking) **A7.** Item no label (serial no, production date)

A8. Rear wheel **A9.** User brake

A10. Height adjustment

Warranty / maintenance

See Figure B

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com

Product lifetime:

Service life 7 years. For complete information regarding the service life of the product, see www.etac.com.

Maintenance:

To keep the chair in good and safe working order, you should regularly:

- Clean the chair with ordinary non-abrasive detergent. Rinse and dry.
- Keep the castors free of hair and soap deposits, etc.
- If your shower and toilet chair is not in perfect working order, contact your dealer.

Symbols

Safety

See Figure B

- Defective chairs must not be used.

Symbols

B3. Do not stand on the footrest, risk of tipping!

B4. Risk of pinching at armrest.

B5. Maximum user weight: 130 kg.

B6. Seating position affects the stability.

Transfers

See Figure C

Over a threshold:

C1. Chair only, push down on the wheel attachment.

C2. With occupant, pull the chair backwards.

In stationary position:

C3. Always lock as many castors as possible when the chair is to be used in a stationary position.

C4. Out-turned castors improve the chair's stability.

Transferring:

Assess the risks and take notes. You as a carer are responsible for the safety of the user. Aids should be used if the transfer is deemed as risky. Visit www.etac.com for manual transfer aids.

C3-C4. It is important to ensure that the castors are locked and pointing outward before carrying out transfers to and from Clean.

- Remove the armrest on the transfer side of the chair.

C5-C7. Carry out the transfer.

English

English				Clean	Clean 24"	Clean HRB
Armrest				See Figure D		
D1. The armrests are swing away and detachable.				●	●	●
D2. The armrests are lockable.						
Height adjustment				See Figure E		
E1-3. The height can be adjusted with ease by placing the screw in the required position.						●
 Check that all four castors are similarly adjusted, otherwise stability will be reduced.				 Do not use the chair if the wing nuts are loose.		
Footrest				See Figure F		
F1. The carer can easily push in the footrest with their foot.				●	●	●
F2. The footrest is detachable.				●	●	●
F3. The footrest has 4 different angle and height settings.				●	●	
F4. The footrest has 8 different angle and height settings.						●
F5. More adjustment options are available by moving the pin.						
 Remove the pins once the footrest has been removed.				 When the chair is adjusted to a height of 47.5 cm, the lowest footrest position must not be used.		
Backrest upholstery				See Figure G		
				○	○	○
User brake				See Figure H		
				○	●	
Accessories				See Figure I		
The accessories section shows all the options that are available for your shower chair. Variations from standard can occur in different markets. Limitations in combinations of accessories may occur. For more information visit www.etac.com .						
See Figure I				Clean	Clean 24"	Clean HRB
I1. Pan				○	○	○
I2. Pan and lid with handle				○	○	○
I3. Lid with handle				○	○	○
I4. Disposable pan				○	○	○
I5. Pan holder				○	○	○
I6. Bucket holder				○	○	○
I7. Disposable pan holder				○	○	○
I8. Attachment for low mounting of pan/bucket				○	○	○
I9. Soft seat				○	○	○
I10. Comfort seat				○	○	○
I11. Comfort cushion				○	○	○
I12. ROHO protect cushion, pressure-relieving				○	○	○
I13. Comfort padding				○	○	○
I14. Armrest cushion, pair				○	○	○
See Figure I				Clean	Clean 24"	Clean HRB
I15. Soft back				○	○	○
I16. Trunk support				○	○	○
I17. Heightening spacers				○		
I18. Two 75 mm lockable castors				○		
I19. Rear wheel adaptor				○	●	
I20. Soap dish				○		○
I21. Crossbar				○	○	○
I22. Legrest, angle adjustable						○
I23. Foot support, foldable and detachable						○
I24. Backrest upholstery				○	○	○
I25. Safety belt				○	○	○
I26. Positioning belt				○	○	○
I27. Chest harness belt				○	○	○

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Etac A/S
R82 UK Limited.
Snug Seat, Inc.
Etac Supply Center AB