


Catalyst E Hangers - 60 Degree Extension Tube

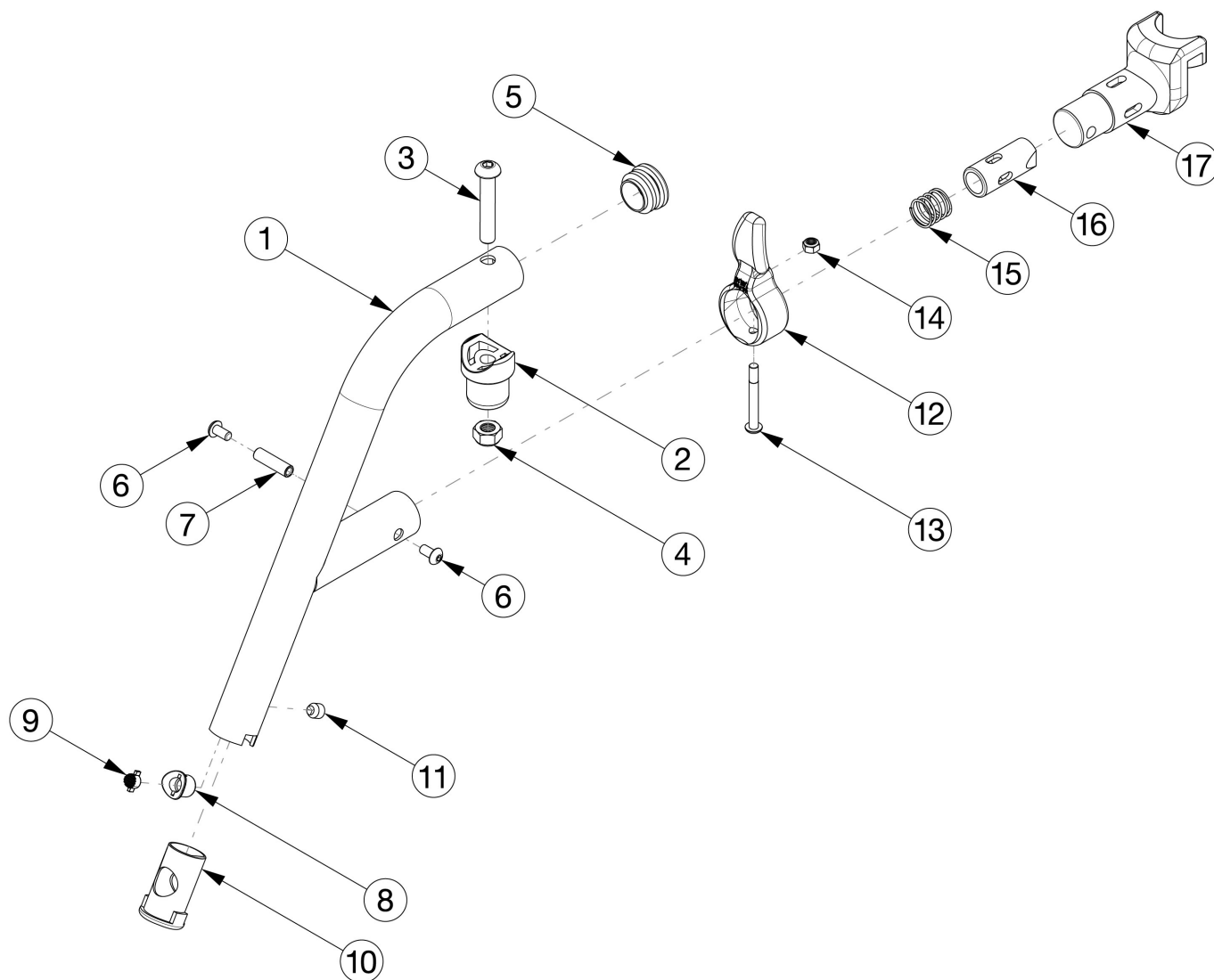

If changing degree of hanger, new extension tubes may be needed

Item Number	Part Number	Description	Quantity
1-17	121299_ _ _	60° Cleat Hanger Assembly, Classic Latch Right, Catalyst E	1
1-17	121332_ _ _	60° Cleat Hanger Assembly, Classic Latch Left, Catalyst E	1
1-17	115921_ _ _	60° Cleat Hanger Assembly, 4-Way-In-Line, Catalyst E	1
1-17	115916_ _ _	60° Cleat Hanger Assembly, 4-Way - Out Left, Catalyst E _ _ _	1
1-17	115917_ _ _	60° Cleat Hanger Assembly, 4-Way - Out Right, Catalyst E _ _ _	1
12 - 17	003225	Assembled 4-Way In Line Hanger Release	1

Item Number	Part Number	Description	Quantity
12 - 17	003225INST	Assembled 4-Way In Line Hanger Release with Instruction	1
12 - 17	003643	Assembled 4-Way Out Hanger Release, Left	1
12 - 17	003643INST	Assembled 4-Way Out Hanger Release, Left with Instruction	1
12 - 17	003642	Assembled 4-Way Out Hanger Release, Right	1
12 - 17	003642INST	Assembled 4-Way Out Hanger Release, Right with Instruction	1
8 - 11	115845	Cleat Hanger Adjustment Hardware	1
2	000032	Footrest Hanger Pivot Saddle	1
3	100673	M8 x 45 BTN SHCS BLKZC	1
4	100659	M8 Nylock Nut Blk Zn	1
5	100804	Tube End Plug 1"	1
6	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2
7	000073	M5 x 24 Threaded Barrel	1
8	111949	Hanger Cleat Housing	1
9	111950	Hanger Cleat	1
10	111948	Cleat Hanger Sleeve	1
11	114457	M8 x 8 x 1.25 Cup Point Set Screw w/ Patch	1
12	003228	Hanger Latch, 4-Way, Lever	1
13	001374	M5 x 40 BHCS P/THD BLKZC	1
14	100657	M5 Nylock Nut Blk Zn	1
15	000119	Hanger Spring	1
16	003227	Hanger Latch Pin	1
17	003226	Hanger Latch Bracket	1

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
BRD	Red
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer