

# Oxford<sup>®</sup>



## Journey



 **Joerns**  
Healthcare

Redefining patient handling

Compact, portable and  
exceptionally versatile

# Journey

The Journey standaid from Oxford incorporates the latest design and manufacturing technologies to produce the most compact and portable, folding standaid in the market today.

## Key Features:

- Folding design for easy storage and onward transportation
- Unique adjustable cow-horn function
- Compact and easy to manoeuvre
- 155kg/24st safe working load
- Ergonomic design



The Journey has the option of both a standing and transport sling, providing the carer with the option of performing various care duties, including:

- Toileting
- Standing
- Point to point transfers
- Walking/Rehab



## Adjustable cow-horn

The Journey's unique adjustable cow-horn ensures maximum flexibility and improved patient comfort. No matter what the patient task or environment, the cow-horn can be adjusted to suit.



With three different cow-horn height options, the Journey can support a wider range of patient heights and sizes. This additional method of adjustment ensures total flexibility for the care-giver.



The Journey's three optional cow-horn positions.

## Superior ergonomic design

# 5

The Journey was designed with the 5 key principles of moving and handling in mind.

- 1 Get close to the load
- 2 Use a wide stable base
- 3 Ensure a comfortable firm grasp
- 4 Keep spine close to neutral
- 5 Make sure movement is smooth

This direct correlation ensures Oxford's lift designs are simple, safe and above all comfortable to use.

This view is shared by our consultant teams, who have assisted us in conducting detailed usability surveys to ensure that the Journey meets the true needs of the patient, the carer and the environment in which it operates.

## Easy to transport and store

Unlike any other standard on the market, the Journey can fold making it easy to transport and store. This functionality gives active users the option to travel outside the confines of their own home.



To fold the Journey, adjust the cow-horn to the fold position then raise the knee pad to its upper most point.



Next, unlock the mast-locking device and lower the mast assembly towards the legs.



The mast then locks into position. The Journey is now ready to be stored or transported.



With its quick-release technology, the Journey folds or unfolds in a matter of seconds without the need for tools or special attachments.



In environments where space and storage are at a premium, the Journey can fold quickly and be stored upright to conserve storage space.



Once folded, the Journey fits comfortably into the boot of most family sized saloons.



## Adjustable knee pad

The Journey's knee pad is height adjustable to ensure the patient feels comfortable and fully supported throughout the transfer.



The knee pad is contoured for improved patient comfort and is easy to clean to help prevent cross infection.



A retractable safety belt provides additional support for patients needing further reassurance.



The knee pad can be adjusted in height within a 10 inch (260mm) range.

## Removable foot tray

By removing the foot tray, the Journey transforms into a rehabilitation aid that allows a patient to fully stand. Once standing, the patient can be encouraged to walk independently.



Foot tray attached.



The foot tray is easily removed for cleaning or rehabilitation activity.



Foot tray removed for gait training.

Specification	Imperial	Metric
Safe Working Load	341 lbs	155 kgs
Maximum Overall Length	39.4 inches	1000 mm
Minimum Overall Length	36.8 inches	935 mm
Maximum Overall Height	63.0 inches	1600 mm
Minimum Overall Height	44.5 inches	1130 mm
Height (Folded)	20.5 inches	525 mm
Depth (Folded)	41.0 inches	1040 mm
Widest Point (Between support handles)	26.8 inches	680 mm
Maximum Height to Attachment Point	60.2 inches	1530 mm
Minimum Height to Attachment Point	28.4 inches	720 mm
Turning Radius	45.1 inches	1145 mm
Legs Open - External Width	38.2 inches	970 mm
Legs Open - Internal Width	34.5 inches	875 mm
Legs Closed - External Width	27.0 inches	685 mm
Legs Closed - Internal Width	22.2 inches	590 mm
Overall Height of Legs	4.3 inches	110 mm
Ground Clearance	1.2 inches	30 mm
Front Twin Castors	3.0 inches	75 mm
Rear Braked Castors	4.0 inches	100 mm

Weights	Imperial	Metric
Mast, Base & Boom Assembly	82.4 lbs	37.4 kgs
Power Pack	6.2 lbs	2.8 kgs
Total	88.6 lbs	40.2 kgs
Base Assembly	30.2 lbs	13.7 kgs
Mast & Boom Assembly (not including battery)	42.3 lbs	19.2 kgs

Specifications may vary due to manufacturing tolerances.  
Patent Pending.

#### Electrical Specifications

Battery 1 X 24 volt rechargeable sealed lead acid type  
Battery capacity 3.2 Ampere hours  
Charger rated input 100 - 240 V AC/ 50/60Hz  
Charger rated output 29.5 V DC Max 19 W

#### Warranty

The Journey comes with a 2 year warranty as standard.  
Please refer to your user manual for further information.

#### Electrical Shock Protection

Charger - class II  
Lift - internal power source

Degree of shock protection

Charger - Type B


Lift - Type B

Intended operating environment : >+5° <+40°

Outside this environment functionality and safety may be compromised



Oxford®

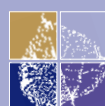

Joerns Healthcare Limited  
High Street, Wollaston, Stourbridge, West Midlands DY8 4PS England  
(T) +44 (0)1384 446622 (F) +44 (0)1384 446601  
www.joerns.co.uk • email: info@joerns.co.uk