
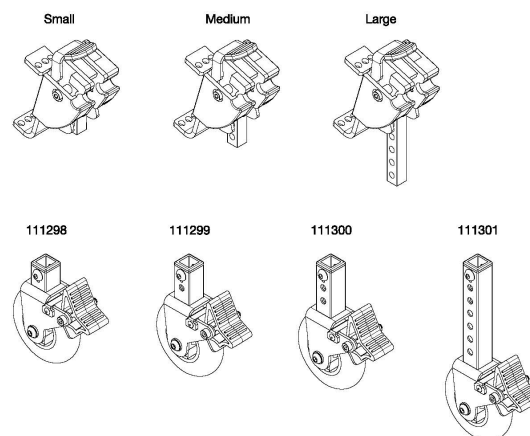


# Liberty Transit Wheels - Folding Frame

| Rear Seat Height | Item 1 Left Side | Item 1 Right Side | Item 2 |
|------------------|------------------|-------------------|--------|
| 14"              | 113951           | 113950            | 111298 |
| 14.5"            | 113951           | 113950            | 111299 |
| 15" - 15.5"      | 113953           | 113952            | 111300 |
| 16" - 16.5"      | 113955           | 113954            | 111300 |
| 17" - 18"        | 113955           | 113954            | 111301 |

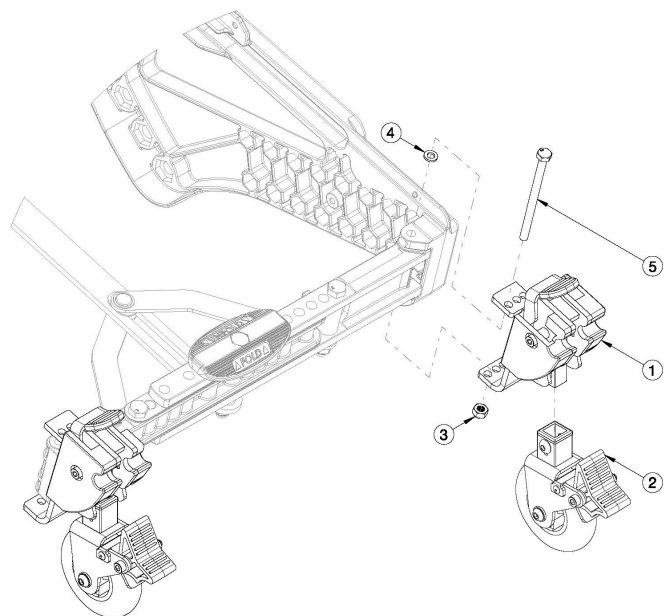

Not available with Long Seat Depth Range

| Item Number       | Part Number | Description  | Quantity |
|-------------------|-------------|--|----------|
| 1a, 1b, 2a, 3 - 5 | 115588      | Liberty Transit Wheels Kit, Small/Extra Small (PAIR), Short/Medium Depth Range, Folding<br><i>Used with 14" rear seat height</i>         | 1        |
| 1a, 1b, 2b, 3 - 5 | 115589      | Liberty Transit Wheels Kit, Small/Small (PAIR), Short/Medium Depth Range, Folding<br><i>Used with 14.5" rear seat height</i>             | 1        |
| 1c, 1d, 2c, 3 - 5 | 115590      | Liberty Transit Wheels Kit, Medium/Medium (PAIR), Short/Medium Depth Range, Folding<br><i>Used with 15" - 15.5" rear seat heights</i>    | 1        |
| 1e, 1f, 2c, 3 - 5 | 115591      | Liberty Transit Wheels Kit, Large/Medium (PAIR), Short/Medium Depth Range, Folding<br><i>Used with 16" - 16.5" rear seat heights</i>     | 1        |
| 1e, 1f, 2d, 3 - 5 | 115592      | Liberty Transit Wheels Kit, Large/Large (PAIR), Short/Medium Depth Range, Folding<br><i>Used with 17" - 18" rear seat heights</i>        | 1        |
| 1a, 2a, 3 - 5     | 115593      | Liberty Transit Wheel Kit, Small/Extra Small, Left Side, Short/Medium Depth Range, Folding<br><i>Used with 14" rear seat height</i>      | 1        |
| 1a, 2b, 3 - 5     | 115594      | Liberty Transit Wheel Kit, Small/Small, Left Side, Short/Medium Depth Range, Folding<br><i>Used with 14.5" rear seat height</i>          | 1        |
| 1c, 2c, 3 - 5     | 115595      | Liberty Transit Wheel Kit, Medium/Medium, Left Side, Short/Medium Depth Range, Folding<br><i>Used with 15" - 15.5" rear seat heights</i> | 1        |

| Item Number   | Part Number | Description   | Quantity |
|---------------|-------------|---|----------|
| 1e, 2c, 3 - 5 | 115596      | Liberty Transit Wheel Kit, Large/Medium, Left Side, Short/Medium Depth Range, Folding<br><i>Used with 16" - 16.5" rear seat heights</i>   | 1        |
| 1e, 2d, 3 - 5 | 115597      | Liberty Transit Wheel Kit, Large/Large, Left Side, Short/Medium Depth Range, Folding<br><i>Used with 17" - 18" rear seat heights</i>      | 1        |
| 1b, 2a, 3 - 5 | 115598      | Liberty Transit Wheel Kit, Small/Extra Small, Right Side, Short/Medium Depth Range, Folding<br><i>Used with 14" rear seat height</i>      | 1        |
| 1b, 2b, 3 - 5 | 115599      | Liberty Transit Wheel Kit, Small/Small, Right Side, Short/Medium Depth Range, Folding<br><i>Used with 14.5" rear seat height</i>          | 1        |
| 1d, 2c, 3 - 5 | 115600      | Liberty Transit Wheel Kit, Medium/Medium, Right Side, Short/Medium Depth Range, Folding<br><i>Used with 15" - 15.5" rear seat heights</i> | 1        |
| 1f, 2c, 3 - 5 | 115601      | Liberty Transit Wheel Kit, Large/Medium, Right Side, Short/Medium Depth Range, Folding<br><i>Used with 16" - 16.5" rear seat heights</i>  | 1        |
| 1f, 2d, 3 - 5 | 115602      | Liberty Transit Wheel Kit, Large/Large, Right Side, Short/Medium Depth Range, Folding<br><i>Used with 17" - 18" rear seat heights</i>     | 1        |
| 1a            | 113951      | Liberty Transit Wheel Upper Assembly, Small, Left<br><i>Used with 14" - 14.5" rear seat heights</i>                                       | 1        |
| 1b            | 113950      | Liberty Transit Wheel Upper Assembly, Small, Right<br><i>Used with 14" - 14.5" rear seat heights</i>                                      | 1        |
| 1c            | 113953      | Liberty Transit Wheel Upper Assembly, Medium, Left<br><i>Used with 15" - 15.5" rear seat heights</i>                                      | 1        |
| 1d            | 113952      | Liberty Transit Wheel Upper Assembly, Medium, Right<br><i>Used with 15" - 15.5" rear seat heights</i>                                     | 1        |
| 1e            | 113955      | Liberty Transit Wheel Upper Assembly, Large, Left<br><i>Used with 16" - 18" rear sea heights</i>  | 1        |
| 1f            | 113954      | Liberty Transit Wheel Upper Assembly, Large, Right<br><i>Used with 16" - 18" rear sea heights</i>   | 1        |
| 2a            | 111298      | Liberty Transit Wheel Bottom Assembly, Extra Small, Short/Medium Depth Range<br><i>Used with 14" rear seat height</i>                     | 1        |
| 2b            | 111299      | Liberty Transit Wheel Bottom Assembly, Small, Short/Medium Depth Range<br><i>Used with 14.5" rear seat height</i>                         | 1        |
| 2c            | 111300      | Liberty Transit Wheel Bottom Assembly, Medium, Short/Medium Depth Range<br><i>Used with 15" - 16.5" rear seat heights</i>                 | 1        |
| 2d            | 111301      | Liberty Transit Wheel Bottom Assembly, Large, Short/Medium Depth Range<br><i>Used with 17" - 18" rear seat heights</i>                    | 1        |
| 3             | 100658      | M6 Nylock Nut Blk Zn  | 1        |

| <b>Item Number</b> | <b>Part Number</b> | <b>Description</b>                          | <b>Quantity</b> |
|--------------------|--------------------|---|-----------------|
| 4                  | 102072             | Washer Nylon .234 ID x .406 OD x .062 Thick | 1               |
| 5                  | 100704             | M6 x 90 CLS8.8 DIN931 HHCS ZC               | 1               |

| <b>Liberty FT Transit Wheels</b> |                                |
|----------------------------------|--------------------------------|
| <b>Initial Rear Seat Height</b>  | <b>Height Adjustment Range</b> |
| 14"                              | N / A                          |
| 14.5"                            | N / A                          |
| 15"                              | 15" - 15.5"                    |
| 15.5"                            | 15.5" - 16.5"                  |
| 16"                              | 15.5" - 16.5"                  |
| 16.5"                            | 16.5" - 18"                    |
| 17"                              | 16.5" - 18"                    |
| 17.5"                            | 17.5" - 18"                    |
| 18"                              | 17.5" - 18"                    |

See Order Form for No-Go's