
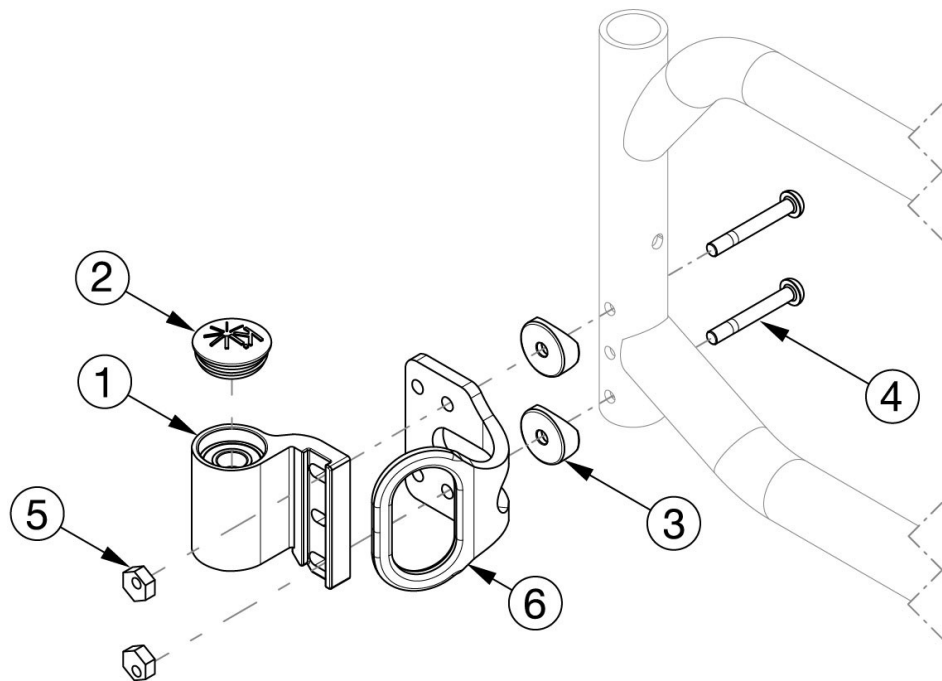


Catalyst E Caster Housing - Super Low

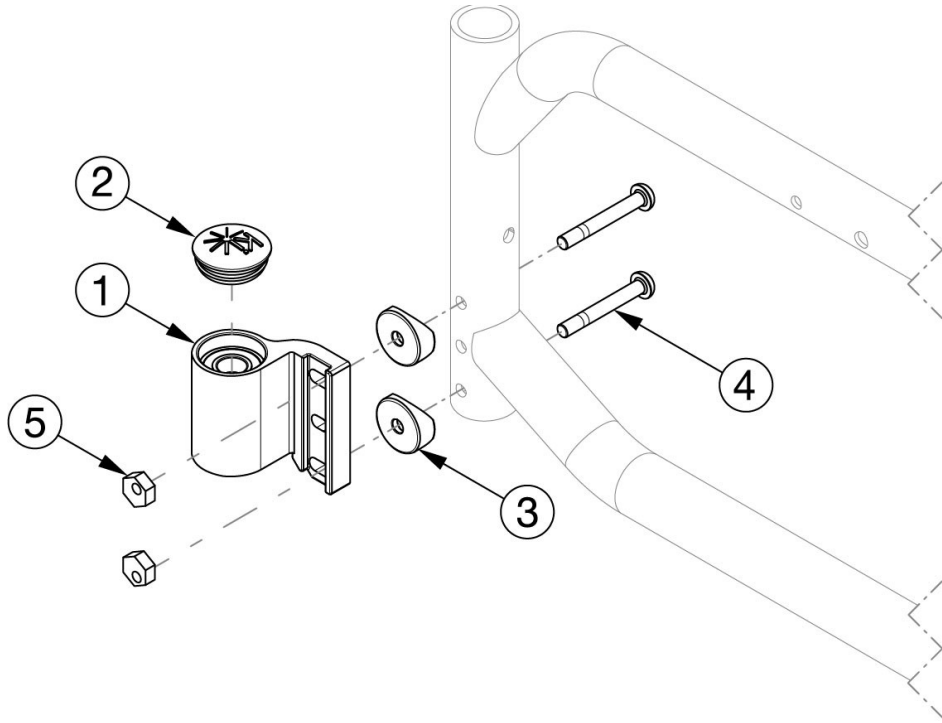

Reference order form for detailed front seat height matrix. Please Note: The shown orientation of the caster housing above is in the "Forward" position.

Item Number	Part Number	Description	Quantity
1a, 2-5	115819	Super Low Caster Housing Assembly, Black, Left, No Transit	1
1b, 2-5	115820	Super Low Caster Housing Assembly, Black, Right, No Transit	1
1a, 2-6	115822	Super Low Caster Housing Assembly, Black, Left, with Transit	1
1b, 2-6	115823	Super Low Caster Housing Assembly, Black, Right, with Transit	1
1a, 1b, 2-5	115821	Super Low Caster Housing Assembly, Black, PAIR, No Transit	1
1a, 1b, 2-6	115824	Super Low Caster Housing Assembly, Black, PAIR, with Transit	1
1a	111982	Super Low Caster Housing Assembly, Left	1
1b	111983	Super Low Caster Housing Assembly, Right	1
2	102005	Ki Caster Housing Cap	1
3	000056	Saddle 1 1/8"	2
4	200367	M6 x 45 Custom Shouldered BHCS	2
5	100745	Eccentric nut (threaded)	2
6	000426	Front Transit Bracket CAT	1