


(Discontinued) Catalyst 5Vx Rear Frame

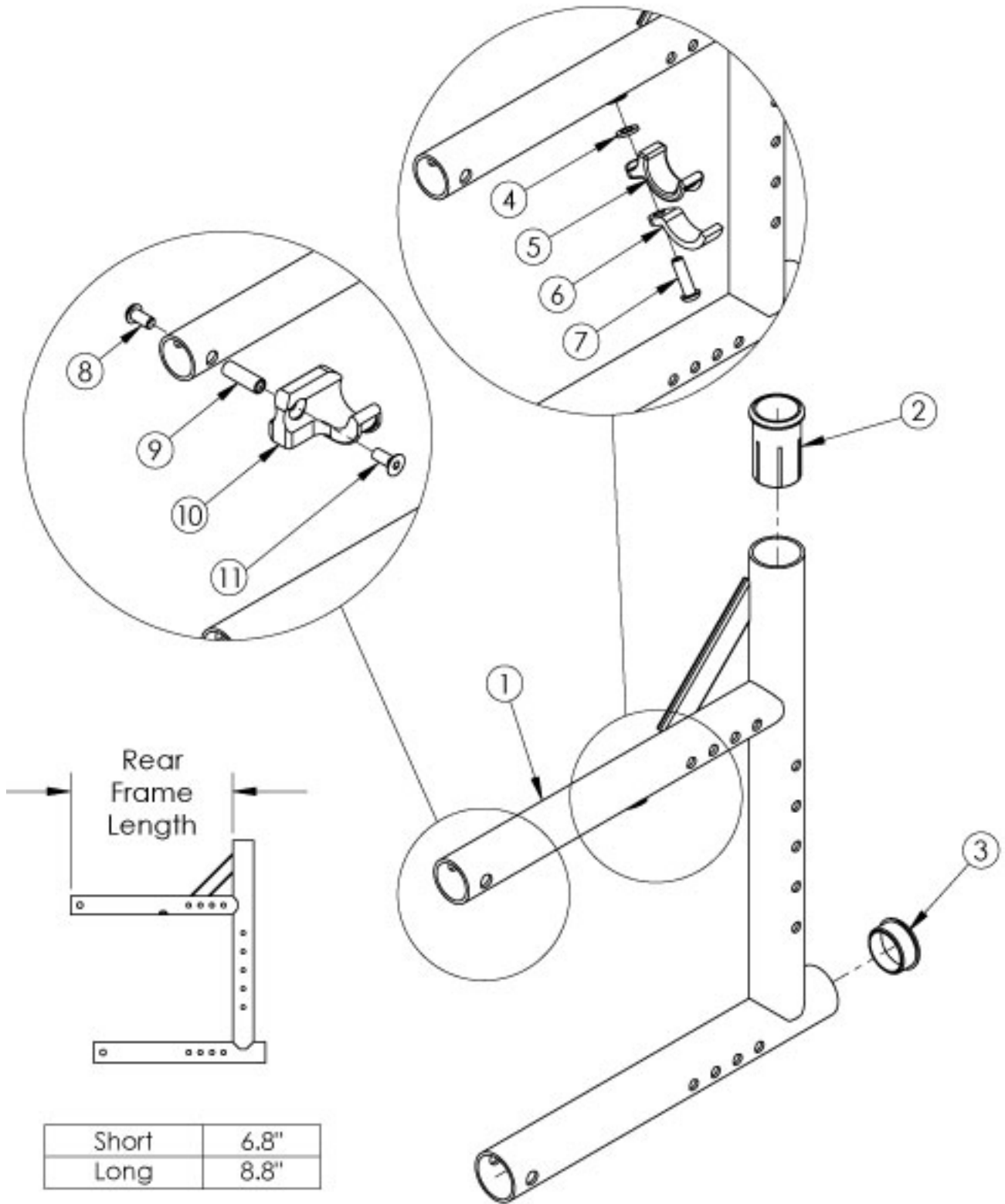

Use the short frame for 14-17" Seat Depths. Use the long frame for 18-20" Seat Depths. This rear frame was used on serial numbers prior to C5053765, C5HD003546, and CVX0091137. If switching to a new Seat Depth, new Height Adjustable Flip Back T-Arms may be needed.

Item Number	Part Number	Description	Quantity
4, 5, 6, 7a	501005	Seat Hook and Support Hardware Kit, Standard, Catalyst <i>Used with Standard Weight Capacity. This kit contains quantity of 1 each</i>	1
4, 5, 6, 7b	114528	Seat Hook and Support Hardware Kit, Standard, for Heavy Duty Catalyst <i>Used with Heavy Duty / 350 lbs Weight Capacity. This kit contains quantity of 1 each</i>	1
2	000030	Backrest Sleeve Cat	1
3	100532	Tube End Plug 1 1/8"	1
4	100831	1/2" Washer OD .040" Thick	1
5	000003	Seat Hook, Catalyst	1
6	000004	Seat Hook Support, Catalyst	1
7a	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch <i>Used with Standard Weight Capacity</i>	1
7b	101180	M6 x 20 SHCS BLK ZC <i>Used with Heavy Duty / 350 lbs Weight Capacity</i>	1
8	100667	M6 x 12 BTN SHCS BLKZC w/Patch	1
9	000065	M6 x 23.68 Threaded Barrel	1
10a	000960R	Seat Hook Open Frame Right Front Frame / Left Rear Frame, Catalyst	1
10b	000960L	Seat Hook Open Frame Left Front Frame / Right Rear Frame, Catalyst	1
11	101594	M6 x 14 CLS10.9 DIN7991 FSHCS BLKZC w/Patch	1

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer