


# Projection Handrim

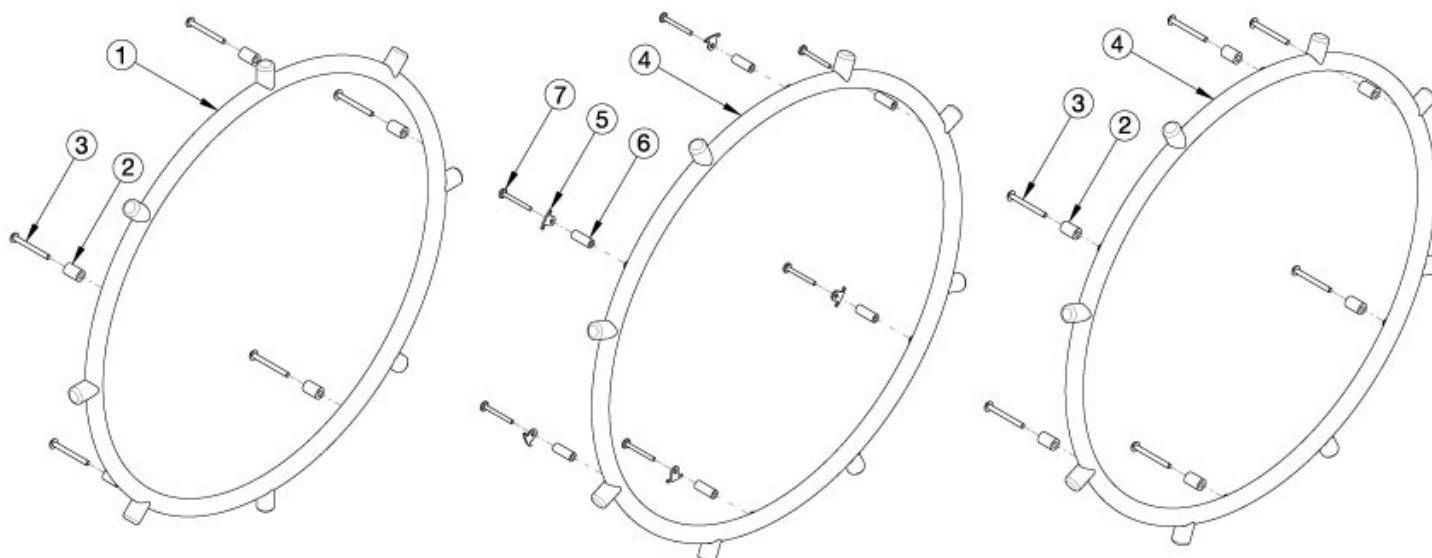

For more information see Handrim Hardware Chart. Quantity in Handrim Hardware Kits are for one wheel.

| Item Number | Part Number | Description                                                                                                                                             | Quantity |
|-------------|-------------|---------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| 2, 3a       | 502171      | Handrim Hardware Kit, 5 Rivet Nut, EACH<br><i>Used with X Core Mag wheels</i>                                                                           | 1        |
| 5-7         | 502168      | Handrim Hardware Kit, 6 Rivet Nut, EACH<br><i>Used with Ki Spoke, MAXX Spoke, and Spoke wheels</i>                                                      | 1        |
| 2, 3b       | 117377      | Handrim Hardware Kit, Standard Mount, MAXX Mag, EACH<br><i>Used with MAXX Mag wheels</i>                                                                | 1        |
| 4a          | 112824      | Handrim Projection, Vertical, 6 Rivet Nut 22"<br><i>Used on 22" Ki Spoke, 22" MAXX Spoke, 22" Spoke, and 22" MAXX Mag wheels. Replaced 100569.</i>      | 1        |
| 4b          | 112825      | Handrim Projection, Vertical, 6 Rivet Nut 24"<br><i>Used on 24" Ki Spoke, 24" MAXX Spoke, 24" Spoke, and 24" MAXX Mag wheels. Replaced part 200547.</i> | 1        |
| 4c          | 200548      | Handrim Projection, Vertical, 6 Rivet Nut 25"<br><i>Used on 25" Ki Spoke and 25" MAXX Spoke wheels</i>                                                  | 1        |
| 4d          | 112827      | Handrim Projection Vertical 6 Rivet Nut 26"<br><i>Used on 26" Ki Spoke and 26" MAXX Spoke wheels</i>                                                    | 1        |
| 2           | 100629      | Handrim Spacer Alum Ano Riv<br><i>Used with X Core Mag wheels (Qty 5 needed) and MAXX Mag wheels (Qty 6 needed)</i>                                     | 1        |
| 3a          | 100724      | M5 x 55 PH TRS HD M/S ZC<br><i>Used with X Core Mag wheels</i>                                                                                          | 5        |

| Item Number | Part Number | Description                                                                                                           | Quantity |
|-------------|-------------|-----------------------------------------------------------------------------------------------------------------------|----------|
| 3b          | 122064      | PHMS, M5x0.8x50, CLS4.8, PHL, BLZ, NYL,<br>DIN7985, Custom<br><i>Used with MAXX Mag wheels</i>                        | 6        |
| 5           | 100698      | Handrim Tab<br><i>Used with Ki Spoke, MAXX Spoke, and Spoke wheels</i>                                                | 6        |
| 6           | 100653      | Handrim Spacer Spoke<br><i>Used with Ki Spoke, MAXX Spoke, and Spoke wheels</i>                                       | 6        |
| 7           | 122063      | PHMS, M5x0.8x40, CLS4.8, PHL, BLZ, NYL,<br>DIN7985, Custom<br><i>Used with Ki Spoke, MAXX Spoke, and Spoke wheels</i> | 6        |

| Handrim Hardware Chart |                |                    |                                                              |                                              |                                                                  |                   |                |                  |                  |                    |                  |                      |                  |          |  |
|------------------------|----------------|--------------------|--------------------------------------------------------------|----------------------------------------------|------------------------------------------------------------------|-------------------|----------------|------------------|------------------|--------------------|------------------|----------------------|------------------|----------|--|
| Wheel                  |                | Handrim Connection |                                                              |                                              |                                                                  | Handrim           |                |                  |                  |                    |                  |                      |                  |          |  |
| Wheel                  | Wheel Part No. | Connection Points  | Rivet/Tab                                                    | Spacer*                                      | Screw*                                                           | Aluminum Anodized | Superlight     | Plastic Coated   | Projection       | Ergonomic Standard | Ergonomic LT     | Natural Fit Standard | Natural Fit LT   | Flex Rim |  |
| 22" Spoke              | 116730         | 5 or 6             | 116732 (Used with 5 Rivet)<br><br>100698 (Used with 6 Rivet) | 100653 (Not used w/ Superlight Handrim)      | Aluminum, Plastic Ctd, Projection<br>Ergonomic: 122063           | 101898 (5 Rivet)  |                | 101964 (5 Rivet) | 112824 (6 Rivet) | 113082 (6 Rivet)   | 113085 (6 Rivet) | 200538 (6 Rivet)     | 200201 (6 Rivet) |          |  |
| 24" Spoke              | 116728         | 5 or 6             | Not used w/ Superlight Handrims                              |                                              | Natural Fit: 100835<br><br>Superlight: 100669<br><br>Nut: 100657 | 100975 (5 Rivet)  | 101161 (6 Tab) | 100976 (5 Rivet) | 112825 (6 Rivet) | 113083 (6 Rivet)   | 113086 (6 Rivet) | 100793 (6 Rivet)     | 200202 (6 Rivet) |          |  |
| 18" MAXX Spoke         | 109285         | 3                  | 100698 (Not used w/ Superlight Handrim)                      | 100653 (Not used w/ Superlight Handrim)      | Aluminum, Plastic Ctd, Projection, Ergonomic: 122063             | 100206            |                |                  |                  |                    |                  |                      |                  |          |  |
| 20" MAXX Spoke         | 108244         | 6                  |                                                              |                                              | Natural Fit: 100835                                              | 200536            |                | 112819           |                  |                    |                  |                      |                  |          |  |
| 22" MAXX Spoke         | 105135         |                    |                                                              |                                              | Superlight: 100669                                               | 100560            |                | 112820           | 112824           | 113082             | 113085           | 200538               | 200201           |          |  |
| 24" MAXX Spoke         | 105136         |                    |                                                              |                                              | Nut: 100657                                                      | 200349            | 101161         | 112821           | 112825           | 113083             | 113086           | 100793               | 200202           |          |  |
| 25" MAXX Spoke         | 107436         |                    |                                                              |                                              | 200350                                                           |                   | 112822         | 200548           | 113084           | 113087             | 200539           | 200540               |                  |          |  |
| 26" MAXX Spoke         | 107437         |                    |                                                              |                                              | 200351                                                           |                   | 112823         | 200549           |                  |                    | 100907           | 101454               |                  |          |  |
| 20" MAXX Mag           | 111853         | 6                  |                                                              | Aluminum, Plastic Coated, Projection: 100629 | Aluminum, Plastic Coated, Projection, Ergonomic: 122064          | 200536            |                | 112819           |                  |                    |                  |                      |                  |          |  |
| 22" MAXX Mag           | 111854         |                    |                                                              | Natural Fit & Ergonomic: 101756              | Natural Fit: 101893                                              | 100560            |                | 112820           | 112824           | 113082             | 113085           | 200538               | 200201           |          |  |
| 24" MAXX Mag           | 111855         |                    |                                                              |                                              |                                                                  | 105137            |                | 112821           | 112825           | 113083             | 113086           | 100793               | 200202           |          |  |
| 24" Superlight         | 101159         | 6                  |                                                              |                                              | 100536                                                           | 100754            | 101161         | 100836           |                  | 113080             | 113081           | 100830               | 100828           |          |  |
| 25" Superlight         | 101160         |                    |                                                              |                                              |                                                                  |                   |                | 101160           | 101091           |                    |                  |                      | 101464           | 101460   |  |

\* Spacer and screw part numbers listed in the chart are for standard handrim mounting. If using close mount handrim mounting on aluminum anodized, plastic coated or projection handrims, use part number 100792 for the spacer, part number 122065 for the screw on Ki Spoke and MAXX Spoke wheels, part number 100666 for the screw on 5-Spoke X Core wheels, or part number 122063 for the screw on MAXX Mag wheels.

Effective 4/19/2024

| Handrim Hardware Chart (Continued) |                         |                    |           |         |                    |                   |            |                |            |                    |              |                      |                |                       |
|------------------------------------|-------------------------|--------------------|-----------|---------|--------------------|-------------------|------------|----------------|------------|--------------------|--------------|----------------------|----------------|-----------------------|
| Wheel                              |                         | Handrim Connection |           |         |                    | Handrim           |            |                |            |                    |              |                      |                |                       |
| Wheel                              | Wheel Part No.          | Connection Points  | Rivet/Tab | Spacer* | Screw*             | Aluminum Anodized | Superlight | Plastic Coated | Projection | Ergonomic Standard | Ergonomic LT | Natural Fit Standard | Natural Fit LT | Flex Rim              |
| 20" Spinerger Spox                 | See Spinerger Spox Page | 6                  |           |         | Ergonomic: 100536  | 103125            |            | 103179         |            |                    |              |                      |                |                       |
| 22" Spinerger Spox                 |                         |                    |           |         | All Others: 100669 | 100827            |            | 100808         |            |                    |              | 100889               | 100888         |                       |
| 24" Spinerger Spox                 |                         |                    |           |         |                    | 100766            | 101161     | 100615         |            |                    |              | 100830               | 100828         |                       |
| 25" Spinerger Spox                 |                         |                    |           |         | Nut: 100657        | 100767            | 101160     | 100765         |            |                    |              | 101464               | 101460         |                       |
| 26" Spinerger Spox                 |                         |                    |           |         |                    | 101477            |            | 101148         |            |                    |              | 200200               | 100950         |                       |
| 22" Spinerger LX                   | See Spinerger LX Page   | 6                  |           |         | Ergonomic: 100536  | 100827            |            | 100808         |            |                    |              | 100889               | 100888         |                       |
| 24" Spinerger LX                   |                         |                    |           |         | All Others: 100669 | 100766            | 101161     | 100615         |            | 113080             | 113081       | 100830               | 100828         | See Spinerger LX Page |
| 25" Spinerger LX                   |                         |                    |           |         |                    | 100767            | 101160     | 100765         |            |                    |              | 101464               | 101460         |                       |
| 26" Spinerger LX                   |                         |                    |           |         | Nut: 100657        | 101477            |            | 101148         |            |                    |              | 200200               | 100950         |                       |

| Handrim Hardware Chart (Discontinued Wheels) |                |                    |           |                                           |                                           |                   |            |                |            |                      |                |          |
|----------------------------------------------|----------------|--------------------|-----------|-------------------------------------------|-------------------------------------------|-------------------|------------|----------------|------------|----------------------|----------------|----------|
| Wheel                                        |                | Handrim Connection |           |                                           |                                           | Handrim           |            |                |            |                      |                |          |
| Wheel                                        | Wheel Part No. | Connection Points  | Rivet/Tab | Spacer*                                   | Screw*                                    | Aluminum Anodized | Superlight | Plastic Coated | Projection | Natural Fit Standard | Natural Fit LT | Flex Rim |
| 18" Ki Spoke                                 | 200529         | 3                  | 100698    | 100653                                    | Aluminum, Plastic Ctd, Projection: 122063 | 100206            |            | 101106         |            |                      |                |          |
| 20" Ki Spoke                                 | 200530         | 6                  |           |                                           |                                           | 100536            |            | 200542         |            |                      |                |          |
| 22" Ki Spoke                                 | 200531         |                    |           |                                           |                                           |                   | 100560     | 100576         | 100569     | 200538               | 200201         |          |
| 24" Ki Spoke                                 | 200532         |                    |           |                                           |                                           |                   | 200349     | 100577         | 200547     | 100793               | 200202         |          |
| 25" Ki Spoke                                 | 200533         |                    |           |                                           | 200350                                    |                   | 101870     | 200548         | 200539     | 200540               |                |          |
| 26" Ki Spoke                                 | 200534         |                    |           |                                           | 200351                                    |                   | 100578     | 200549         | 100907     | 101454               |                |          |
| 20" 5-Spoke X Core                           | 101961         | 5                  |           | Aluminum, Projection, Natural Fit: 100629 | 100724                                    | 101897            |            | 101963         |            |                      |                |          |
| 22" 5-Spoke X Core                           | 101962         |                    |           |                                           |                                           | 101898            |            | 101964         |            |                      |                |          |
| 24" 5-Spoke X Core                           | 100960         |                    |           |                                           |                                           | 100975            |            | 100976         | 200546     |                      |                |          |
| 24" 5-Spoke X Core                           | 100960         |                    |           | Plastic Ctd: 101756                       | 101893                                    |                   |            |                |            | 100768               | 100769         |          |

\* Spacer and screw part numbers listed in the chart are for standard handrim mounting. If using close mount handrim mounting on aluminum anodized, plastic coated or projection handrims, use part number 100792 for the spacer, part number 122065 for the screw on Ki Spoke and MAXX Spoke wheels, part number 100666 for the screw on 5-Spoke X Core wheels, or part number 122063 for the screw on MAXX Mag wheels.

Effective 4/19/2024