
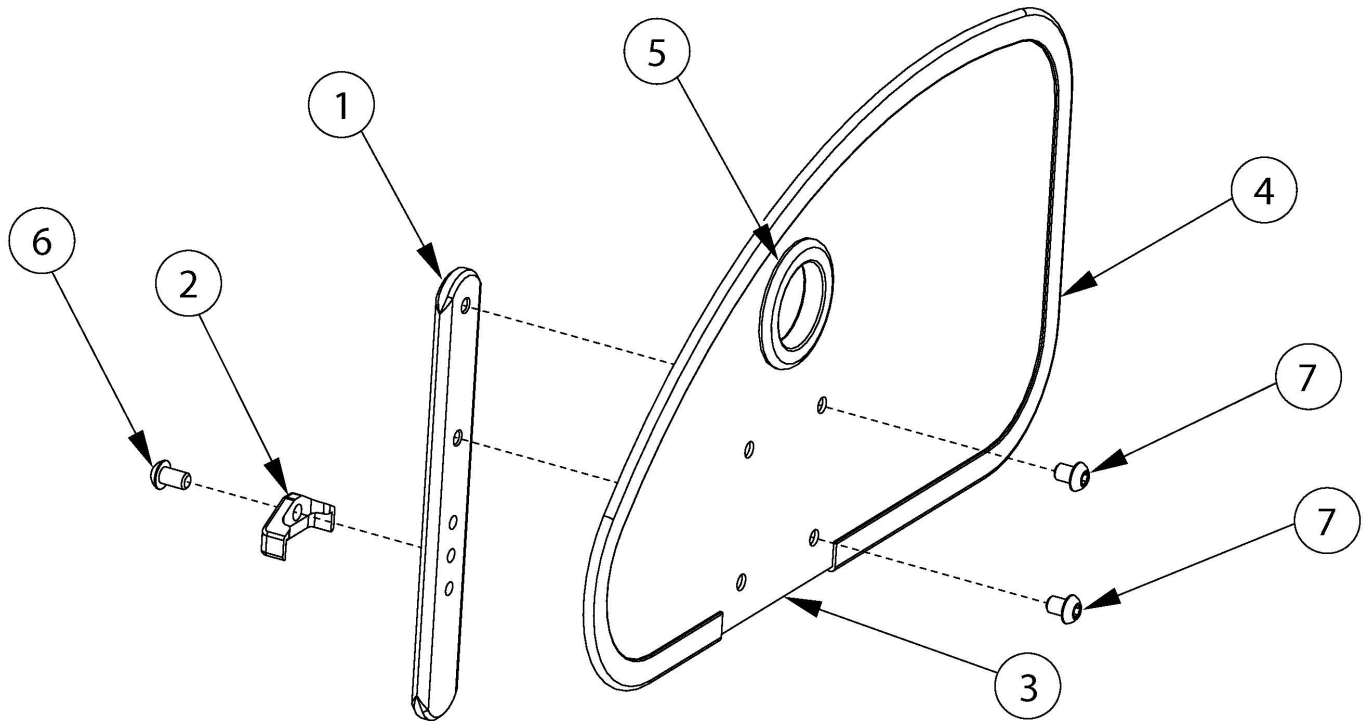


Catalyst 5Ti Aluminum Straight Side Guard

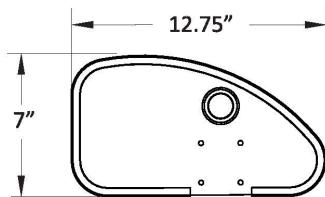

If retrofitting, side guard receivers may be required

Item Number	Part Number	Description	Quantity
1, 2, 3a, 4-7	108346	Aluminum Straight Side Guard Assembly Left, Black Matte	1
1, 2, 3b, 4-7	108347	Aluminum Straight Side Guard Assembly Right, Black Matte	1
1, 2, 3a, 3b, 4-7	108348	Aluminum Straight Side Guard Assembly (PAIR), Black Matte	1
1	000140	Side Guard Post	1
2	001342	Side Guard Post Stop	1
3a, 4, 5	108342	Aluminum Straight Side Guard Left, Black Matte	1
3b, 4, 5	108343	Aluminum Straight Side Guard Right, Black Matte	1
6	101184	M6 x 10 BTN SHCS BLKZC w/Patch	1

Item Number	Part Number	Description	Quantity
7	101700	M6 x 8 BTN SHCS BLK ZC	2