

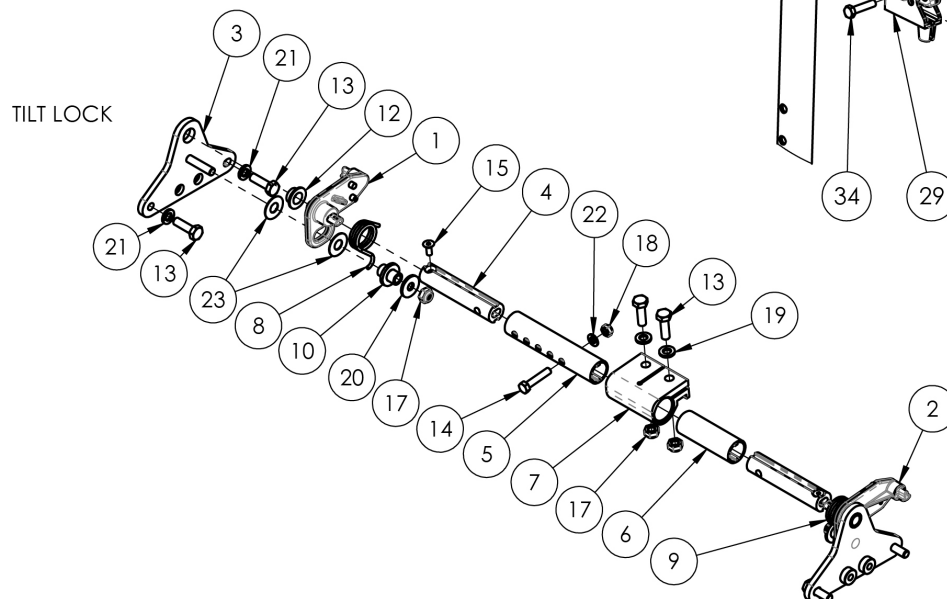
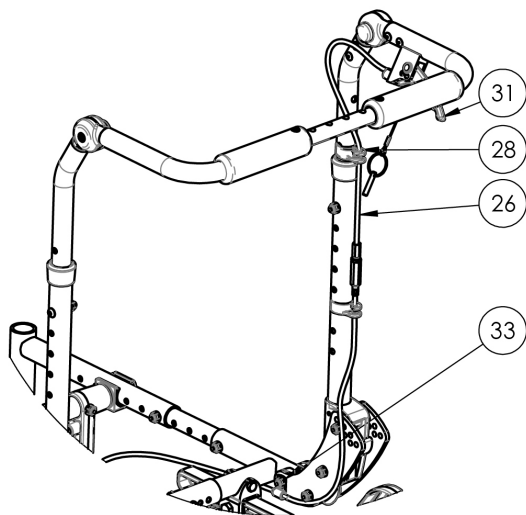
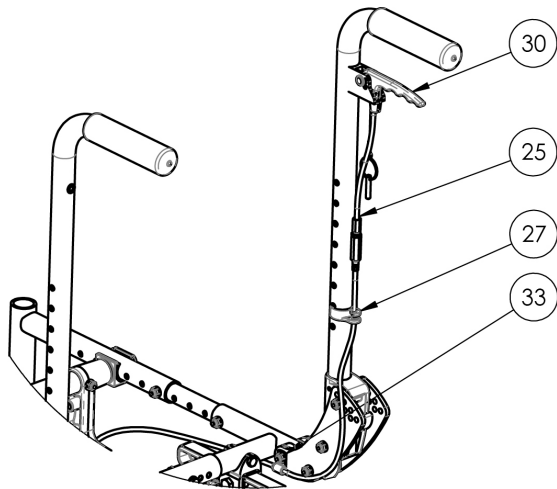
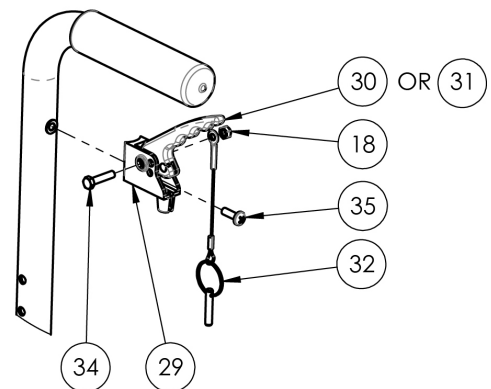


(Discontinued 1) Focus CR Hand Tilt

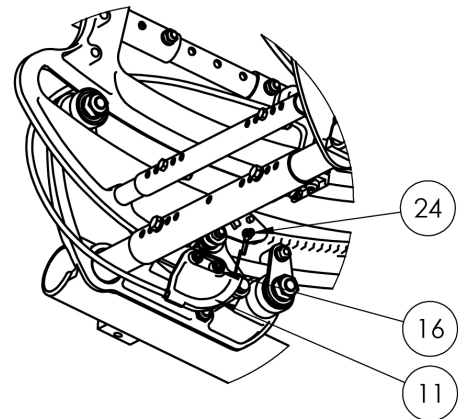

CABLE DRIVE SYSTEM

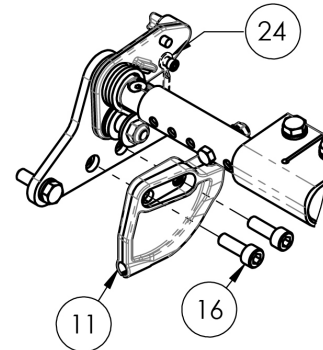

This Focus CR Hand Tilt is discontinued and was used on serial numbers prior to FCR0013026.

Item Number	Part Number	Description	Quantity	Retail
1	001102	Cog Lever Tilt Lock Right TILT	1	\$60.72
2	001103	Cog Lever Tilt Lock Left TILT	1	\$60.72
3	001101	Mounting Bracket Tilt Lock TILT	2	\$78.48
5	001282	Drive Tube TILT	1	\$11.15
6a	001254	Growth Strut Short TILT	1	\$8.04
6b	001255	Growth Strut Medium TILT	1	\$8.04
6c	001256	Growth Strut Long TILT	1	\$8.04
8	001109	Spring Tilt Lock Right TILT	1	\$4.71
9	001110	Spring Tilt Lock Left TILT	1	\$4.71
10	001104	Retainer Lock Lever TILT	2	\$5.71
11	001112	Mount Cable 90 Deg TILT	1	\$9.89
12	001111	Bearing GFI-0607-04 TILT	2	\$3.15
13	101823	M6 x 20 HHCS Blk Zn	4	\$1.05
14	101821	M5 x 25 HHCS Blk Zn	1	\$1.05
15	101818	M4 x 10 FHCS BLKZC w/Patch	2	\$1.05
16	101844	M6 x 18 SHCS Blk Zn	2	\$1.05
17	100658	M6 Nylock Nut Blk Zn	2	\$1.05
18	100657	M5 Nylock Nut Blk Zn	1	\$0.53
19	100746	M6 Flat Washer Blk Zn	2	\$1.58
20	101853	M6 Washer Blk Zn	2	\$0.53
21	101835	M6 Lock Washer Blk Zn	4	\$0.53
22	101856	Flat Washer, M5, Blk Zn	1	\$0.53
23	102082	Washer Nylon .406IDX.75ODX.040	4	\$0.56
24	101842	M3 x 8 SHCS Blk Zn	1	\$1.05
25a	001113	Cable Tilt Short TILT	1	\$100.60
25b	001114	Cable Tilt Medium TILT	1	\$100.60

Item Number	Part Number	Description	Quantity	Retail
26a	001115	Cable Tilt Long TILT	1	\$124.65
26b	001116	Cable Tilt Extra Long TILT	1	\$124.65
27	001117	Cable Clip 1"	1	\$3.83
28	001118	Cable Clip 7/8"	1	\$3.83
33	001239	P-Clip 3/16"	1	\$6.30
34	101820	M5 x 22 HHCS Blk Zn	1	\$5.78
35	100734	M5 x 16 PHMS ZC	1	\$1.58