



Please read these instructions carefully before beginning the installation. Failure to understand and follow installation instructions may result in injury to installer and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at 800-333-4000.

Hanger receiver parts:

1. 1 ea Receiver assembly RH
2. 1 ea Receiver assembly LH
3. 2 ea Wire harness



Required tools:






1. 3mm hex key
2. 5mm hex key
3. Torque wrench

A. Removing receiver assembly:

1. Using the 3mm hex key remove screw (A) on the bottom side of the seat pan, Fig 1.
2. Loosen screw (B) with the 5mm hex key and then pull the receiver out, Fig 2. Discard parts.

B. Installing receiver assembly:

1. Using the 3mm hex key remove screw (A) from the new receiver, Fig 3.
2. Install the new receiver into the seat pan. Line up the screw hole with the slot under the pan, Fig 4. Reinsert screw, (A) and tighten securely. Torque to 27 - 32 in-lbs. 
3. Slide hanger receiver to desired width position. Using the torque wrench tighten screw (B) to 60 - 70 in-lbs, Fig 5. 
4. For power ELR hangers, insert the small end of the wire harness (C) into the top of the receiver. Tighten securely set screw (D) with a 3mm hex key to secure the harness to receiver, Fig 5. Torque to 6 - 8 in-lbs.

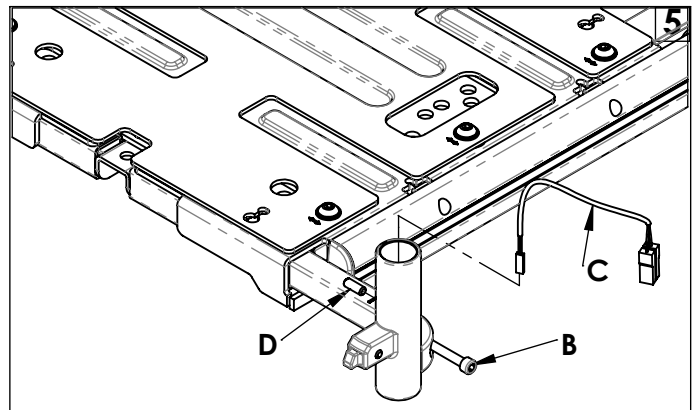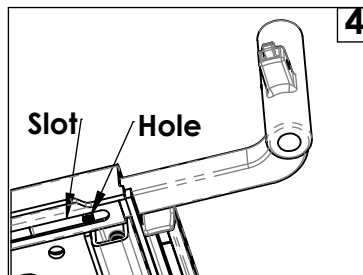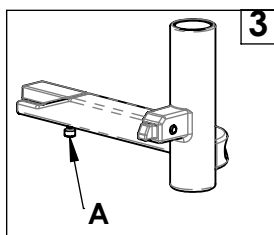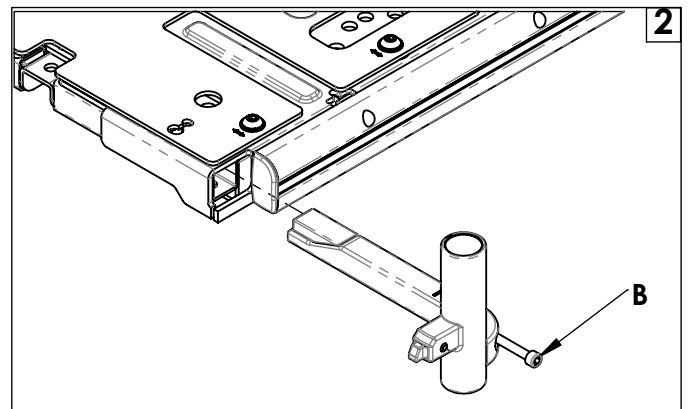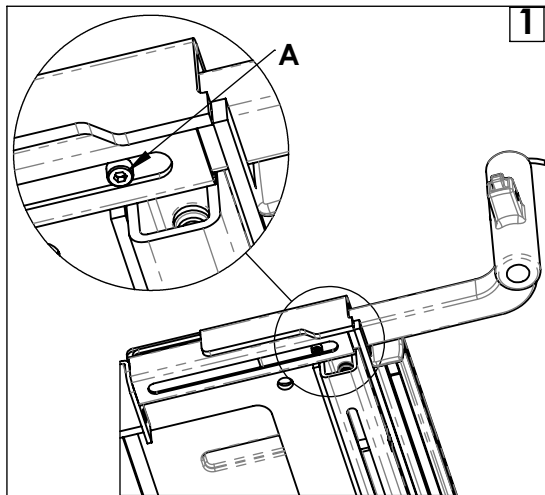

DEALER/TECHNICIAN WARNING



WARNING
Attention dealers and qualified technicians, do not operate or service this device without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Quickie Technical Service Department before operating and/or servicing the Quickie device. Failure to do so may result in damage and/or injury.

Find more information and important warnings see the wheelchair owner's manual or at: www.SunriseMedical.com.

C. Inspect:

1. Check that screw (A) is in the slot.
2. Check torque of screws (A & B). 