


# Focus CR Height Adjustable Low T-Arm

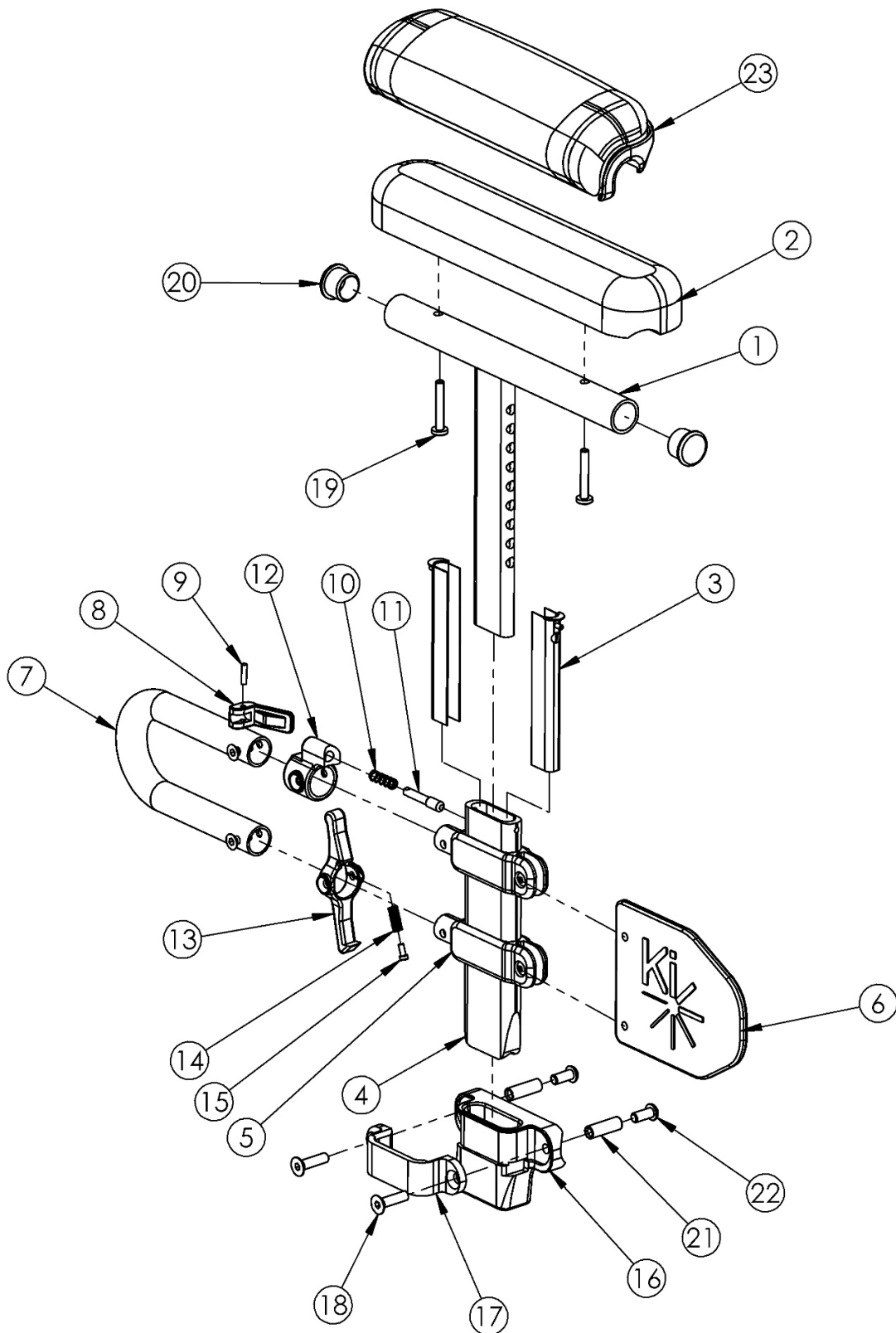

Not available with Heavy Duty Option

Item Number	Part Number	Description	Quantity	Retail
1a, 2b, 3-18, 19a, 20-22	113452	Height Adj T-Arm Assm Low, Desk w/ Std Pad and Receiver PAIR	1	\$440.32
1a, 2b, 3-18, 19a, 20-22	113347	Height Adj T-Arm Assm Low, Desk w/ Std Pad and Receiver Right	1	\$220.16
1a, 2b, 3-18, 19a, 20-22	113348	Height Adj T-Arm Assm Low, Desk w/ Std Pad and Receiver Left	1	\$220.16
1a, 2b, 3-15, 19a, 20	503167	Height Adj T-Arm Assm Low, Desk w/ Std Pad Right	1	\$160.81
1a, 2b, 3-15, 19a, 20	503168	Height Adj T-Arm Assm Low, Desk w/ Std Pad Left	1	\$160.81
1b, 2a, 3-18, 19a, 20-22	113453	Height Adj T-Arm Assm Low, Full w/ Std Pad and Receiver PAIR	1	\$440.32
1b, 2a, 3-18, 19a, 20-22	113349	Height Adj T-Arm Assm Low, Full w/ Std Pad and Receiver Right	1	\$220.16
1b, 2a, 3-18, 19a, 20-22	113350	Height Adj T-Arm Assm Low, Full w/ Std Pad and Receiver Left	1	\$220.16
1b, 2a, 3-15, 19a, 20	503169	Height Adj T-Arm Assm Low, Full w/ Std Pad Right	1	\$160.81
1b, 2a, 3-15, 19a, 20	503170	Height Adj T-Arm Assm Low, Full w/ Std Pad Left	1	\$160.81
1b, 3-18, 19b, 20-22, 23a	113454	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and Receiver PAIR	1	\$458.64
1a, 3-18, 19b, 20-22, 23b	113351	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and Receiver Right	1	\$229.32
1a, 3-18, 19b, 20-22, 23b	113352	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and Receiver Left	1	\$229.32
1a, 3-15, 19b, 20, 23b	503183	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad Right	1	\$169.97
1a, 3-15, 19b, 20, 23b	503184	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad Left	1	\$169.97
1b, 3-18, 19b, 20-22, 23a	113455	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and Receiver PAIR	1	\$458.64
1b, 3-18, 19b, 20-22, 23a	113353	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and Receiver Right	1	\$229.32
1b, 3-18, 19b, 20-22, 23a	113354	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and Receiver Left	1	\$229.32
1b, 3-15, 19b, 20, 23a	503185	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad Right	1	\$169.97
1b, 3-15, 19b, 20, 23a	503186	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad Left	1	\$169.97
1a, 3-15, 20	002681	Arm T Height Adj Low, Desk Right Assembly	1	\$134.01
1a, 3-15, 20	002682	Arm T Height Adj Low, Desk Left Assembly	1	\$134.01
1b, 3-15, 20	002683	Arm T Height Adj Low, Full Right Assembly	1	\$134.01
1b, 3-15, 20	002684	Arm T Height Adj Low, Full Left Assembly	1	\$134.01
16-18, 21, 22	501730	Height Adj T-Arm Receiver Assm 1" CAT	1	\$55.66
1a, 20	002648	Post Weldment, T-Arm, Low, Desk	1	\$67.08

Item Number	Part Number	Description	Quantity	Retail
1b, 20	002649	Post Weldment, T-Arm, Low, Full	1	\$69.35
2a	100500	Arm Pad Full	1	\$23.64
2b	100501	Arm Pad Desk	1	\$23.64
3	000090	Height Adj T Armrest Shim	2	\$2.55
4	002105	Tube, Outer, T-Arm, Low	1	\$29.62
5	000092	Height Adj T Armrest Clamp	2	\$8.97
6	002107	Sideguard, T-Arm, Low	1	\$8.10
7	002104	Transfer Tube, T-Arm, Low	1	\$16.81
8	000098	Height Adj Armrest Lever	1	\$3.58
9	200257	Dowel Pin 1/8 x 1/2	1	\$2.63
10	000134	Height Adj Armrest Lever Spring	1	\$1.11
11	000099	Height Adj Armrest Lever Pin	1	\$19.13
12	000096	Height Adj T Armrest Pin Lock Housing	1	\$14.71
8-12	500541	Arm Height Adj T Upper Lever Assembly	1	\$40.56
13	002106	Lever, T-Arm, Low	1	\$3.47
14	000135	Height Adj T Armrest Release Lever Spring	1	\$1.09
15	000152	Height Adj T Armrest Release Spring Button	1	\$1.09
16	000087	Height Adjustable T-Arm Receiver 1" CAT	1	\$15.25
17	000088	Height Adj T Armrest Bracket	1	\$27.42
18	100866	M6 x 20 CLS12.9 DIN7991 FSHCS Blk Zc w/Patch	2	\$2.63
19a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2	\$1.58
19b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2	\$1.58
20	100803	Tube End Plug 7/8"	2	\$1.58
21	000065	M6 x 23.68 Threaded Barrel	2	\$4.13
22	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.58
23a	102452	Waterfall Arm Pad Full	1	\$32.80

Item Number	Part Number	Description	Quantity	Retail
23b	102521	Waterfall Arm Pad Desk	1	\$32.80