




(Discontinued) Klik Caster Forks and Stems

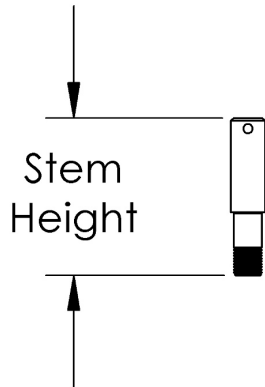

Std	2.0"
+ 3/4"	2.8"

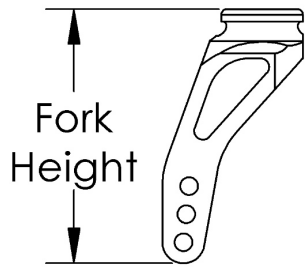

Ex. Short	3.6"
Short	4.5"
Tall	4.9"

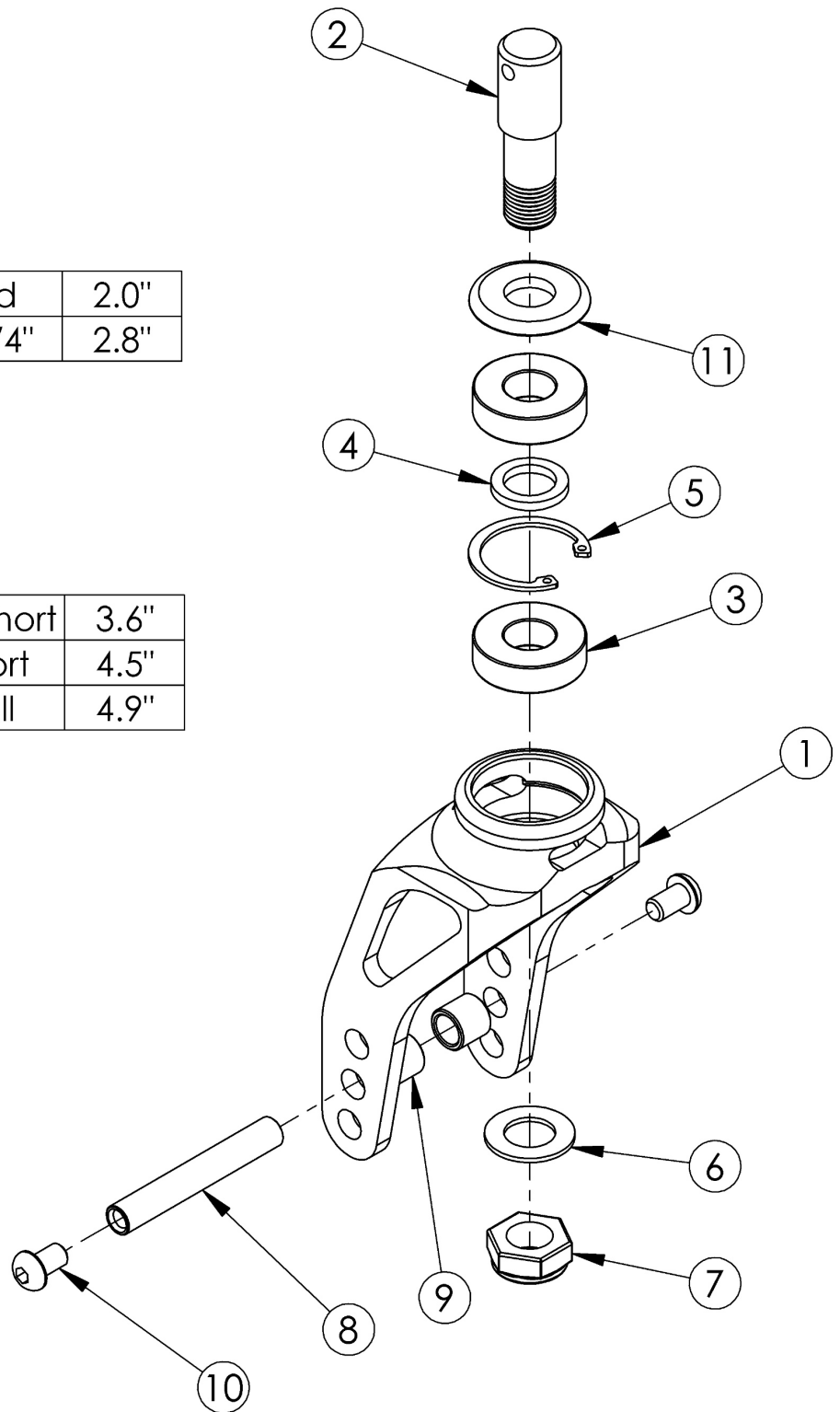

Item Number	Part Number	Description	Quantity	Retail
1b, 2b, 3-7, 11	502646_ _ _	Color Anodized Fork and Stem Assembly Extra Short 3/4	1	\$117.18
1d, 2b, 3-7, 11	500783_ _ _	Color Anodized Fork and Stem Assembly Short 3/4	1	\$122.80
1f, 2b, 3-7, 11	500785_ _ _	Color Anodized Fork and Stem Assembly Tall 3/4	1	\$122.80
1b, 3-5	001812_ _ _	Color Anodized Extra Short Caster Fork w/ Bearing Housing LW <i>Replaced with 004361_ _ _</i>	1	\$88.44
1d, 3-5	000927_ _ _	Color Anodized Caster Fork Short w/ Bearing Housing RG <i>Replaced with 004354_ _ _</i>	1	\$91.32
1f, 3-5	000928_ _ _	Color Anodized Caster Fork Tall w/ Bearing Housing RG <i>Replaced with 004362_ _ _</i>	1	\$92.22 - \$94.06
2b	000930INST	Caster Stem In Line +3/4 Rogue/Clik <i>Replaced with 004371INST</i>	1	\$12.82
3	100902	Bearing 1 1/8" x 1/2" R8RS	2	\$6.30
6	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	1	\$1.05
7	101459	1/2-20 NTE JAM NYL INS L/N BLK ZC	1	\$1.58
8	000422	M6 x 56 Threaded Barrel	1	\$10.64
9	100792	Caster Spacer D8 x D12 x 10mm	2	\$2.14
10	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.58
11a	001355	Caster Bearing Cap Black Anodized <i>Used with 3/4" stem only</i>	1	\$3.69
11b	001355_ _ _	Color Anodized Caster Bearing Cap	1	\$13.29

Color Anodizing Samples

