


# Dynamic 5th Wheel

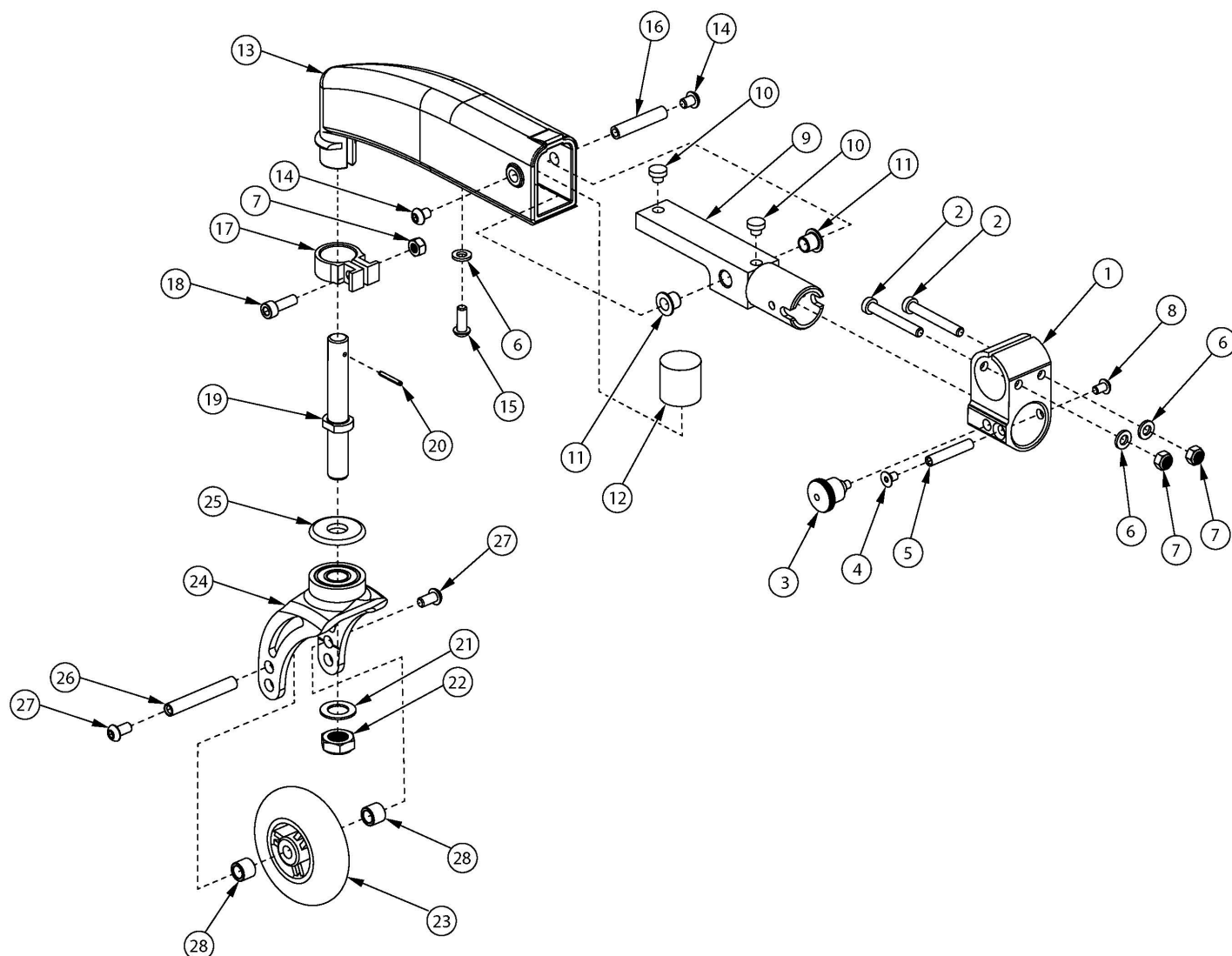

A new camber tube is needed if retrofitting to the Dynamic 5th Wheel. The Dynamic 5th wheel assemblies contain two of the camber tube mounting clamp set screw, part number 105015. For a complete Dynamic 5th Wheel Assembly, select the Dynamic 5th Wheel Clamp Assembly (502464) and the desired Dynamic 5th Wheel Assembly with Caster.

| Item Number                                                    | Part Number | Description                                                               | Quantity |
|----------------------------------------------------------------|-------------|---------------------------------------------------------------------------|----------|
| 9a, 10, 11, 12a, 12b, 13 - 16, 17a, 18 - 22, 23a, 24a, 25 - 28 | 107343      | Dynamic 5th Wheel Assembly w/ 3" Rollerblade for 18" Rear Wheels          | 1        |
| 9a, 10, 11, 12a, 12b, 13 - 16, 17b, 18 - 22, 23a, 24a, 25 - 28 | 107343_---  | Dynamic 5th Wheel Assembly w/ 3" Rollerblade for 18" Rear Wheels Anodized | 1        |

| Item Number                                                                | Part Number | Description                                                                          | Quantity |
|----------------------------------------------------------------------------|-------------|--------------------------------------------------------------------------------------|----------|
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23a, 24b,<br>25 - 28 | 107344      | Dynamic 5th Wheel Assembly w/ 3" Rollerblade<br>for 20" Rear Wheels                  | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23a, 24b,<br>25 - 28 | 107344_ _ _ | Dynamic 5th Wheel Assembly w/ 3" Rollerblade<br>for 20" Rear Wheels Anodized         | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23a, 24c,<br>25 - 28 | 107345      | Dynamic 5th Wheel Assembly w/ 3" Rollerblade<br>for 22" Rear Wheels                  | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23a, 24c,<br>25 - 28 | 107345_ _ _ | Dynamic 5th Wheel Assembly w/ 3" Rollerblade<br>for 22" Rear Wheels Anodized         | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23b, 24a,<br>25 - 28 | 107346      | Dynamic 5th Wheel Assembly w/ 3" Lighted<br>Rollerblade for 18" Rear Wheels          | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23b, 24a,<br>25 - 28 | 107346_ _ _ | Dynamic 5th Wheel Assembly w/ 3" Lighted<br>Rollerblade for 18" Rear Wheels Anodized | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23b, 24b,<br>25 - 28 | 107347      | Dynamic 5th Wheel Assembly w/ 3" Lighted<br>Rollerblade for 20" Rear Wheels          | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23b, 24b,<br>25 - 28 | 107347_ _ _ | Dynamic 5th Wheel Assembly w/ 3" Lighted<br>Rollerblade for 20" Rear Wheels Anodized | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23b, 24c,<br>25 - 28 | 107348      | Dynamic 5th Wheel Assembly w/ 3" Lighted<br>Rollerblade for 22" Rear Wheels          | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23b, 24c,<br>25 - 28 | 107348_ _ _ | Dynamic 5th Wheel Assembly w/ 3" Lighted<br>Rollerblade for 22" Rear Wheels Anodized | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23c, 24a,<br>25 - 28 | 107349      | Dynamic 5th Wheel Assembly w/ 4" Rollerblade<br>for 20" Rear Wheels                  | 1        |

| Item Number                                                                | Part Number | Description                                                                          | Quantity |
|----------------------------------------------------------------------------|-------------|--------------------------------------------------------------------------------------|----------|
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23c, 24a,<br>25 - 28 | 107349_ _ _ | Dynamic 5th Wheel Assembly w/ 4" Rollerblade<br>for 20" Rear Wheels Anodized         | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23c, 24c,<br>25 - 28 | 107350      | Dynamic 5th Wheel Assembly w/ 4" Rollerblade<br>for 22" Rear Wheels                  | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23c, 24c,<br>25 - 28 | 107350_ _ _ | Dynamic 5th Wheel Assembly w/ 4" Rollerblade<br>for 22" Rear Wheels Anodized         | 1        |
| 9b, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23c, 24c,<br>25 - 28 | 107351      | Dynamic 5th Wheel Assembly w/ 4" Rollerblade<br>for 24" Rear Wheels                  | 1        |
| 9b, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23c, 24c,<br>25 - 28 | 107351_ _ _ | Dynamic 5th Wheel Assembly w/ 4" Rollerblade<br>for 24" Rear Wheels Anodized         | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23d, 24a,<br>25 - 28 | 107352      | Dynamic 5th Wheel Assembly w/ 4" Lighted<br>Rollerblade for 20" Rear Wheels          | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23d, 24a,<br>25 - 28 | 107352_ _ _ | Dynamic 5th Wheel Assembly w/ 4" Lighted<br>Rollerblade for 20" Rear Wheels Anodized | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23d, 24c,<br>25 - 28 | 107353      | Dynamic 5th Wheel Assembly w/ 4" Lighted<br>Rollerblade for 22" Rear Wheels          | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23d, 24c,<br>25 - 28 | 107353_ _ _ | Dynamic 5th Wheel Assembly w/ 4" Lighted<br>Rollerblade for 22" Rear Wheels Anodized | 1        |
| 9b, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23d, 24c,<br>25 - 28 | 107354      | Dynamic 5th Wheel Assembly w/ 4" Lighted<br>Rollerblade for 24" Rear Wheels          | 1        |
| 9b, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23d, 24c,<br>25 - 28 | 107354_ _ _ | Dynamic 5th Wheel Assembly w/ 4" Lighted<br>Rollerblade for 24" Rear Wheels Anodized | 1        |

| Item Number                                                                | Part Number | Description                                                               | Quantity |
|----------------------------------------------------------------------------|-------------|---------------------------------------------------------------------------|----------|
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23e, 24a,<br>25 - 28 | 107355      | Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for<br>20" Rear Wheels          | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23e, 24a,<br>25 - 28 | 107355_ _ _ | Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for<br>20" Rear Wheels Anodized | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23e, 24c,<br>25 - 28 | 107356      | Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for<br>22" Rear Wheels          | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23e, 24c,<br>25 - 28 | 107356_ _ _ | Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for<br>22" Rear Wheels Anodized | 1        |
| 9b, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23e, 24c,<br>25 - 28 | 107357      | Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for<br>24" Rear Wheels          | 1        |
| 9b, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23e, 24c,<br>25 - 28 | 107357_ _ _ | Dynamic 5th Wheel Assembly w/ 4 x 1" Poly for<br>24" Rear Wheels Anodized | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23f, 24c, 25<br>- 28 | 107358      | Dynamic 5th Wheel Assembly w/ 5 x 1" Poly for<br>22" Rear Wheels          | 1        |
| 9a, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23f, 24c, 25<br>- 29 | 107358_ _ _ | Dynamic 5th Wheel Assembly w/ 5 x 1" Poly for<br>22" Rear Wheels Anodized | 1        |
| 9b, 10, 11,<br>12a, 12b, 13 -<br>16, 17a, 18 -<br>22, 23f, 24b,<br>25 - 28 | 107359      | Dynamic 5th Wheel Assembly w/ 5 x 1" Poly for<br>24" Rear Wheels          | 1        |
| 9b, 10, 11,<br>12a, 12b, 13 -<br>16, 17b, 18 -<br>22, 23f, 24b,<br>25 - 28 | 107359_ _ _ | Dynamic 5th Wheel Assembly w/ 5 x 1" Poly for<br>24" Rear Wheels Anodized | 1        |
| 1 - 8                                                                      | 502464      | Dynamic 5th Wheel Clamp Assembly                                          | 1        |
| 1                                                                          | 001807      | 5th Wheel Mount                                                           | 1        |
| 2                                                                          | 101123      | M6 x 45 Low SHCS BLK ZC                                                   | 2        |
| 3                                                                          | 002050      | Mini Indexing Plunger                                                     | 1        |

| Item Number | Part Number | Description                                                    | Quantity |
|-------------|-------------|----------------------------------------------------------------|----------|
| 4           | 102308      | M5 x 8mm FHCS Blk Zc                                           | 1        |
| 5           | 001813      | Barrel 5th Wheel                                               | 1        |
| 6           | 100746      | M6 Flat Washer Blk Zn                                          | 3        |
| 7           | 100658      | M6 Nylock Nut Blk Zn                                           | 3        |
| 8           | 102621      | M5 x 8 BHCS BLK ZC                                             | 1        |
| 9a          | 001808      | Bump Stop, 5th Wheel<br><i>Used with 18" - 22" rear wheels</i> | 1        |
| 9b          | 002362      | Bump Stop Long, 5th Wheel<br><i>Used with 24" rear wheels</i>  | 1        |
| 10          | 102631      | Stop Bumper                                                    | 2        |
| 11          | 002041      | Flange Bushing For M6                                          | 2        |
| 12a         | 002409      | 5th Wheel Bumper, 40 Durometer, Yellow, Soft Elastomer         | 1        |
| 12b         | 002410      | 5th Wheel Bumper, 60 Durometer, Black, Firm Elastomer          | 1        |
| 13          | 001809      | Swing Arm Composite, 5th Wheel                                 | 1        |
| 14          | 101700      | M6 x 8 BTN SHCS BLK ZC                                         | 2        |
| 15          | 102634      | M6 X 1.0 X 16 BHCS BLK ZC                                      | 1        |
| 16          | 001814      | Barrel Swing Arm, 5th Wheel                                    | 1        |
| 17a         | 001810      | Split Clamp 3/4" Black                                         | 1        |
| 17b         | 001810_ _ _ | Color Anodized Split Clamp 3/4""                               | 1        |
| 18          | 101844      | M6 x 18 SHCS Blk Zn                                            | 1        |
| 19          | 001811INST  | Stem 5th Wheel                                                 | 1        |
| 20          | 102613      | .094 X 75 SPRING PIN ZINC PLATED                               | 1        |
| 21          | 100682      | Washer 1/2" .518 x .875 x .047 F/W Black Zinc                  | 1        |
| 22          | 101459      | 1/2-20 NTE JAM NYL INS L/N BLK ZC                              | 1        |
| 23a         | 100790      | Caster 3" Rollerblade                                          | 1        |
| 23b         | 101140      | Caster 3" Lighted Rollerblade                                  | 1        |
| 23c         | 100791      | Caster 4" Rollerblade                                          | 1        |
| 23d         | 100952      | Caster 4" Lighted Rollerblade                                  | 1        |

| Item Number | Part Number | Description                                                                                            | Quantity |
|-------------|-------------|--------------------------------------------------------------------------------------------------------|----------|
| 23e         | 100809      | Caster 4 x 1.25" Soft Roll                                                                             | 1        |
| 23f         | 100807      | Caster 5 x 1.25" Soft Roll                                                                             | 1        |
| 24a         | 004361INST  | Short Rigid Fork Streamline Assembly, Black Anodized with Instruction<br><i>Replaced 001812INST</i>    | 1        |
| 24b         | 004354INST  | Standard Rigid Fork Streamline Assembly, Black Anodized with Instruction<br><i>Replaced 000927INST</i> | 1        |
| 24c         | 004362INST  | Tall Rigid Fork Streamline Assembly, Black Anodized with Instruction<br><i>Replaced 000928INST</i>     | 1        |
| 25          | 001355      | Caster Bearing Cap Black Anodized                                                                      | 1        |
| 26          | 000422      | M6 x 56 Threaded Barrel                                                                                | 1        |
| 27          | 100667      | M6 x 12 BTN SHCS BLKZC w/Patch                                                                         | 2        |
| 28          | 100792      | Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer                                             | 2        |

# Color Anodizing Samples

