


Rogue Style Transit Backrest Option on Rogue ALX

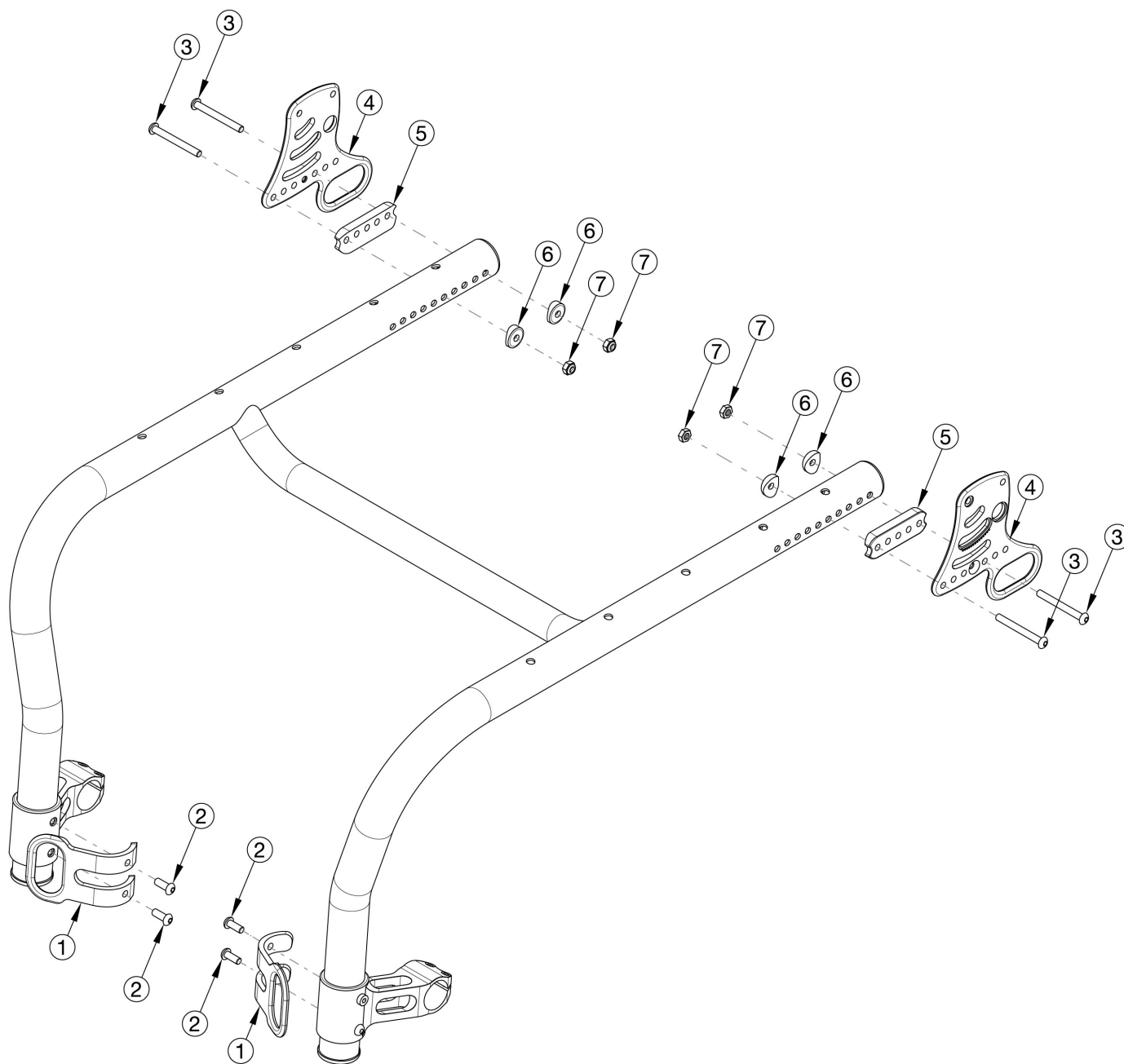

Utilize if chair has Rigid Locking Fold Down Back

Item Number	Part Number	Description	Quantity
1 - 7	117931	Transit Bracket Retrofit Kit for Rigid Locking Fold Down Backrest, Rogue ALX <i>Not available with Heavy Duty Option. This kit is a complete retrofit, containing a pair of front and rear transit brackets and hardware.</i>	1
1 - 7	117931_ _ _	Transit Bracket Retrofit Kit for Rigid Locking Fold Down Backrest, Color Accent, Rogue ALX	1

Item Number	Part Number	Description	Quantity
		<i>Not available with Heavy Duty Option. This kit is a complete retrofit, containing a pair of front and rear transit brackets and hardware.</i>	
2	101708	M6 x 14 BTN SHCS BLKZC w/Patch	4
3a	003232	M6 x 55 BHCS Blk Zc <i>Used when bolt goes through the tower and frame only. Also used when bolt goes through the frame, back bracket and back saddles.</i>	2
3b	107138	M6 x 65, BHCS, P/THD BLZ <i>Used when bolt goes through the tower, frame, back bracket and back saddle</i>	4
3c	101781	M6 x 50mm Partial Thread BHCS Blk ZC <i>Used when bolt goes through the frame, back bracket and back saddle</i>	4
4a	115552	Back Plate, Transit, Right, Black	1
4a	115552_ _ _	Color Accent Back Plate, Transit, Right	1
4b	115553	Back Plate, Transit, Left, Black	1
4b	115553_ _ _	Color Accent Back Plate, Transit, Left	1
5	003715	Back Saddle 1 3/8" Tube	2
6	003716	Saddle 1 3/8" Tube	4
7	100658	M6 Nylock Nut Blk Zn	6