
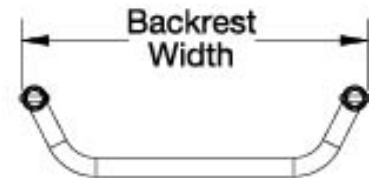


# Rogue ALX Adjustable Height Backrest with Adjustable Height Rigidizer Bar (Formerly Tsunami)

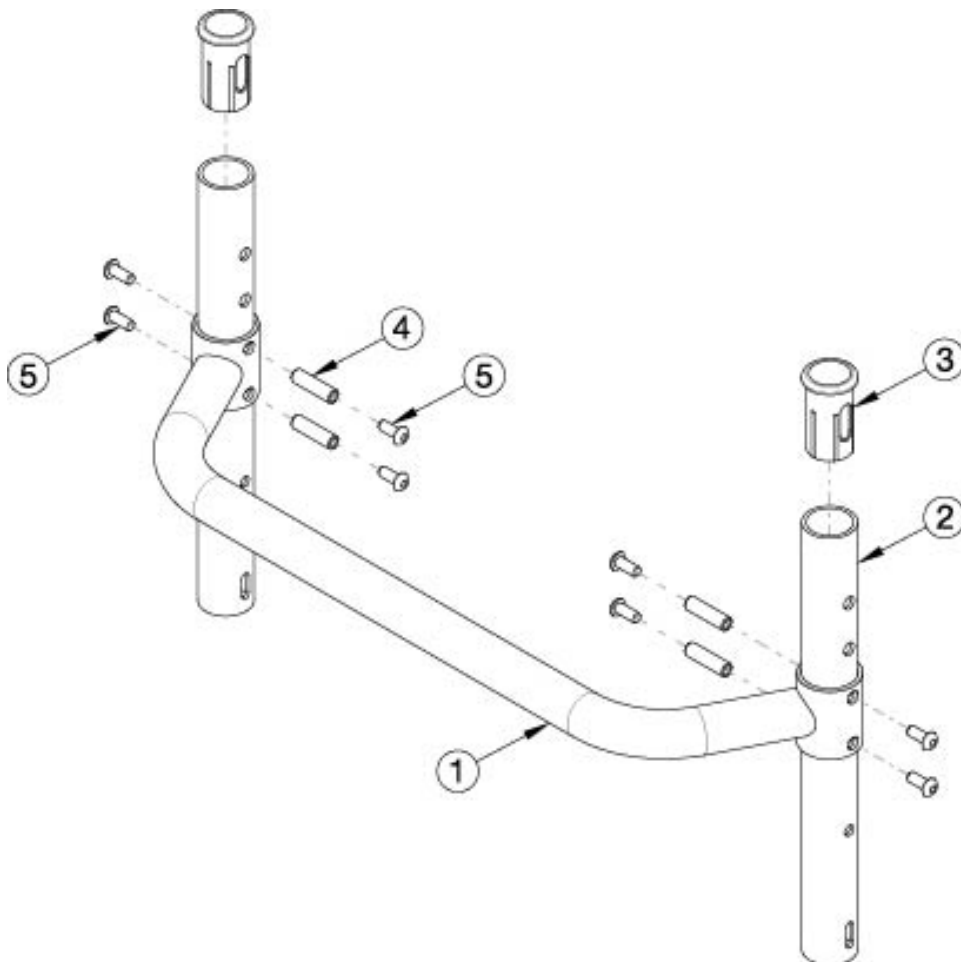

Backrest	Width
Backrest 10"	10.3"
Backrest 11"	11.3"
Backrest 12"	12.3"
Backrest 13"	13.3"
Backrest 14"	14.3"
Backrest 15"	15.3"
Backrest 16"	16.3"
Backrest 17"	17.3"
Backrest 18"	18.3"
Backrest 19"	19.3"
Backrest 20"	20.3"
Backrest 21"	21.3"
Backrest 22"	22.3"

Item Number	Part Number	Description	Quantity
1a	000525	Adjustable Height Rigidizer Bar 10"	1
1b	000526	Adjustable Height Rigidizer Bar 11"	1
1c	000527	Adjustable Height Rigidizer Bar 12" Wide	1
1d	000528	Adjustable Height Rigidizer Bar 13" Wide	1
1e	000185	Adjustable Height Rigidizer Bar 14" Wide	1
1f	000186	Adjustable Height Rigidizer Bar 15" Wide	1
1g	000187	Adjustable Height Rigidizer Bar 16" Wide	1
1h	000188	Adjustable Height Rigidizer Bar 17" Wide	1
1i	000189	Adjustable Height Rigidizer Bar 18" Wide	1

Item Number	Part Number	Description	Quantity
1j	000190	Adjustable Height Rigidizer Bar 19" Wide	1
1k	000191	Adjustable Height Rigidizer Bar 20" Wide	1
1l	000529	Adjustable Height Rigidizer Bar 21"	1
1m	000530	Adjustable Height Rigidizer Bar 22"	1
2a	000177	Rigid Backrest Tube, 9.25" <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	2
2b	109099	Rigid Backrest Tube, 8.25" <i>Used with fold down push handles with a short or medium back height range</i>	2
2a, 3	110663	Rigid Backrest Tube, 9.25" with Sleeve <i>Used with standard push handles, low profile push handles or omit push handles in any back height range. Also used with fold down push handles with a tall back height range.</i>	2
2b, 3	110664	Rigid Backrest Tube, 8.25" with Sleeve <i>Used with fold down push handles with a short or medium back height range</i>	2
3	000034	Sleeve 3/4" x 1 1/2"	2
4	000073	M5 x 24 Threaded Barrel	4
5	103524	M5 x 10 BTN SHCS BLK ZN	8