


(Discontinued 3) Focus CR Hand Tilt Mechanism Stroller Handle Back

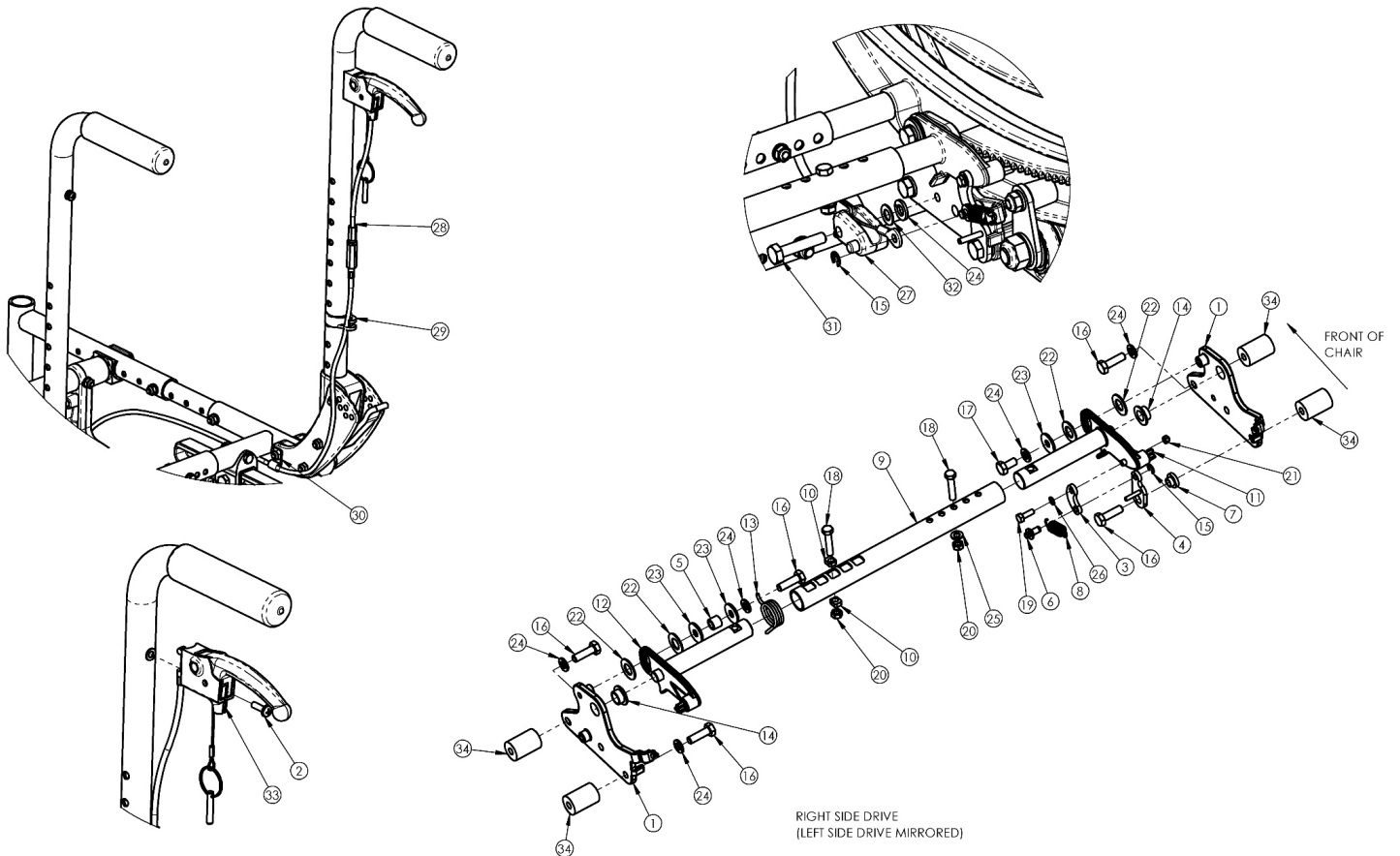

This Focus CR Hand Tilt is discontinued and was used on chairs starting with serial numbers FCR0016745 and FCHD002213 prior to a March 5th, 2018 invoice date.

Item Number	Part Number	Description	Quantity	Retail
1a, 14	500872	Tilt Lock Mount Bracket w/ Bearing Kit, Right, Focus, Fixed Height	1	\$19.27
1b, 14	500873	Tilt Lock Mount Bracket w/ Bearing Kit, Left, Focus, Fixed Height	1	\$19.27
2, 33	110139	Lever Assembly with Lanyard Vertical Tilt	1	\$34.00
2	100734	M5 x 16 PHMS ZC	1	\$1.58
3	003247	Link, Drive, Tilt	1	\$12.87
4a	002084	Toggle, Drive, Tilt, R	1	\$11.28
4b	002085	Toggle, Drive, Tilt, L	1	\$20.45
5	002086	Spacer, .38ODx.24IDx.31LG	1	\$2.42

Item Number	Part Number	Description	Quantity	Retail
6	003364	Pin, Toggle, Tilt	1	\$7.25
9a	002094	Growth Strut, Tilt Lock, Small	1	\$33.62
9b	002095	Growth Strut, Tilt Lock, Med	1	\$35.72
9c	002096	Growth Strut, Tilt Lock, Large	1	\$37.82
10	002097	Insert, Drive, Tilt	2	\$3.86
11	002411	Cog Lever, Tilt Lock, R	1	\$38.59
12	002412	Cog Lever, Tilt Lock, L	1	\$38.59
13a	001109	Spring Tilt Lock Right TILT <i>Used with left side assembly</i>	1	\$4.71
13b	001110	Spring Tilt Lock Left TILT <i>Used with right side assembly</i>	1	\$4.71
16a	101823	M6 x 20 HHCS Blk Zn <i>Standard weight capacity requires qty 5; Heavy Duty requires qty of 1</i>	5	\$1.05
16b	100674	M6 x 45 CLS10.9 DIN931 HHCS ZC <i>Used with Heavy Duty option/spacer</i>	3	\$1.58
16c	101826	M6 x 50 HHCS Blk Zn <i>Used on drive side tilt with Heavy Duty option/spacer</i>	1	\$2.10
17	101822	M6 x 12 HHCS BLKZC w/Patch	1	\$1.05
18	101821	M5 x 25 HHCS Blk Zn	2	\$1.05
19	102783	M3 x 12 HHCS, CL8.8, Blk Zn	1	\$1.05
20	100657	M5 Nylock Nut Blk Zn	2	\$0.53
21	102784	Nut, Nylock, M3, Blk	2	\$1.05
22	102082	Washer Nylon .406IDX.75ODX.040	4	\$0.56
23	101853	M6 Washer Blk Zn	3	\$0.53
24	101835	M6 Lock Washer Blk Zn	6	\$0.53
25	101856	Flat Washer, M5, Blk Zn	1	\$0.53
26	102785	Flat Washer, M3, Blk Zn	1	\$0.53
27	003387	Cable Mount, Tilt	1	\$3.68
28a	002100	Cable, Tilt, Short <i>Used with short back height range</i>	1	\$107.16

Item Number	Part Number	Description	Quantity	Retail
28b	002101	Cable, Tilt, Med <i>Used with tall back height range</i>	1	\$107.16
29	001117	Cable Clip 1"	1	\$3.83
30	001239	P-Clip 3/16"	1	\$6.30
31	101824	M6 x 30 HHCS Blk Zn	1	\$2.10
32	102803	Washer Flat .250 x .500 x .027 HX BLZ	1	\$1.58
34	001405	Spacer, .69OD x .24ID x .1.01Lg <i>Only used with Heavy Duty Option</i>	4	\$26.05