
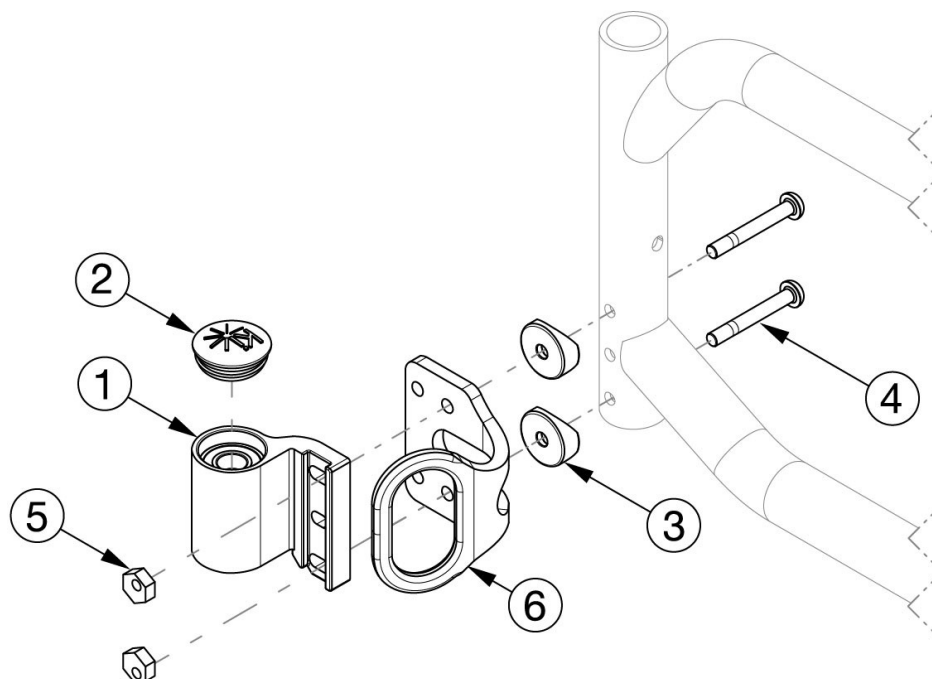


## Catalyst E Caster Housing - Super Low

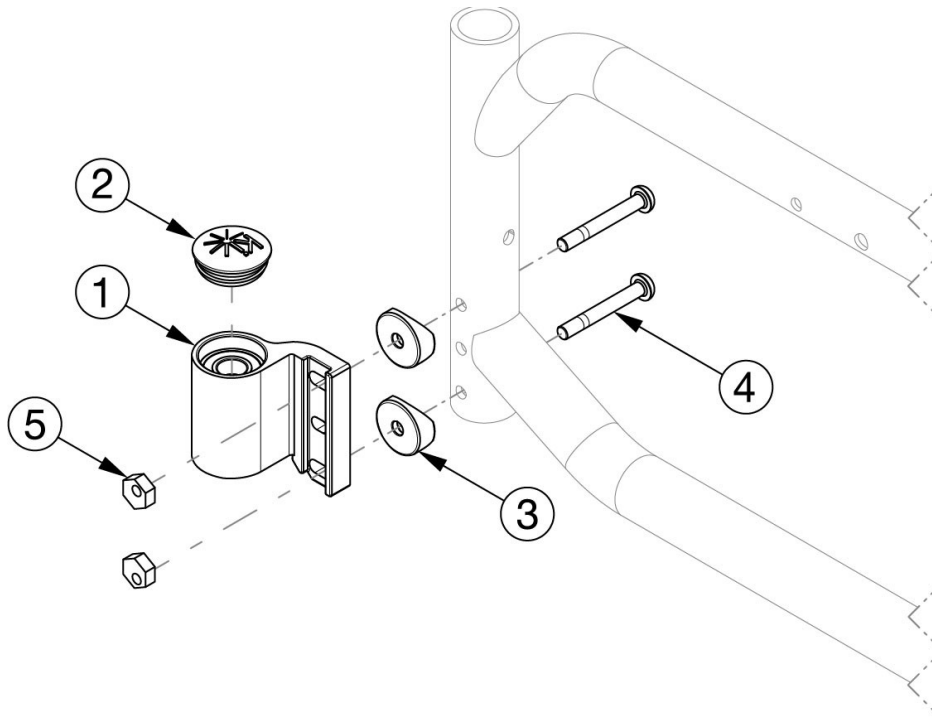

Reference order form for detailed front seat height matrix. Please Note: The shown orientation of the caster housing above is in the "Forward" position.

| Item Number | Part Number | Description                                                   | Quantity |
|-------------|-------------|---------------------------------------------------------------|----------|
| 1a, 2-5     | 115819      | Super Low Caster Housing Assembly, Black, Left, No Transit    | 1        |
| 1b, 2-5     | 115820      | Super Low Caster Housing Assembly, Black, Right, No Transit   | 1        |
| 1a, 2-6     | 115822      | Super Low Caster Housing Assembly, Black, Left, with Transit  | 1        |
| 1b, 2-6     | 115823      | Super Low Caster Housing Assembly, Black, Right, with Transit | 1        |
| 1a, 1b, 2-5 | 115821      | Super Low Caster Housing Assembly, Black, PAIR, No Transit    | 1        |
| 1a, 1b, 2-6 | 115824      | Super Low Caster Housing Assembly, Black, PAIR, with Transit  | 1        |
| 1a          | 111982      | Super Low Caster Housing Assembly, Left                       | 1        |
| 1b          | 111983      | Super Low Caster Housing Assembly, Right                      | 1        |
| 2           | 102005      | Ki Caster Housing Cap                                         | 1        |
| 3           | 000056      | Saddle 1 1/8"                                                 | 2        |
| 4           | 200367      | M6 x 45 Custom Shouldered BHCS                                | 2        |
| 5           | 100745      | Eccentric nut (threaded)                                      | 2        |
| 6           | 000426      | Front Transit Bracket CAT                                     | 1        |