


Straight No Push Handle and Caregiver Height Adjustable Stroller Handles

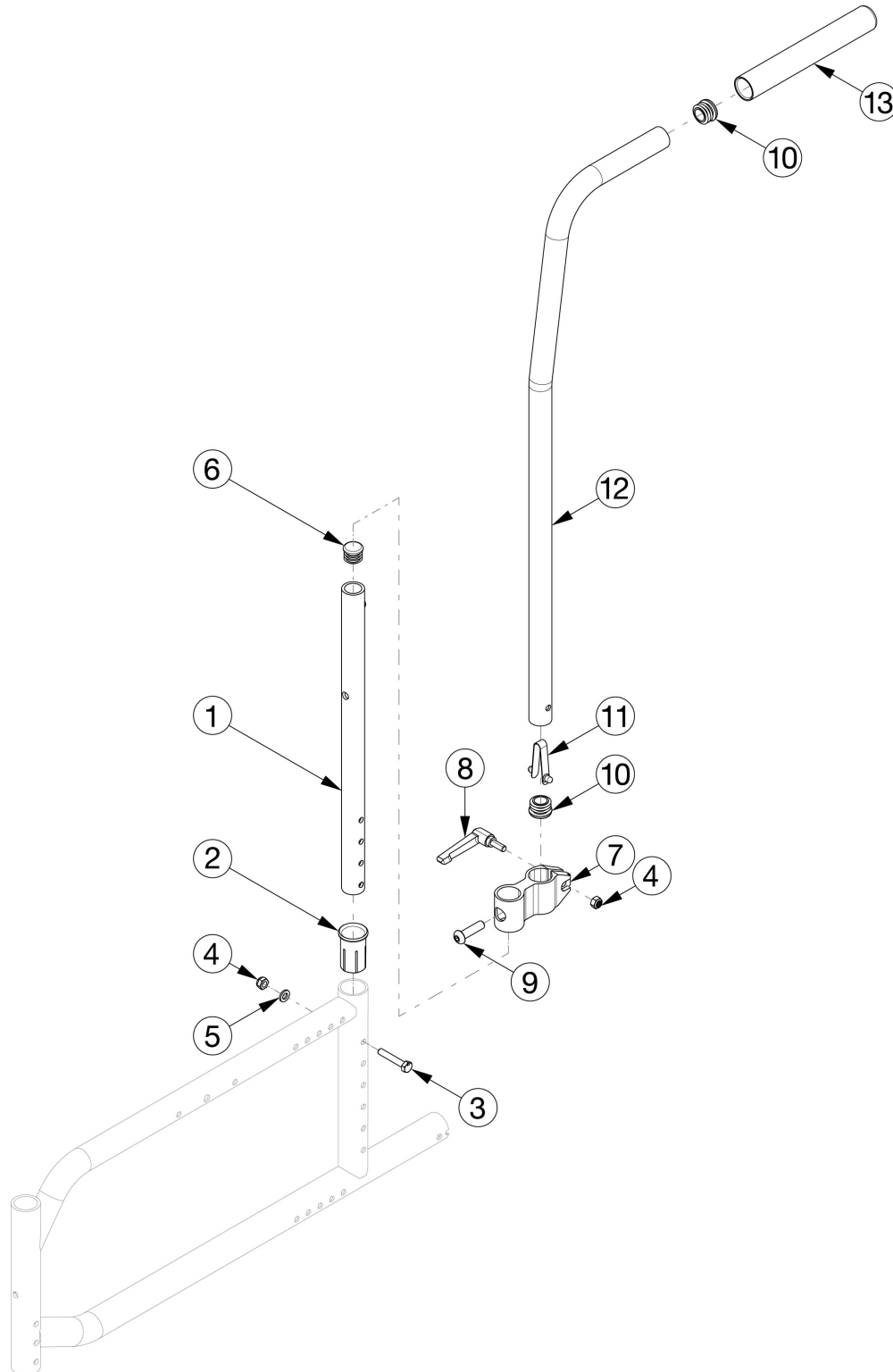

Not compatible with BTX and FXT Axiom Back Hardware

Item Number	Part Number	Description	Quantity
1a, 6	115853	Straight NPH Back Assembly - Short	1
1b, 6	115854	Straight NPH Back Assembly - Medium	1
1c, 6	115855	Straight NPH Back Assembly - Tall	1
2-4	115852	Backrest Mount Hardware	1
4, 7-13	115837	Caregiver HA Stroller Assembly (Clamp Assy + Stroller Handle)	1
10-13	115838	Caregiver HA Stroller Handle	1
4, 7-9	115839	Caregiver HA Stroller - Clamp + Hardware	1
2	111965	Frame Sleeve, Angle Adjustable Backrest	1
3	100860	M6 X 35 Hhcs P/Thd Zc	1
4	100658	M6 Nylock Nut Blk Zn	2
5	100746	M6 Flat Washer Blk Zn	1
6	100638	Tube End Plug 3/4"	1
7	112423	Caregiver Height Adjustable Stroller Handle Clamp	1
8	102734	Stroller Handle Clamp Handle (16mm) Rigid/Flip/Tyke	1
9	103551	BHCS, M8 x 1.25 x 30 BLZ	2
10	100803	Tube End Plug 7/8"	1
11	100534	Detent Button Lower	1