


CR45 Rear Anti-Tips

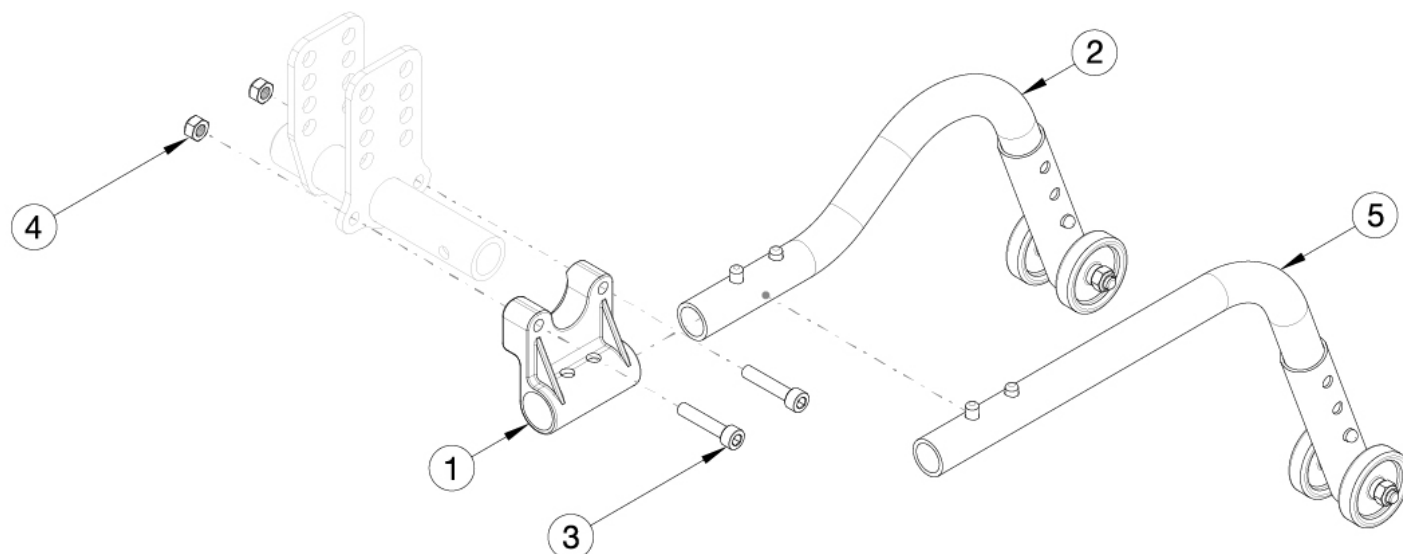

Item Number	Part Number	Description	Quantity
1, 2a, 3, 4	116478	Anti-Tip Assembly, 12" Wheel with Receiver, CR45	1
1, 2b, 3, 4	116479	Anti-Tip Assembly, 16" Wheel with Receiver, CR45	1
1, 2c, 3, 4	116480	Anti-Tip Assembly, 20" Wheel with Receiver, CR45	1
1, 2d, 3, 4	116481	Anti-Tip Assembly, 22" - 24" Wheel with Receiver, CR45	1
1	112496	Anti-Tip Receiver, CR45	2
2a	112499	CR45 Low Long Anti-Tip Assembly	2
2b	119680	CR45 Low Short Anti-Tip Assembly <i>Replaces 112498</i>	2
2d	119679	Anti-Tip Assembly, 22-24", Liberty, Arc, Cr45	2
3	102627	M6 X 30 SHCS BLK ZC	4
4	100658	M6 Nylock Nut Blk Zn	4