


Rogue XP Standard Camber Tube

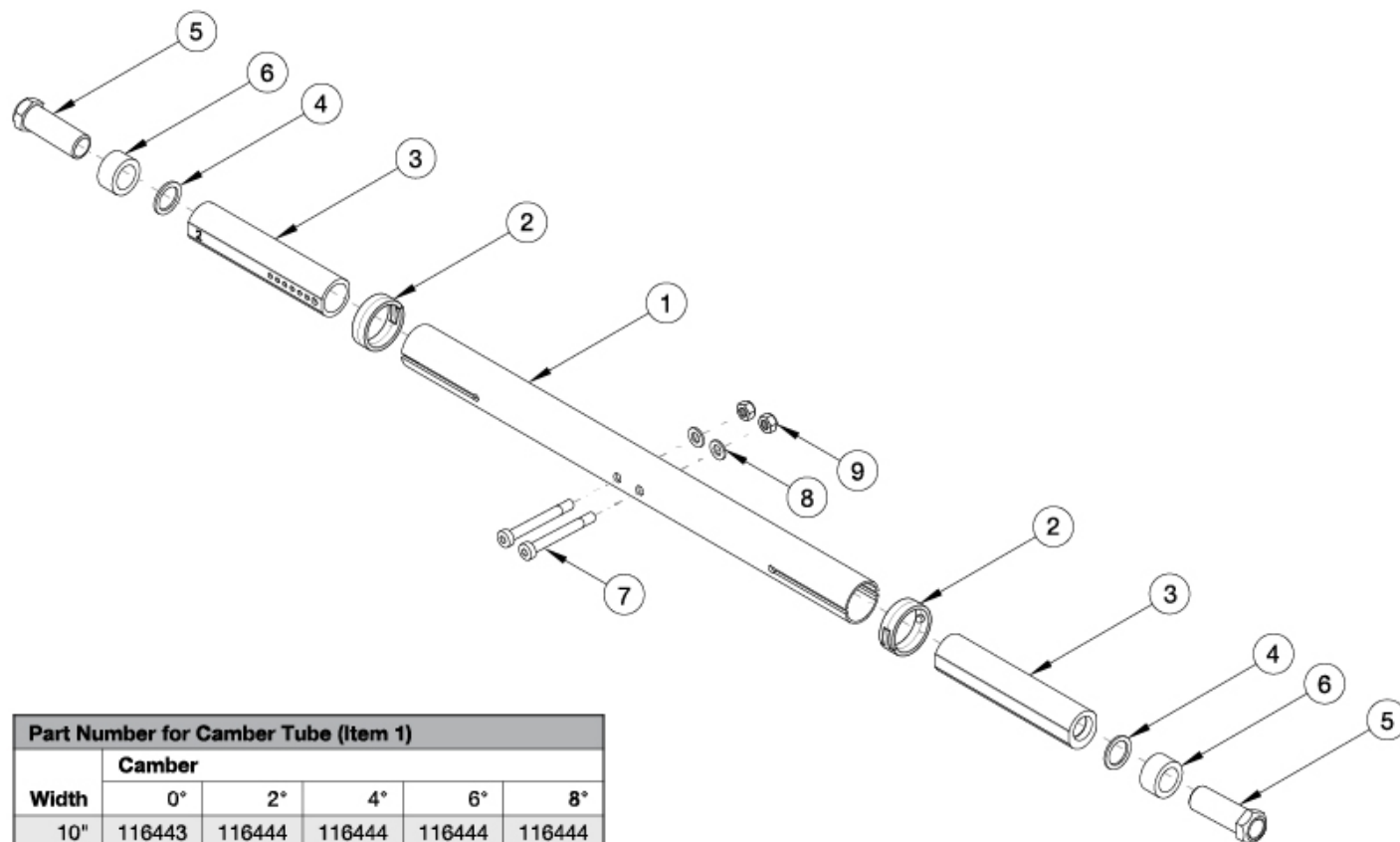

Part Number for Camber Tube (Item 1)					
	Camber				
Width	0°	2°	4°	6°	8°
10"	116443	116444	116444	116444	116444
11"	116444	116445	116445	116445	116445
12"	116445	004165	004165	004165	004165
13"	004165	004166	004166	004166	004166
14"	004166	004167	004167	004167	004167
15"	004167	004168	004168	004168	004168
16"	004168	004169	004169	004169	004169
17"	004169	004170	004170	004170	004170
18"	004170	004171	004171	004171	004171

If One Arm Drive is present, refer to the corresponding One Arm Drive parts manual page. Used on chairs invoiced starting on 3/12/25

Item Number	Part Number	Description	Quantity
1a	116443	9" WIDE CAMBERTUBE BLACK	1
1b	116443_ _ _	9" Wide Camber Tube Color Anodized/Color Accent	1
1c	116444	10" WIDE CAMBERTUBE BLACK	1
1d	116444_ _ _	10" Wide Camber Tube Color Anodized/Color Accent	1
1e	116445	11" Wide Camber Tube, Black Anodized	1
1f	116445_ _ _	11" Wide Camber Tube, Color Anodized/Color Accent	1

Item Number	Part Number	Description	Quantity
1g	004165	12" Wide Camber Tube, Black Anodized	1
1h	004165_ _ _	12" Wide Camber Tube, Color Anodized/Color Accent	1
1i	004166	13" Wide Camber Tube, Black Anodized	1
1j	004166_ _ _	13" Wide Camber Tube, Color Anodized/Color Accent	1
1k	004167	14" Wide Camber Tube, Black Anodized	1
1l	004167_ _ _	14" Wide Camber Tube, Color Anodized/Color Accent	1
1m	004168	15" Wide Camber Tube, Black Anodized	1
1n	004168_ _ _	15" Wide Camber Tube, Color Anodized/Color Accent	1
1o	004169	16" Wide Camber Tube, Black Anodized	1
1p	004169_ _ _	16" Wide Camber Tube, Color Anodized/Color Accent	1
1q	004170	17" Wide Camber Tube, Black Anodized	1
1r	004170_ _ _	17" Wide Camber Tube, Color Anodized/Color Accent	1
1s	004171	18" Wide Camber Tube, Black Anodized	1
1t	004171_ _ _	18" Wide Camber Tube, Color Anodized/Color Accent	1
2	111808	Detent Clip For Camber Tube	2
3a	004174	Camber Plug 0 Degrees	2
3a, 4, 5	004355	Camber Plug 0 Degrees Assembly	2
2, 3a, 4, 5	108093	Camber Plug 0 Degrees Assembly with Camber Tube Clip <i>This Kit is for one side</i>	1
3b	004175	Camber Plug 2 Degrees	2
3b, 4, 5	004356	Camber Plug 2 Degrees Assembly	2
2, 3b, 4, 5	108094	Camber Plug 2 Degrees Assembly with Camber Tube Clip <i>This Kit is for one side</i>	1
3c	004176	Camber Plug 4 Degrees	2
3c, 4, 5	004357	Camber Plug 4 Degrees Assembly	2
2, 3c, 4, 5	108095	Camber Plug 4 Degrees Assembly with Camber Tube Clip <i>This Kit is for one side</i>	1
3d	004177	Camber Plug 6 Degrees	2
3d, 4, 5	004358	Camber Plug 6 Degrees Assembly	2

Item Number	Part Number	Description	Quantity
2, 3d, 4, 5	108096	Camber Plug 6 Degrees Assembly with Camber Tube Clip <i>This Kit is for one side</i>	1
3e	004178	Camber Plug 8 Degrees	2
3e, 4, 5	004359	Camber Plug 8 Degrees Assembly	2
2, 3e, 4, 5	108097	Camber Plug 8 Degrees Assembly with Camber Tube Clip <i>This Kit is for one side</i>	1
4	004180	Washer .635 ID x .895 OD x .06 T Steel Blk Zc	2
5	004179	Axle Sleeve 16mm x 1.0 X 2" Long	2
6	004182	Spacer Aluminum .635 Id X .945 Od X .500 T <i>Used as needed. Increases rear wheel spacing</i>	2
7	101123	M6 x 45 Low SHCS BLK ZC <i>Used when Dynamic 5th Wheel is present</i>	2
8	100746	M6 Flat Washer Blk Zn <i>Used when Dynamic 5th Wheel is present</i>	2
9	100658	M6 Nylock Nut Blk Zn <i>Used when Dynamic 5th Wheel is present</i>	2